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**IRRITANTE**

MI0004 – REV. 04

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## INSTRUCTIONS FOR USE FOR PROFESSIONALS

Street: Ottokar Doerffel, 1112  
Neighborhood: Atiradores. City: Joinville.  
State: SC. Country: Brazil  
CEP: 89203-212  
CNPJ: 17.910.389/0001-05  
BRAZILIAN INDUSTRY  
E-MAIL: vendas@phsdo brasil.com.br  
Health Ministry's Operating License: X401M5619X7L(8.11992.7)  
Registration at ANVISA nº 81199270006  
Technical manager: ÉLEN ALVES LOOS - CRQ/SC N. 13303733

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**Product's commercial name:** POTENZA BIANCO CARBAMIDE PEROXIDE PF 10%, 16%, 22%.

**Product's technical name:** DENTAL WHITENING AGENTS.

**READ CAREFULLY ALL THE INFORMATION OF THIS DOCUMENT BEFORE USING THE PRODUCT. KEEP THIS DOCUMENT UNTIL TOTAL CONSUMPTION OF THE PRODUCT OR UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

## **BASIC COMPOSITION:**

Carbamide peroxide, thickeners, deionized water, desensitizers and humectant.

## **PRODUCT'S DESCRIPTION:**

POTENZA BIANCO PF is a bleaching agent for home use, supervised by the dentist, based on carbamide peroxide at 10%, 16% or 22%. The product has an aspect of thixotropic colorless gel and is presented in syringes that are ready to use. Its formulation contents potassium nitrate and sodium fluoride that act dually desensitizing the teeth, providing more comfort to the patient. Besides, the product has high content of water and has the humectant Glicerol, that prevents dental dehydration.

## **PRODUCT'S INDICATION:**

POTENZA BIANCO PF is indicated for home bleaching, supervised by the dentist. It can be used for vital and non vital teeth (externally) that present shade alterations. The product is also indicated for mild cases of fluorosis or tetracycline staining.

## **PRECAUTIONS:**

- Keep away from sunlight;
- The contact of the product to eyes and/or skin can cause burning;
- Ingestion of high quantities of the product can cause nausea. Professional must tell the patient not to swallow product. If patient accidentally swallows high volume of the product, professional must take him to medical support with this instructions document;
- This product is not for injection;
- Do not reuse the package;
- If there is any suspect of allergy, discontinue using the product immediately;
- If the patient reports pre-existing hypersensitivity during his daily routine (i.e. consumption of cold liquids, cold air or pressure, etc.) and there is no obvious cause that leads the patient to this discomfort, apply the desensitizing (Potenza Esente 0,2% + Potenza Esente GLU-HEMA) previously to bleaching;

- In cases of important gingival recession, restore the region with provisional material (i.e. GIC) before using bleaching agent;
- Dental surfaces that will receive the bleaching gel must be clean (absence of calculus or plaque). If necessary, dental prophylaxis must be performed with minimum of 5 days before bleaching;
- It is essential for the bleaching gel to contact soft tissue as minimum as possible, because it can lead to irritation in different magnitudes according to period of exposure, concentration of the product and gingival phenotype of the patient. Instruct patient to apply small amount of gel (half of a rice grain) on the space corresponding to each teeth. This will avoid excessive overflow through the edges of the custom tray;
- If the patient present high level of hypersensitivity while using the product, reduce bleaching gel concentration to 10% and instruct the patient to use desensitizing agent (POTENZA ESENTE 0,2%) for 10 minutes, before and after bleaching applications daily;

**IMPORTANT:**

- Dental bleaching must be supervised by a dentist;
- The efficacy of the treatment depends on the correct usage of the product;
- The efficacy of the treatment varies from one patient to another according to individual factors (i.e. type of stains, level of tissue permeability, instrinsic color of teeth, etc.).

In some cases, it is suggested to combine bleaching approaches (home and office use).

General safety instructions:

- Product handling must be performed exclusively in a dental office;
- The professional must use all the protection equipments during session, most importantly protective goggles and gloves;
- The patient must wear disposable cap, protective goggles and cloth covering his clothes;

- The carbamide peroxide inside the syringe may be under pressure if stored or transported in adverse conditions. Before opening the package, protect hands and eyes and observe if there is any leakage;
- Avoid contact of the product to skin, mucosa or clothes;
- Instruct the patient not to swallow the product. While using the tray with the gel, instruct him not to drink or eat;
- If the gel contacts skin or mucosa intraorally, clean the area abundantly with water;
- In case the gel contacts the eye, rinse abundantly with water and lead the patient to medical assistance with this instructions;
- If patient present allergic reactions, the treatment must be suspended immediately. Lead the patient to medical assistance with this instructions;
- Expired leftovers of gel must not be used. Instruct the patient to follow correctly local legislation on how to discard it properly;

## **CONTRAINDICATIONS:**

Before initiating bleaching, perform detailed anamnesis of the patient. Identify habits, general conditions, chronical and acute diseases. In the clinical exam, any transitory (A) alterations must be treated previously bleaching. Some conditions (B) contraindicate bleaching in an absolute way.

### **A) TRANSITORY CONTRAINDICATIONS** – solve before bleaching

- Caries, leaked restorations, dental cracks or fractures, periodontal diseases (i.e. gingivitis or periodontitis), presence of plaque or calculus, teeth with doubtful pulp vitality or showing non treated endodontic problems, teeth with big gingival recession.

### **B) ABSOLUTE CONTRAINDICATIONS** – do not perform bleaching

- Teeth showing amelogenesis or dentinogenesis imperfecta, teeth stricken by severe fluorosis or tetracycline staining, teeth with relevant bone loss or mobility, teeth with incomplete rhizogenesis, individuals under 15 years old, pregnant or breastfeeding women, patient treating severe disease or those with unbalanced diseases, allergic to any of the formula contents.

## **SIDE EFFECTS:**

- It is common for the patient to present a certain degree of hypersensitivity during or after the application of the whitening gel. This is due to the product's own mechanism of action. The level of discomfort varies greatly from patient to patient. The desensitizing agent Potenza Esente 0.2% should be associated with the treatment for greater patient comfort. In extreme cases, it is suggested to stop the treatment;
- The bleaching can highlight pre-existing areas of hypocalcification (hypoplasia) due to dehydration or even more intense bleaching of these areas. In most cases, these spots disappear after rehydration or remineralization of the region, which occurs within a few hours or days after using the bleaching agent. Whenever you identify the stains, suspend the treatment until they have disappeared. If the stains are persistent, discuss the continuation or not of the treatment with the patient, informing him of any need for restorative intervention;
- Some patients may have non-uniform whitening in different thirds of the teeth. This is due to a physiological issue and will hardly be changed, because in the cervical region of the teeth there is a greater thickness of dentin, which is naturally more stained than the predominant enamel in the other thirds.

## **CAUTION:**

- Respect the expiration date. Never use the product if it has expired;
- Keep out of the reach of children;
- Only for dental use;
- Before using, observe the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use. It is the professional's responsibility to check before using the product if this product is compatible and suitable for the intended use;
- Do not reuse empty packaging for other purposes;
- To carry out the disposal of the product, follow the local legislation;

## **INSTRUCTIONS FOR USE:**

Before using the product, read all items in this document. Make a detailed history and clinical examination of the patient and solve any existing problems before starting treatment (see contraindications and precautions).

### **SUGGESTION OF USAGE TIME ACCORDING TO GEL CONCENTRATION:**

**POTENZA BIANCO PF 10%:** 3 to 4 hours

**POTENZA BIANCO PF 16%:** 2 to 3 hours

**POTENZA BIANCO PF 22%:** 1 hour

Attention: The dentist must monitor the patient throughout the treatment carried out at home. Weekly return visits for clinical follow-up are suggested.

**1st Step:** Select and register the shade of the patient's teeth using a shade guide. Photographs are suggested;

**2nd Step:** obtain the patient's plaster models through impressions with alginate and common plaster. Cut out the models in a "horseshoe" shape, removing the palatal area (upper model) and the tongue area (lower model);

**3rd Step:** With the model already cut out, submit it to a vacuumforming machine. In this machine, place a 1.25mm EVA plate (Potenza Stampo) and turn on the heating. When you notice a 2cm bulging (plastification), lower the plate against the model to activate the vacuum and conform the tray;

**4th Step:** Allow the plate to cool down until it reaches room temperature. Then, separate de model from the tray and cut out the individual tray as desired, with small scissors. It is suggested for the tray edges to cover up to 1mm of the gingival margin. Remove any sharp edges that might bother gingiva or lips;

**Step 5:** Instruct the patient to use the tray. The whitening gel must be applied directly from the syringe, with the aid of the applicator tip, in the spaces of each tooth to be whitened (usually from 2nd premolar to another). The inner wall of the tray that corresponds to the buccal surface of the tooth must receive an amount of gel with a volume proportional to half of a grain of rice; deposited in the center of the space;

**6th Step:** Instruct the patient to fit the loaded tray to the dental arch. Any excess that overflows the edges must be removed with moistened fingers;

**7th Step:** After the recommended use time, instruct the patient to remove and wash the tray under running water at room temperature. After removing all the whitening gel and saliva residues, the tray should be dried with a paper towel and stored in an appropriate case, in a place free from humidity, heat and direct sunlight.

**NOTE:** If the patient reports that they are swallowing gel during use or their gingiva show signs of inflammation, consider to cut lower the tray edges and instruct them to apply a smaller amount of gel in the tray.

#### **CONSERVATION AND STORAGE:**

- Keep the product in it's original package;
- Protect the product from direct sunlight.
- Keep the product in temperatures between 0 °C a 30 °C (32°F to 86°F).

#### **TRANSPORT CONDITIONS:**

- The product must be transported in temperatures

#### **EXPIRY DATE:**

- TWO YEARS AFTER MANUFACTURING DATE.

**SOLD UNDER PRESCRIPTION – ACCORDING TO RDC N° 6 OF 2015.**

**PRODUCT FOR DENTAL OFFICE USE ONLY. TO BE USED ONLY BY DENTAL PROFESSIONALS.**

**KEEP THE PACKAGE CLOSED WHILE NOT IN USE.**

**FOLLOW THE INSTRUCTIONS  
FOR USE CORRECTLY**

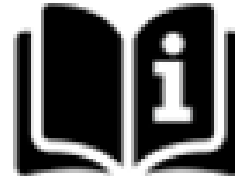
**INTENDED FOR  
PRESCRIPTION USE  
ONLY**

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**Product's commercial name:** POTENZA BIANCO CARBAMIDE PEROXIDE PF 10%, 16%, 22%.

**Product's technical name:** DENTAL WHITENING AGENTS.

**READ CAREFULLY ALL THE INFORMATION OF THIS DOCUMENT BEFORE USING THE PRODUCT. KEEP THIS DOCUMENT UNTIL TOTAL CONSUMPTION OF THE PRODUCT OR UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

## **BASIC COMPOSITION:**

Carbamide peroxide, thickeners, deionized water, desensitizers and humectant.

## **PRODUCT'S DESCRIPTION:**

POTENZA BIANCO PF is a bleaching agent for home use, supervised by the dentist, based on carbamide peroxide at 10%, 16% or 22%. The product has an aspect of thixotropic colorless gel and is presented in syringes that are ready to use. Its formulation contains potassium nitrate and sodium fluoride that act dually desensitizing the teeth, providing more comfort to the patient. Besides, the product has high content of water and has the humectant Glicerol, that prevents dental dehydration.

## **PRODUCT'S INDICATION:**

POTENZA BIANCO PF is indicated for home bleaching, supervised by the dentist. It can be used for vital and non vital teeth (externally) that present shade alterations. The product is also indicated for mild cases of fluorosis or tetracycline staining.

## **PRECAUTIONS:**

- This product should only be performed under the supervision of a dentist;
- Use the product exactly for the period recommended by your dentist;
- While using trays, do not ingest food or drink;
- Avoid smoking during treatment. To avoid hypersensitivity, avoid consumption of acidic foods/drinks;
- After ingesting strongly colored food/drinks, rinse the mouth copiously for several times until the substance is eliminated from the mouth;
- If you identify dental hypersensitivity, report it to your dentist;
- In case of allergic reactions, stop the treatment immediately and contact your dentist;
- Non-injectable product;
- Do not reuse the packaging.

## **CONTRAINDICATIONS:**

Some oral problems (caries, gingivitis, endodontic problems, etc.) require professional treatment prior to bleaching. These problems are identified by your dentist during the initial clinical examination visit.

In addition to the specific problems of teeth or their supporting tissues - which must be diagnosed by the professional - there are still problems or conditions that cannot be treated and therefore contraindicate treatment:

- Gestational or lactation period;
- Patients under 15 years of age;
- Patients allergic to any of the components of the formulation;
- Patients being treated for serious illnesses or patients with decompensated serious illnesses;

## **SIDE EFFECTS:**

- It is common to have a degree of hypersensitivity during or after the application of the whitening gel. This is due to the product's own mechanism of action. The level of discomfort varies greatly from patient to patient. The professional may recommend the use of a desensitizing agent (Potenza Esente 0.2%) which should be associated with the treatment for greater comfort. At the first sign of discomfort, report it to your patient. If the intensity is high, stop the treatment;

- Whitening can highlight areas of hypocalcification (white spots) that are pre-existing on teeth due to dehydration or even more intense whitening of these areas. In most cases, these spots were virtually invisible to the naked eye before bleaching began. Its behavior varies, but the stains usually disappear after rehydration or remineralization of the region, which occurs within a few hours or days after using the bleach. If you identify any stain on your teeth, discontinue the treatment and contact your dentist;

- Some patients may have non-uniform whitening in different thirds of the teeth. This is for a physiological reason. If you identify this condition, contact your dentist;

- Ingestion of large amounts of the product may cause nausea and/or irritation of the upper gastrointestinal tract (mouth, pharynx, esophagus and stomach). Reduce the volume of product used in trays and inform your dentist;

- The whitening gel has the potential to irritate the gums. If you notice redness, burning or bleeding, reduce the volume of product used in trays and inform your dentist.

## **CAUTION:**

- Respect the expiration date. Never use the product if it has expired;
- Keep out of the reach of children;
- Only for dental use;
- Before using, observe the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use. It is the professional's responsibility to check before using the product if this product is compatible and suitable for the intended use;
- Do not reuse empty packaging for other purposes;
- To carry out the disposal of the product, follow the local legislation;

## **INSTRUCTIONS FOR USE:**

Before using the product, read all items in this document. Please resolve all doubts with your dentist before starting treatment. Make sure the process is conducted under the professional's full supervision.

## **SUGGESTION OF USAGE TIME ACCORDING TO GEL CONCENTRATION:**

**POTENZA BIANCO PF 10%:** 3 to 4 HOURS

**POTENZA BIANCO PF 16%:** 2 to 3 HOURS

**POTENZA BIANCO PF 22%:** 1 HOUR

**\*NOTE:** This chart is for information only. Follow your dentist's recommendations.

**1st Step:** Perform oral hygiene (brushing and flossing) and prepare the individual tray for use (clean and dry). To do this, remove the syringe cap from the product and fit the applicator tip. With this, gently push the product syringe plunger until the material begins to dispense. Apply a small dose of the product, with a volume equivalent to half a grain of rice, in the space of each tooth to be whitened.

**2nd Step:** Place the tray on the teeth so that it fits perfectly over them. The gel content that eventually leaks into the gums should be removed with moistened fingers.

**3rd Step:** Let the product act for the time indicated by your dentist. During this period, do not take food or drink.

**4th Step:** After the period, remove the tray, wash it with running water at room temperature and dry it. So store away from direct sunlight, heat and moisture. Rinse your mouth several times until the whitening gel is completely removed and brush your teeth after 30 minutes of using the product.

**NOTE:** Treatment should be conducted as recommended by your dentist. Any questions should be directed to him.

#### **CONSERVATION AND STORAGE:**

- Keep the product in it's original package;
- Protect the product from direct sunlight;
- Keep the product in temperatures between 0°C a 30 °C (32°F to 86°F).

#### **TRANSPORT CONDITIONS:**

- The product must be transported in temperatures between -5 °C a 50 °C (23°F to 122°F).
- Protect the product from direct sunlight;

#### **MANUFACTURING DATE:**

- REFER TO PACKAGE.

#### **EXPIRY DATE:**

- TWO YEARS AFTER MANUFACTURING DATE.

**SOLD UNDER PRESCRIPTION – ACCORDING TO RDC N° 6 OF 2015.**

**PRODUCT FOR DENTAL OFFICE USE ONLY. TO BE USED ONLY BY DENTAL PROFESSIONALS.**

**KEEP THE PACKAGE CLOSED WHILE NOT IN USE.**

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