# Potenza

## ORTHOBLUE

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**PHS DO BRASIL LTDA**

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# Potenza

## ORTHOBLUE

**INSTRUCTIONS FOR USE FOR PROFESSIONALS**

**PHS DO BRASIL LTDA**

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 81199279025

**CINTERQUAL – Commerce Solutions**

**International Lda**; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

**Product Trade Name:** POTENZA ORTHOBLUE

**Alternative Trade Name:** LIGHT-CURABLE OCCLUSAL BUILD-UP RESIN

**Product Technical Name:** DENTAL CEMENTS

**PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.**

**KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

**BASIC COMPOSITION:**

Methacrylate-based monomers (including dimethacrylates such as Bis-GMA and UDMA), inorganic fillers, catalysts, pigments, and fluoride.

**PRODUCT DESCRIPTION:**

POTENZA ORTHOBLUE is a blue-colored, light-curable resin with medium viscosity and static behavior.

**PRODUCT INDICATION:**

POTENZA ORTHOBLUE is indicated for temporary occlusal build-ups during orthodontic treatment or rehabilitation procedures. It is also indicated for cementation of orthodontic bands.

**PRECAUTIONS:**

- Incomplete polymerization of the product or excessive contact time during the pre-curing phase may lead to mild gingival irritation;
- Protect the resin from ambient light;
- Do not apply the material under operatory light, as this may cause premature curing;
- Avoid contact with skin and eyes;
- Use gloves when handling the product;
- Use protective eyewear during handling.

**CONTRAINDICATIONS:**

- Do not use in patients with allergy to any component of the formulation.

**POSSIBLE SIDE EFFECTS:**

- The product contains methacrylate monomers (such as Bis-GMA and UDMA), which may cause allergic reactions or mild irritation in sensitive patients;
- Discontinue use in case of allergic reactions;
- In case of allergic reaction, discontinue use and seek medical advice. Notify the manufacturer and keep these Instructions for use available.

**WARNING:**

- Observe the expiration date;
- Do not use after the expiration date;
- Keep out of reach of children;
- For dental use only;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is intended exclusively for dental use and must be handled according to the instructions for use;

- It is the responsibility of the professional to verify, prior to use, that the material is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;
- This product is supplied in a multi-dose package. To prevent microbial contamination, infection control measures must be adopted during use, including the use of disposable applicator tips, with mandatory replacement for each patient, and the non-reuse of any dispensed material;
- Contact of the product or its packaging with potentially contaminated surfaces must be avoided in order to prevent cross-contamination between patients;
- After opening, the product must be kept properly closed and stored according to the recommended conditions. As a light-sensitive material, it must be protected from exposure to ambient light in order to prevent premature polymerization and potential impairment of performance;
- Improper handling after opening, including failures in contamination control or inadequate storage, may compromise the adhesive properties of the cement and result in unsatisfactory clinical performance;
- Changes in the physical characteristics of the product, such as consistency, texture, or polymerization behavior, may indicate compromised quality, and its use is not recommended under such conditions;
- For safe disposal of the medical device after use, follow applicable local regulations.

## **INSTRUCTIONS FOR USE:**

### **Cementation of Orthodontic Bands:**

**Step 1:** To increase band retention, apply air abrasion to its internal surface;

**Step 2:** Keep the tooth receiving the band clean and isolated. Etch the enamel and apply adhesive (only on free surfaces), light-curing it according to the product instructions;

**Step 3:** Apply ORTHOBLUE in a thin layer inside the band and seat it onto the tooth;

**Step 4:** Remove excess cement and light-cure for 30 seconds from the occlusal surface using equipment with a minimum power of 800 mW/cm<sup>2</sup>;

**Step 5:** If excess cement remains, remove it using multi-fluted burs.

### **Removal:**

When removing the band, use a band removing plier. Residual material adhered to enamel can be removed using multi-fluted burs.

### **Occlusal Build-Up Procedure:**

**Step 1:** Keep the tooth clean and isolated. Etch the enamel and apply adhesive, light-curing according to the product instructions;

**Step 2:** Apply ORTHOBLUE in increments, shaping with a spatula to the desired form. Then light-cure each increment using equipment with a minimum power of 800 mW/cm<sup>2</sup>;

**Step 3:** If necessary, adjust the shape preferably using multi-fluted burs.

**CONSERVATION AND STORAGE:**

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

**TRANSPORT CONDITIONS:**

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

**MANUFACTURING DATE:** SEE PACKAGING.

**EXPIRATION DATE:** 2 YEARS AFTER THE MANUFACTURE DATE.

**ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.**

**TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.**

**PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.**

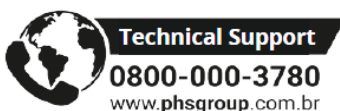
**KEEP THE PACKAGE CLOSED WHEN NOT IN USE.**

**IMPORTANT: DISCONTINUE USE OF THE PRODUCT IN CASE OF ALLERGIC REACTIONS.**

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