Potenza

ORTHOBLUE

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PHS DO BRASIL LTDA

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Potenza

ORTHOBLUE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 81199279025

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Trade Name: POTENZA ORTHOBLUE

Alternative Trade Name: LIGHT-CURABLE OCCLUSAL BUILD-UP RESIN

Product Technical Name: DENTAL CEMENTS

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Methacrylate-based monomers (including dimethacrylates such as Bis-GMA and UDMA), inorganic fillers, catalysts, pigments, and fluoride.

PRODUCT DESCRIPTION:

POTENZA ORTHOBLUE is a blue-colored, light-curable resin with medium viscosity and static behavior.

PRODUCT INDICATION:

POTENZA ORTHOBLUE is indicated for temporary occlusal build-ups during orthodontic treatment or rehabilitation procedures. It is also indicated for cementation of orthodontic bands.

PRECAUTIONS:

- Incomplete polymerization of the product or excessive contact time during the pre-curing phase may lead to mild gingival irritation;
- Protect the resin from ambient light;
- Do not apply the material under operatory light, as this may cause premature curing;
- Avoid contact with skin and eyes;
- Use gloves when handling the product;
- Use protective eyewear during handling.

CONTRAINDICATIONS:

- Do not use in patients with allergy to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- The product contains methacrylate monomers (such as Bis-GMA and UDMA), which may cause allergic reactions or mild irritation in sensitive patients;
- Discontinue use in case of allergic reactions;
- In case of allergic reaction, discontinue use and seek medical advice. Notify the manufacturer and keep these Instructions for use available.

WARNING:

- Observe the expiration date;
- Do not use after the expiration date;
- Keep out of reach of children;
- For dental use only;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is intended exclusively for dental use and must be handled according to the instructions for use;

- It is the responsibility of the professional to verify, prior to use, that the material is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;
- This product is supplied in a multi-dose package. To prevent microbial contamination, infection control measures must be adopted during use, including the use of disposable applicator tips, with mandatory replacement for each patient, and the non-reuse of any dispensed material;
- Contact of the product or its packaging with potentially contaminated surfaces must be avoided in order to prevent cross-contamination between patients;
- After opening, the product must be kept properly closed and stored according to the recommended conditions. As a light-sensitive material, it must be protected from exposure to ambient light in order to prevent premature polymerization and potential impairment of performance;
- Improper handling after opening, including failures in contamination control or inadequate storage, may compromise the adhesive properties of the cement and result in unsatisfactory clinical performance;
- Changes in the physical characteristics of the product, such as consistency, texture, or polymerization behavior, may indicate compromised quality, and its use is not recommended under such conditions;
- For safe disposal of the medical device after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Cementation of Orthodontic Bands:

Step 1: To increase band retention, apply air abrasion to its internal surface;

Step 2: Keep the tooth receiving the band clean and isolated. Etch the enamel and apply adhesive (only on free surfaces), light-curing it according to the product instructions;

Step 3: Apply ORTHOBLUE in a thin layer inside the band and seat it onto the tooth;

Step 4: Remove excess cement and light-cure for 30 seconds from the occlusal surface using equipment with a minimum power of 800 mW/cm²;

Step 5: If excess cement remains, remove it using multi-fluted burs.

Removal:

When removing the band, use a band removing plier. Residual material adhered to enamel can be removed using multi-fluted burs.

Occlusal Build-Up Procedure:

Step 1: Keep the tooth clean and isolated. Etch the enamel and apply adhesive, light-curing according to the product instructions;

Step 2: Apply ORTHOBLUE in increments, shaping with a spatula to the desired form. Then light-cure each increment using equipment with a minimum power of 800 mW/cm²;

Step 3: If necessary, adjust the shape preferably using multi-fluted burs.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

IMPORTANT: DISCONTINUE USE OF THE PRODUCT IN CASE OF ALLERGIC REACTIONS.

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