Potenza

ABRASIONE

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PHS DO BRASIL LTDA

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Potenza

ABRASIONE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79028

Product Commercial Name: POTENZA ABRASIONE.

Alternative Commercial Name: MICROABRASION PASTE

Technical Product Name: PASTES FOR DENTAL PROPHYLAXIS AND POLISHING.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Hydrochloric acid 6.6%, microparticles of silicon carbide, dye, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ABRASIONE is a paste that contains acid and microparticles of abrasive grains aiming chemical and mechanical abrasion of dental enamel. The microabrasion paste has medium viscosity, static characteristic and sandy appearance, in addition to gray coloration. The syringe offers the ready-to-use product.

PRODUCT INDICATION:

POTENZA ABRASIONE is indicated for the elimination of superficial stains present in dental enamel from fluorosis, hypoplasia or structural defects that accumulate pigment. Proper use of the product promotes controlled removal of the stained surface layer of enamel.

PRECAUTIONS:

- Avoid direct contact of POTENZA ABRASIONE with skin or mucous membranes. In case of contact, wash the affected region with plenty of water;
- After use, remove the tip used in the session and discard it properly;
- Relieve the syringe pressure by pulling the plunger back and close it with its original cap to avoid leaving residue in the syringe tip;
- During handling of the product, the professional should wear gloves and protective goggles;
- Protect the patient with a gown and protective goggles;
- In case of contact with the eyes, rinse with plenty of water and, if necessary, refer the patient to an emergency service, providing these instructions for use.

CONTRAINDICATIONS:

- Potenza Abrasione is contraindicated for individuals allergic to any of the components of the formula;
- It is contraindicated for the removal of deep discolorations where significant enamel loss is required;
- It is contraindicated for patients with periodontal diseases and patients with incomplete risogenesis.

POSSIBLE SIDE EFFECTS:

- Hypersensitivity may occur after treatment. In these cases, a desensitizing agent containing sodium fluoride and potassium nitrate should be used. In more severe cases, a clinical evaluation of the patient is recommended;
- In case of an allergic reaction, contact a doctor, pharmacist, or manufacturer,

providing this instruction for use.

WARNING:

- Respect the expiry date;
- Never use the product if it has expired;
- Keep out of reach of children;
- Dental use only;
- Before use, check the product's appearance. If you notice any abnormalities, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is manufactured for dental use only and should be handled according to the instructions for use;
- It is the professional's responsibility to verify, before use, whether this product is compatible and suitable for the intended purpose;
- In case of serious incidents related to the use of the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing the microabrasion paste may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during previous use and grow under ideal storage conditions. This may pose a risk of cross-contamination for the patient;
- Reuse may reduce the concentration of silicon carbide and hydrochloric acid in the microabrasion paste, resulting in a loss of treatment effectiveness. This can affect the paste's ability to remove stains or imperfections from the teeth satisfactorily;
- Reuse may deteriorate the texture and quality of the microabrasion paste. This can affect the paste's ability to properly adhere to the tooth surface and deliver a quality treatment;
- To ensure safe disposal of the DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

1st STEP: Before choosing the case for the microabrasion procedure, make sure that the stain is superficial in enamel. The transillumination examination can assist in diagnosis and planning.

2nd STEP: Install absolute insulation to prevent the patient from swallowing the product during washing and also to minimize contact with soft tissue.

3rd STEP: Lightly moisturize the stained region of the tooth and apply a small portion of the paste on the surface. Two options are suggested for applying toothpaste to teeth:

Mechanized - Rubber Bowls in Low Rotation: Apply light/moderate pressure in intermittent movements for 10 seconds.

Manual - Plastic Spatula: Apply moderate pressure in intermittent movements for 10 seconds.

4th STEP: After application, wash your teeth for at least 10 seconds and vacuum the entire contents. Observe the immediate result achieved and apply the paste in the same way up to 15 times, in the same session or in separate sessions.

5th STEP: Remove the absolute insulation and polish the rubbed region with felt disc and polishing paste (e.g., Potenza Specchi PRO). Then wash and dry the content.

6th STEP: Apply a sodium fluoride-based desensitizer (e.g., Potenza Esente 2%) for 5 minutes to complete the session.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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