Potenza

ORTHOBLUE

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PHS DO BRASIL LTDA

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Potenza

ORTHOBLUE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79025

EC REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ORTHOBLUE.

Alternative Commercial Name: RESIN FOR LIGHT-CURED OCCLUSAL LIFTING.

Technical Product Name: DENTAL CEMENTS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Bis-GMA, ethylene urethane dimethacrylate, inorganic fillers, pigments and catalysts.

PRODUCT DESCRIPTION:

POTENZA ORTHOBLUE is a light curing resin of blue color, medium viscosity and static characteristic.

PRODUCT INDICATION:

POTENZA ORTHOBLUE is indicated for temporary occlusal lifts during orthodontic treatment or rehabilitation. It is also indicated for cementation of orthodontic bands.

PRECAUTIONS:

- Incomplete polymerization of the product or excessive contact time in the pre-cure phase may lead to mild gingival irritation;
- Protect resin from ambient light;
- Do not apply the material under the light of a reflector, as this may cause its premature curing;
- Avoid contact with skin and eyes;
- Use gloves when handling the product;
- Wear goggles during handling.

CONTRAINDICATIONS:

- Do not use in patients with allergies to the components of the formulation.

POSSIBLE SIDE EFFECTS:

- The product contains methacrylate monomers that may cause allergic reactions or mild irritations in patients sensitive to these substances;
- Discontinue use of the product in case of allergic reactions;
- In case of an allergic reaction, contact the doctor, pharmacist, or manufacturer, presenting this usage instruction.

WARNING:

- Respect the expiry date. Never use the product if it has expired;
- Keep out of reach of children;
- Dental use only;
- Before use, check the appearance of the product. If you notice any anomalies,

contact the manufacturer immediately;

- The manufacturer is not responsible for any damage caused by improper use or incorrect handling;
- This product is manufactured for dental use only and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, before using the product, if this material is compatible and suitable for the intended use;
- In case of a serious incident related to the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing orthodontic cement may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during previous use and grow under ideal storage conditions. This can pose a risk of cross-contamination for the patient;
- Orthodontic cement is designed to create a reliable adhesive bond between the orthodontic bands and the patient's teeth. Reuse may compromise the cement's ability to effectively bond to dental surfaces, which could lead to the loss of retention of the bands and negatively affect the treatment progress;
- With reuse, the physical quality of the orthodontic cement may deteriorate. This includes changes in texture, consistency, and polymerization properties. These changes may affect the cement's ability to provide a secure and effective fixation;
- To dispose of the DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

CEMENTATION OF ORTHODONTIC BANDS:

1st STEP: To increase band retention, blast with abrasive on its inner perimeter.

2nd STEP: Keep the tooth that will receive the band clean and insulated. Make the enamel conditioning and subsequent application of adhesive (only on free faces), light curing it according to the instructions for use of the product.

3rd STEP: Apply ORTHOBLUE in a thin layer to the inside of the band and seat it on the tooth.

4th STEP: Remove excess cement and light curing for 30 seconds per occlusal using equipment with a minimum power of 800 mW/cm².

5th STEP: If excess cement is still left in the tooth, remove it with multi laminated drills.

REMOVAL:

When removing the band, use the bond removing pliers. Residues of the material adhered to enamel can be removed with multi-laminated drills.

PERFORMING OCCLUSAL LIFT:

1st STEP: Keep the tooth that will receive the lift clean and insulated. Make the enamel conditioning and subsequent application of adhesive, light curing it according to the instructions for use of the product.

2nd STEP: Apply ORTHOBLUE in increments by shaping the desired shape for lifting with spatula. Next, light cure each increment using equipment with a minimum power of 800 mW/cm².

3rd STEP: If necessary, preferably adjust with multi-laminated drills.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5 °C to 50 °C (23 °F to 122 °F);
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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