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## PHS DO BRASIL LTDA

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**IRRITANTE**

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**INSTRUCTIONS FOR USE FOR PROFESSIONALS**

## **PHS DO BRASIL LTDA**

Street Ottokar Doerffel, 1112, District: Atiradores, City: Joinville,

State: Santa Catarina, Country: Brasil Zip Code: 89203-212

CNPJ: 17.910.389/0001-05

BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)

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**Product's name:** POTENZA BLOCCO

**Product's Technical name:** Light-curing gingival barrier

**READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.**

**KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

### **BASIC COMPOSITION:**

Dimethacrylate groups, BHT, catalizers, pigments and dyes.

### **PRODUCT'S DESCRIPTION:**

**POTENZA BLOCCO** is a light-curing resin indicated to protect gingival margins during in office bleaching, once hydrogen peroxide may injury soft tissues in a situation of direct contact. POTENZA BLOCCO is also indicated to improve sealing of total dam, mainly for cases of complex restorations/cementations and endodontic treatments.

**PRODUCT'S INDICATION:**

Indicated to protect gingival tissue from direct contact to harmful substances (i.e. hydrogen peroxide) and also to improve sealing of total dam isolation.

**PRECAUTIONS:**

Avoid application on gingival tissue that has recently undergone surgical procedure or presenting unhealthy aspect. In some cases, incomplete light-curing of the product may cause mild irritation.

**SIDE EFFECTS:**

The product has methacrylate monomers in its composition that may cause allergic reactions or mild irritation to patient sensitive to these substances. Discontinue use in case of allergic reactions. If the patient present allergy, professional must take him/her to a medic along with the product's instructions for use.

**WARNINGS:**

Respect the expiry date of the product and do not use it after its expiry date. Maintain the product out of children's reach. Only for professional use. Before using, observe the aspect of the product, if any anomaly is noticed, contact the manufacturer immediately. Manufacturer does not take responsibility for damage caused by inadequate use or incorrect manipulation. This product is intended for dental use only and should be used strictly according to this instructions document. It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use. Do not reuse the empty package. For proper product disposal, follow local legislation.

**INSTRUCTIONS FOR USE:**

**1ST STEP:** Dry gingiva and teeth on the region to receive the product.

**2nd STEP:** Apply the product with the aid of the application tip. Make sure that the layer has about 1mm of thickness.

**3rd STEP:** For groups of 3 to 4 teeth, light-cure the resin for 20 to 30 seconds.

**4TH STEP:** Make sure there is no gap and that the resin is completely cured.

**5th STEP:** to remove the product, dettach it gently with an explorer until it comes off in big pieces or single piece.

**CONSERVATION AND STORAGE:**

Maintain the product in its original package. Protect it from direct sunlight. Keep it in temperatures of 0°C a 30°C (32o F to 86o F).

**TRANSPORTATION CONDITIONS:**

Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F). Protect it from direct sunlight.

**DATE OF MANUFACTURE:**

See package

**EXPIRY DATE:**

2 YEARS AFTER DATE OF MANUFACTURE

**SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).**

**PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.**

**WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.**

**FOLLOW THE INSTRUCTIONS  
FOR USE CORRECTLY**

**INTENDED FOR  
PRESCRIPTION USE  
ONLY**

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