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### PHS DO BRASIL LTDA

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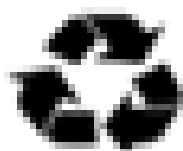
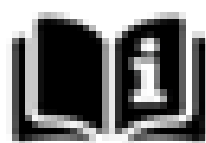
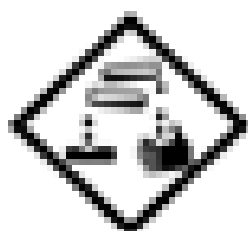
**CORROSIVO**

**IRRITANTE**

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## INSTRUCTIONS FOR USE FOR PROFESSIONALS

### PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville,  
State: SC, Country: Brazil CEP [Zip Code]: 89203-212  
CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279028

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC [Chemist License] #N.  
13303733

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**Product Technical Name:** MICROABRASION PASTE

**READ CAREFULLY ALL THE INFORMATION IN THIS  
INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.**

**KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER  
INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

## **BASIC COMPOSITION:**

Hydrochloric acid 6.6%, microparticles of silicon carbide, dye, thickeners and deionized water.

## **PRODUCT DESCRIPTION:**

**POTENZA ABRASIONE** is a paste that contains acid and microparticles of abrasive grains aiming chemical and mechanical abrasion of dental enamel. The microabrasion paste has medium viscosity, static characteristic and sandy appearance, in addition to gray coloration. The syringe offers the ready-to-use product.

## **PRODUCT INDICATION:**

POTENZA ABRASIONE is indicated for the elimination of superficial stains present in dental enamel from fluorosis, hypoplasia or structural defects that accumulate pigment. Proper use of the product promotes controlled removal of the stained surface layer of enamel.

## **PRECAUTIONS:**

- Avoid direct contact of POTENZA ABRASIONE with skin or mucous membranes. In case of contact, wash the affected region with plenty of water.
- After use, remove the tip used in the session and discard it properly.
- Remove the pressure from the syringe by pulling the plunger back and closing it with its original cap in order not to leave any remnants of paste on the tip of the syringe.
- During the handling of the product, the professional should wear gloves and goggles.
- Protect the patient with apron and goggles.
- In case of contact with the eyes, wash with plenty of water and, if necessary, refer the patient to emergency medical service with these instructions for use at hand.

## **CONTRAINDICATIONS:**

**POTENZA ABRASIONE** is contraindicated for people with a report of allergy to any of the components of the formula.

**POTENZA ABRASIONE** is contraindicated for removal of deep discolorations where very large wear is required with marked loss of dental enamel.

**POTENZA ABRASIONE** is contraindicated for patients with periodontal pathologies and patients with incomplete risogenesis.

### **SIDE EFFECTS:**

- Hypersensitivity may occur after treatment. In such cases, sodium fluoride and potassium nitrate-based desensitizers should be used. In the most acute cases, it is recommended to use analgesics for greater patient comfort.
- Discontinue use of the product in case of allergic reactions. In this case, refer the patient to emergency medical service with the instructions for use of the product at hand.

### **WARNING:**

- Respect the expiry date.
- Never use the product if it has expired.
- Keep out of reach of children.
- Dental use only.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.
- The manufacturer is not responsible for damage caused by improper use or improper handling.
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

### **INSTRUCTIONS FOR USE:**

**Step 1:** Before choosing the case for the microabrasion procedure, make sure that the stain is superficial in enamel. The transillumination examination can assist in diagnosis and planning.

**Step 2:** Install absolute insulation to prevent the patient from swallowing the product during washing and also to minimize contact with soft tissue.

**Step 3:** Lightly moisturize the stained region of the tooth and apply a small portion of the paste on the surface. Two options are suggested for applying toothpaste to teeth:

**a) Mechanized - Rubber Bowls in Low Rotation:** Apply light/moderate pressure in intermittent movements for 10 seconds.

**b) Manual - Plastic Spatula:** Apply moderate pressure in intermittent movements for 10 seconds.

**Step 4:** After application, wash your teeth for at least 10 seconds and vacuum the entire contents. Observe the immediate result achieved and apply the paste in the same way up to 15 times, in the same session or in separate sessions.

**Step 5:** Remove the absolute insulation and polish the rubbed region with felt disc and polishing paste (e.g., Potenza Specchi PRO). Then wash and dry the content.

**Step 6:** Apply a sodium fluoride-based desensitizer (e.g., Potenza Esente 2%) for 5 minutes to complete the session.

### **CONSERVATION AND STORAGE:**

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

### **TRANSPORT CONDITIONS:**

- The product can be transported at temperatures between -5 °C and 50 °C.
- Protect from the incidence of direct sunlight.

### **MANUFACTURING DATE:**

SEE PACKAGING.

### **SHELF LIFE:**

TWO YEARS AFTER THE MANUFACTURING DATE.

MI0030 - REV. 03

**FOLLOW THE INSTRUCTIONS  
FOR USE CORRECTLY**

**INTENDED FOR  
PRESCRIPTION USE  
ONLY**

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