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GLU-HEMA

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PHS DO BRASIL LTDA

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IRRITANTE

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GLU-HEMA

INSTRUCTIONS FOR PROFESSIONAL USE

PHS DO BRASIL LTDA

Street Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, State: Santa Catarina, Country: Brasil Zip Code: 89203-212

CNPJ: 17.910.389/0001-05

BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)

REGISTRY AT ANVISE NUMBER 8.11992.70012

TECHNICAL MANAGER ÉLEN ALVES LOOS - CRQ/SC N. 13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

PRODUCT'S NAME: POTENZA ESENTE GLU-HEMA

PRODUCT'S TECHNICAL NAME: SUBSTANCE FOR TREATMENT OF DENTIN HYPERSENSITIVITY

READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.

KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

2-hidroxietil-methacrylate, glutaraldehyde, deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE GLU-HEMA is an aqueous solution, made with HEMA monomer and glutaraldehyde (disinfectant and desensitizing agent). Its formula is capable of reducing dentin permeability by precipitating proteins that occlude dentin tubuli, besides blocking pain stimulation due to depolarization of the membrane of the nerve cell.

PRECAUTIONS:

- Keep the product away from heat and/or sun light.
- Avoid contact to unprotected skin.
- Product is potentially irritant and toxic.
- Use protective gloves during product manipulation.
- Use in small quantities, immediately close the flask after using.
- Before handling the product, wear protective goggles (professional and patient).
- Professional should orientate patient not to swallow the product. In case of contact with the eyes, wash it immediately with water and seek medical attention. In case of allergic reactions, professional should take the patient to a medic along with the instructions for use document.

CONTRAINDICATIONS:

- POTENZA ESENTE GLU-HEMA is contraindicated for people with known allergy to any of the formula components.
- Do not use the product if its not possible to follow the safety procedures and proper techniques.
- Do not use the product if tooth present any sign of pulp injury;
- Do not use the product in pulp exposure or deep cavities. In such cases, use proper base/lining.

SIDE EFFECTS:

Ingestion of big quantities of the product can cause nausea. Product can cause irritation if inhaled, if in contact to skin and eyes.

WARNINGS:

Respect the expiry date of the product and do not use it after its expiry date. Maintain the product out of children´s reach. Only for professional use. Before using, observe the aspect of the product, if any anomaly is noticed, contact the manufacturer immediately. Manufacturer does not take responsibility for damage caused by inadequate use or incorrect manipulation. This product is intended for dental use only and should be used strictly according to this instructions document. It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use. Do not reuse the empty package. For proper product disposal, follow local legislation.

INSTRUCTIONS FOR USE:_

PROTOCOL FOR TREATMENT OF DENTIN HYPERSENSITIVITY (in association with POTENZA ESENTE 2%):

Recommended: 4 sessions with interval of 48hours.

1st session: 2 applications (1st with gingival retraction cord and 2nd without it) of the desensitizing agent POTENZA ESENTE 2% (PHS).

1st STEP: Insert the lip retractor. Make prophylaxis with pumice and water to eliminate contamination from teeth surface.

2nd STEP: Insert retraction cord on gingival sulcus (without hemostatic).

3rd STEP: Apply POTENZA ESENTE 2% for 5 minutes, rubbing the gel on the buccal surface during this period.

4th STEP: Remove the retraction cord and the gel. Reapply the desensitizer and wait for more 5 minutes, rubbing it again.

2nd session: After 48h, repeat same steps as the 1st session.

3rd session: Apply Glu-Hema:

1st STEP: Insert the lips retractor. Make prophylaxis with pumice and water.

2nd STEP: Apply the product only on the areas showing exposed dentin, for 45 seconds (avoid contact of the product with gingiva and do not trespass the indicated time of application). Then rinse and dry the area.

4th session: After 48h, repeat same steps of the 3rd session.

INSTRUCTIONS FOR USE:_

PRE-BLEACHING DESENSITIZING PROTOCOL:

1st step: Make prophylaxis with pumice and water to eliminate contamination from teeth surface.

2nd step: With a disposable applicator or disposable brush, apply desensitizing gel POTENZA ESENTE 2% on teeth to be bleached.

3rd step: Let it on the buccal surface for 5 minutes, rubbing it during this period. Then, remove the gel with a fine saliva ejector and rinse the residues. Avoid swallowing by the patient.

4th step: If there is any area presenting exposed dentin, apply GLU-HEMA for 45 seconds (avoid contact of the product with gingiva and do not trespass the indicated time of application).

Then rinse and dry the area.

5th step: Initiate bleaching after this protocol. Follow specific product's instructions.

INSTRUCTIONS FOR USE:

TREATMENT FOR PREPARED DENTIN OF TOOTH RECEIVING FIXED PROSTHETICS:

1st step: Protect and isolate mucosa with rubber dam.

2nd step: Gently reduce humidity from prepared dentin, with gentle air stream. If needed, anesthetize patient.

3rd step: Apply POTENZA ESENTE GLU-HEMA and let it undisturbed for 30 to 45 seconds. Apply small amount and make sure the product does not contact exposed soft tissue.

4th step: Then, dry gently with air. Remove the product with gentle water spray.

5th step: Deep preparation or cavities should receive lining/base material.

CONSERVATION AND STORAGE:

Maintain the product in its original package. Protect it from direct sunlight. Keep it in temperatures of 0°C a 30°C (32o F to 86o F).

TRANSPORTATION CONDITIONS:

Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F). Protect it from direct sunlight.

DATE OF MANUFACTURE: See package

EXPIRY DATE: 2 YEARS AFTER DATE OF MANUFACTURE

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.

WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.

**FOLLOW INSTRUCTIONS
FOR USE CORRECTLY**

**INTENDED FOR USE
ONLY BY PRESCRIPTION**

**KEEP THE PRODUCT
OUT OF THE REACH OF
CHILDREN**

**RECYCLABLE
PACKING**

**SEE INSTRUCTION
MANUAL**

MI0076 – REV. 00

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GLU-HEMA

INSTRUCTIONS FOR PROFESSIONAL USE

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