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PHS DO BRASIL LTDA

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IRRITANTE

MI0018 – REV. 04

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville,

State: SC, Country: Brazil CEP [Zip Code]: 89203-212

CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279028

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC N. 13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product Trade Name: POTENZA IDRATA

Product Technical Name: MOUTH MOISTURIZER

READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Glycerin (wetting), polyethylene glycol, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA IDRATA is a glycerin-based and polyethylene glycol-based lubricating gel for topical application (skin or mucosa). The gel protects the application site against dryness that may cause soft tissue cracks. In addition, the product also creates an insulating film, preventing irritating or abrasive chemical agents from coming into direct contact with the skin/mucosa.

POTENZA IDRATA has no medicine in its composition.

PRODUCT INDICATION:

POTENZA IDRATA is indicated for topical application (skin and mucosa) in patients undergoing dental procedures (avoiding lip dryness and providing comfort to the patient), as in cases where the lip reformer will be employed (photographic protocol, dental bleaching, etc.) or on occasions where the site is subject to eventual contact of irritating chemical agents. POTENZA IDRATA can also be used in a rubber dam to facilitate the installation of absolute insulation on teeth.

PRECAUTIONS:

- Do not use in patients allergic to any of the components of the formulation.
- In case of suspected allergy, discontinue use of the product immediately and refer the patient to medical care.
- Do not reuse the packaging.
- Avoid contact with eyes.

CONTRAINDICATIONS:

POTENZA IDRATA is contraindicated for people with a report of allergy to any of the components of the formula.

WARNING:

- Respect the expiry date.
- Never use the product if it has expired.
- Respect the expiry date.
- Dental use only.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.

- The manufacturer is not responsible for damage caused by improper use or improper handling.
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

INSTRUCTIONS FOR USE:

Step 1: Dispense the required amount of POTENZA IDRATA in a sterile Dappen pot. With the aid of a cotton ball, cotton swab or even with the gloved finger, capture a small amount of the gel, apply and spread on the desired area until it forms a thin film.

Step 2: The region that received the product will adopt smooth and bright (damp) appearance. For long-term procedures, it may be necessary to reapply the cream to maintain the moisturizing effect.

Step 3: If it is necessary to remove the gel from the applied area, make it with gauze or cotton rubbing gently.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE:

SEE PACKAGING.

SHELF LIFE:

TWO YEARS AFTER THE MANUFACTURING DATE.

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

IMPORTANT: DISCONTINUE USE OF THE PRODUCT IN CASE OF ALLERGIC REACTIONS.

**FOLLOW THE INSTRUCTIONS
FOR USE CORRECTLY**

**INTENDED FOR
PRESCRIPTION USE
ONLY**

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