POTENZA ORTHOBLUE

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PHS DO BRASIL LTDA

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POTENZA ORTHOBLUE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, State: SC,
Country:

Brazil CEP [Zip Code]: 89203-212

CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279025

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC [Chemist License] #13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product Trade Name: POTENZA ORTHOBLUE

Product Technical Name: LIGHT CURING ORTHODONTIC
BAND CEMENT

**READ CAREFULLY ALL THE INFORMATION IN THIS
INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.**

**KEEP FOR REFERENCE UNTIL THE PRODUCT IS FULLY
ADMINISTERED. KEEP THIS INSTRUCTION UNTIL THERE IS
NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR
PATIENT.**

BASIC COMPOSITION:

Bis-GMA, ethylene urethane dimethacrylate, inorganic fillers, pigments and catalysts.

PRODUCT DESCRIPTION:

POTENZA ORTHOBLUE is a light curing resin of blue color, medium viscosity and static characteristic.

PRODUCT INDICATION:

POTENZA ORTHOBLUE is indicated for temporary occlusal lifts during orthodontic treatment or rehabilitation. It is also indicated for cementation of orthodontic bands.

INSTRUCTIONS FOR USE:

CEMENTATION OF ORTHODONTIC BANDS:

Step 1: To increase band retention, blast with abrasive on its inner perimeter.

Step 2: Keep the tooth that will receive the band clean and insulated. Make the enamel conditioning and subsequent application of adhesive (only on free faces), light curing it according to the instructions for use of the product.

Step 3: Apply ORTHOBLUE in a thin layer to the inside of the band and seat it on the tooth.

Step 4: Remove excess cement and light curing for 30 seconds per occlusal using equipment with a minimum power of 800 mW/cm².

Step 5: If excess cement is still left in the tooth, remove it with multi-laminated drills.

REMOVAL:

When removing the band, use the bond removing pliers. Residues of the material adhered to enamel can be removed with multi-laminated drills.

PERFORMING OCCLUSAL LIFT:

Step 1: Keep the tooth that will receive the lift clean and insulated. Make the enamel conditioning and subsequent application of adhesive, light curing it according to the instructions for use of the product.

Step 2: Apply ORTHOBLUE in increments by shaping the desired shape for lifting with spatula. Next, light cure each increment using equipment with a minimum power of 800 mW/cm².

Step 3: If necessary, preferably adjust with multi-laminated drills.

PRECAUTIONS AND CONTRAINDICATIONS:

- Incomplete polymerization of the product or excessive contact time in the pre-cure phase may lead to mild gingival irritation.
- Protect resin from ambient light.
- Do not apply the material under the light of a reflector, as this may cause its premature curing.
- Avoid contact with skin and eyes.
- Wear gloves to handle the product.
- Wear goggles during handling.

SIDE EFFECTS:

- The product has methacrylate monomers that may cause allergic reactions or mild irritations to patients sensitive to these substances.
- Discontinue use of the product in case of allergic reactions.

WARNING:

- Respect the expiry date. Never use the product if it has expired.
- Keep out of reach of children.
- Dental use only.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.
- The manufacturer is not responsible for damage caused by improper use or improper handling.
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether this material is compatible and suitable for the intended use.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

MANUFACTURING DATE:

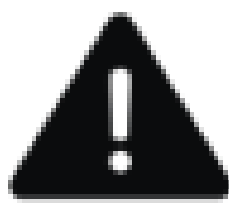
SEE PACKAGING.

SHELF LIFE:

TWO YEARS AFTER THE MANUFACTURING DATE.

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

**KEEP THE PACKAGING CLOSED WHEN NOT IN USE.
IMPORTANT: DISCONTINUE USE OF THE PRODUCT IN CASE OF ALLERGIC REACTIONS.**


**FOLLOW INSTRUCTIONS
FOR USE CORRECTLY**

**INTENDED FOR USE
ONLY BY PRESCRIPTION**

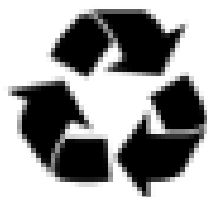**KEEP THE PRODUCT
OUT OF THE REACH OF
CHILDREN**

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