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PRO H2O2

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PHS DO BRASIL LTDA

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PRO H2O2

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79002

Product Commercial Name: POTENZA BIANCO HYDROGEN PEROXIDE PRO 35%.

Alternative Commercial Name: POTENZA BIANCO PRO H2O2 DENTAL WHITENING FOR USE IN OFFICES.

Technical Product Name: DENTAL WHITENING AGENTS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Hydrogen Peroxide at 35%, thickener, neutralizer, mixture of dyes, glycols and deionized water.

PRODUCT DESCRIPTION:

POTENZA BIANCO HYDROGEN PEROXIDE PRO 35% is a professional tooth whitening system based on Hydrogen Peroxide 35% for whitening vital and non-vital teeth. Its use is exclusive in the dental office and can be associated with Potenza Bianco Carbamide Peroxide PF (10%, 16% or 22%) in order to accelerate the whitening procedure (combined whitening). An efficient insulation system must be used for gingival protection (Potenza Blocco - Light curing gingival barrier). After homogeneous mixing of the two phases, a 35% hydrogen peroxide-based gel is formed. The action of bleaching agents is related to their penetration into dental structures (enamel and dentin), reacting with pigmented molecules, oxidizing them and making them simpler and less pigmented. In this way, the pigments incorporated into the tissues are totally or partially eliminated from the dental structure.

PRODUCT' INDICATION:

POTENZA BIANCO HYDROGEN PEROXIDE PRO 35% is for exclusive use in a dental office and indicated for whitening of vital teeth with extrinsic color changes, of a chronic character, which cannot be removed by conventional procedures (prophylaxis). It is also indicated for teeth with less severe intrinsic color changes (physiological darkening by dentin deposition), dental trauma. In the most severe intrinsic color changes such as fluorosis and severe staining by tetracycline, although effective, the results of whitening are less predictable. POTENZA BIANCO Hydrogen Peroxide 35% can still be indicated as a complementary technique for whitening non- vital teeth associated with whitening vitalized teeth.

PRECAUTIONS:

- Before whitening, make a detailed patient history. If the patient has spontaneous sensitivity in the teeth, it should be treated before bleaching, using the pre-bleaching desensitization protocol. The product Potenza Esente 2% is a desensitizer based on Potassium Nitrate and Sodium Fluoride to treat hypersensitivity in the office prior to tooth whitening, as well as Potenza Esente Glu-Hema which is a dentin obliterating agent used only in cases where there is dentin exposure.
- The patient must have a perfectly clean and uncalculated dental surface. The gums should be healthy and without inflammation.
- If the teeth have a stone, perform stone removal at least 2 days before whitening. Make sure that the teeth to be whitened do not present cavities or infiltrations in the restorations.

- In case of teeth with small fractures, incisal wear, dentin exposure or gingival retraction, protect the area with Potenza Blocco before applying the whitening gel to avoid sensitivity, as well as all adjacent gingival tissue.
- During bleaching, avoid contact of the gel with soft tissues. If contact occurs, apply Neutralizer to the area that has been in contact with the whitening gel. If the patient is sensitive (depending on his grade), the professional can stop the bleaching and start the desensitization protocol.

IMPORTANT:

- Teeth whitening treatment should only be carried out under the supervision of a dentist;
- The tooth whitening procedure is effective and safe if performed properly and with the correct products;
- Dental sensitivity is common during treatment. In this case, if the sensitivity persists, start the desensitization protocol.

CONTRAINDICATIONS:

- Pregnant, lactating patients or patients under 15 years old.
- Patients with periodontal pathologies.
- Patients with incomplete rhizogenesis.
- Patients with allergic reaction to the components of the formula.

POSSIBLE SIDE EFFECTS:

- Some patients may experience tooth sensitivity during or after bleaching.
- If the patient is sensitive during treatment, stop it and treat your teeth with the 2% Potenza Esente desensitizer.
- If the sensitivity persists and it is not possible to continue the bleaching, schedule a new session with a minimum interval of 7 days. Assess whether there are any abnormalities in the teeth that can be corrected to eliminate such sensitivity. There are cases where the sensitivity appears only after treatment (1 or 2 hours after).
- In these cases, if the sensitivity is high, it is recommended to prescribe an analgesic and, possibly, an anti-inflammatory, as needed in each case. Whitening can highlight pre-existing areas of hypocalcification (hypoplasia) due to the more intense lightening of these areas. In some cases, these spots disappear due to rehydration of the enamel after the whitening is complete.

WARNING:

- Respect the expiration date. Never use the product if it has expired;
- Keep out of the reach of children;
- For dental use Only;

- Before using, observe the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- **The thickener bottle must be shaken before use;**
- Respect the mixing time of the thickener with the hydrogen peroxide, as indicated in the instructions for use. If the mixture becomes liquid after following the risk of this process, **do not use** the product and contact the manufacturer;
- Due to the presence of hydrogen peroxide, the bottle may **bulge**. Keep container in a cool, tightly sealed place to prevent possible damage or leakage;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product has been manufactured for dental use only and must be handled in accordance with the instructions for use. It is the professional's responsibility to check, before using the product, whether this product is compatible and suitable for the intended use. Do not reuse the empty packaging for other purposes;
- To dispose of the product, follow the local legislation. In non-vital teeth, in cases where the canal sealing is not performed correctly, it can cause root resorption.

ATTENTION: To avoid pigmentation of the teeth, recommend to the patient that during the treatment do not eat products that contain strong dyes such as: coffee, wines, soft drinks, beets, saffron, chocolate, industrial sauces, etc.

INSTRUCTIONS FOR USE:

1st STEP: Select and register the color of the patient's teeth using a color scale. A photographic protocol can be used to monitor whitening.

2nd STEP: Apply Potenza Idrata moisturizing gel on the lips and lip commissure to protect, not dry out and avoid dehydration of the lips providing better comfort to the patient. Use a lip retractor to facilitate the application of the products.

3rd STEP: Perform pre-whitening desensitization protocol. Apply the Poteza Esente 2% desensitizer, leave for at least 5 minutes. Carry out friction movements with the aid of the disposable microapplicator foreach group of 3 or 4 teeth on all dental elements that will be performed to whiten. Always wash with water / air jets with the aid of a sucker (advise the patient not to ingest the product), dry. If there is exposed dentin, apply Potenza Esente Glu-Hema to the area for up to 45 seconds with the help of a disposable brush, being careful not to stick to the gums and do not exceed the determined time as it may cause irritation to the soft tissues, wash and dry.

4th STEP: Do the relative insulation with Potenza Blocco - Light curing gingival barrier, covering the marginal gingiva and the papillae with a layer of 3 to 5 mm wide and at most 1 mm thick. The gingival barrier should cover approximately 0.5 to 1 mm of the dental surface.

Observations: Use a clinical mirror looking from incisal to cervical and observe if there is uncovered gingival tissue. Make the correction if necessary. This step is crucial to avoid contact of Hydrogen Peroxide with the gums.

5th STEP: Shake the thickener bottle vigorously in order to homogenize the product well, as it can undergo decantation between the dye and the thickener, and may not form a static gel when mixed with Hydrogen Peroxide. Use the mixing plate that comes with the kit, mix phase 1 (Hydrogen Peroxide) with phase 2 (Thickener) in the ratio of 3 drops of Hydrogen Peroxide to 1 drop of Thickener (enough to be applied to a tooth). For the smile line (10 teeth), usually 18 drops of Hydrogen Peroxide for 6 drops of Thickener are sufficient. Shake the Thickenerbottle vigorously before using it.

6th STEP: With the help of a brush or spatula, completely cover the buccal surface of the teeth to be bleached, including interproximal teeth, and extend a little on the incisal and occlusal surfaces. The gel layer should be between 0.5 and 1 mm thick. Generally, second premolars to second premolars are used in both arches.

7th STEP: Let it act for 10 to 15 consecutive minutes on the teeth (as soon as the gel changes its color from violet to blue, due to the pH change of the same). Remove the gel with a vacuum cannula and wipe with gauze, making them ready to receive a new application of gel. Repeat steps 5 to 7 for up to two or three more times in the same session (depending on the patient's report in the anamnesis and clinical evaluation). The gel can remain on the dental substrate for up to 45minutes (30 to 45 minutes), monitor the evolution of the results and the patient's sensitivity.

8th STEP: After the recommended time, remove the gel from the teeth with an aspirating cannula and clean them with gauze.

9th STEP: Remove the gingival barrier, detaching it forward with the aid of an explorer probe. Thoroughly wash the dental surface with water jets and / or combined air and water jets, always using the suction device (advise the patient not to ingest the water with the whitening gel residue).

10th STEP: Polish your teeth with Potenza Specchi PRO (diamond polishing paste). Check the final result after applying the whitening gel. Make a record with photos of the initial and final procedure.

Observations: Monitor the patient throughout the whitening process and if excessive dental sensitivity occurs, stop the whitening and apply the 2% Potenza Esente. If there are spots of irritation in the gingival tissue by Hydrogen Peroxide during whitening, apply the Neutralizer to the area. If the whitening obtained does not meet the expectation and if the patient does not present sensitivity or another contraindication, the reapplication of the product can be done for up to two more sessions with a minimum interval of 7 days between sessions.

INSTRUCTIONS FOR USE (VITAL TEETH):

Before starting the bleaching, x-ray the tooth to be bleached to check its condition. The tooth must be within acceptable endodontic standards.

1st STEP: Select and register the color of the patient's teeth using a color scale. A photographic protocol can be used to monitor whitening.

2nd STEP: Make a good coronary opening by removing present restorations, decayed dentin, etc. With a clean pulp chamber, deepen the conduit entrance by removing approximately 3 mm of filling to make the canal seal (the seal prevents the diffusion of the bleaching agent into the cementum and periodontium region). It is recommended that the channel is sealed with a glass ionomer with a minimum thickness of 2mm (for better sealing).

3rd STEP:

Apply the Potenza Idrata moisturizing gel on the lips and lip commissure to protect, not dry out and avoid dehydration of the lips providing better comfort to the patient. Use a lip retractor to facilitate the application of the products.

4th STEP: Do the relative insulation with Potenza Blocco - Light curing gingival barrier, covering the marginal gingiva and the papillae with a layer of 3 to 5 mm wide and at most 1 mm thick. The gingival barrier should cover approximately 0.5 to 1 mm of the dental surface.

Observations: Use a clinical mirror looking from incisal to cervical and observe if there is uncovered gingival tissue. Make the correction if necessary. This step is crucial to avoid contact of Hydrogen Peroxide with the gums. Polymerize the gingival barrier for 20 to 30 seconds using the light curing agent for each group of 3 to 4 teeth. The gingival barrier that forms is rigid and insoluble, preventing possible irritation by aggressive products.

5th STEP: Shake the thickener bottle vigorously in order to homogenize the product well, as it can undergo decantation between the dye and the thickener, and may not form a static gel when mixed with Hydrogen Peroxide. Use the mixing plate that comes with the kit, mix phase 1 (Hydrogen Peroxide) with phase 2 (Thickener) in the ratio of 3 drops of Hydrogen Peroxide to 1 drop of Thickener. Shake the Thickener bottle vigorously before using it.

6th STEP: Apply Potenza Bianco Hydrogen Peroxide PRO 35% on the buccal surface and inside the pulp chamber with the help of a brush (a 1mm thick layer is enough).

7th STEP: Let it act for 10 to 15 consecutive minutes on the teeth (as soon as the gel changes its color from violet to blue, due to the pH change of the same).

8th STEP: Remove the gel using a suction cannula before reapplying the product. Up to three or four gel applications can be made in the same session, totaling 30 to 45 minutes.

9th STEP: After completing the bleaching agent applications, wash and dry the tooth. Temporarily restore the cavity and follow the whitening result for at least a week.

Note: The provisionally restored non-vital tooth has a weakened crown due to the lack of dental structure inside. Instruct the patient to take care not to fracture the crown of the weakened tooth. After removing Potenza Bianco Hydrogen Peroxide PRO 35%, you can use the bleaching Potenza Bianco PRO ENDO 35% - Carbamide Peroxide 35% for internal use and continue tooth whitening during the interval between sessions (see their specific instructions for use).

10th STEP: At the end of the tooth whitening treatment, definitively restore the whitened tooth. Wait 14 days to perform the final restoration.

GENERAL SAFETY INSTRUCTIONS:

- The product must be handled exclusively in a dental office;
- Wear gloves and goggles during the procedure (both the patient and the professional).
- Before opening the packaging, protect your hands and eyes and check that there is no liquid leaking from the packaging;
- Avoid contact of the product with skin, mucous membranes or clothing;
- Advise the patient not to ingest the product. In case of accidental contact with the skin or intra-oral mucosa, clean the area with plenty of water.

If the patient has allergic reactions, the use of the product should be discontinued. For these cases, the professional must take the patient to an Emergency Department along with the instructions for using the product. Leftovers or residues of used materials must not be reused and must be disposed of properly.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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