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PHS DO BRASIL LTDA

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Versão: 11

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PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

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Alternative Commercial Name: POTENZA ATTACCO ACID CONDITIONER FOR ENAMEL AND DENTIN.

Technical Product Name: ACID ETCHANT OF PHOSPHORIC ACID AT 35%.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Phosphoric acid at 35%, dye, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ATTACCO 35% is an acid conditioner based on 35% phosphoric acid, available as a thixotropic gel with an intense blue color. With excellent viscosity, the gel remains only at the application site — without flowing into adjacent areas. Its highly distinctive color allows easy visualization of the application area.

PRODUCT' INDICATION:

POTENZA ATTACCO 35% is indicated for acid conditioning of enamel and dentin, a step that precedes the application of adhesives. Its purpose is to increase microporosity in the tissues, consequently maximizing the mechanical retention of the adhesive.

PRECAUTIONS:

- Use personal protective equipment (gloves and safety glasses) when handling the product;
- Avoid exposing the product to direct sunlight during use and storage;
- Contact of the product with eyes, skin, or soft tissues may cause burns. In case of contact, immediately wash the affected area with plenty of water;
- Ingestion of large amounts of the product may cause nausea. The professional should instruct the patient not to swallow the product. If the patient accidentally ingests a large volume, the professional should take them to an emergency care facility along with the product's instructions for use;
- Not for injection use;
- At the first sign of an allergic reaction, discontinue use immediately.

CONTRAINDICATIONS:

- POTENZA ATTACCO 35% is contraindicated in patients allergic to any of the formula components;
- Do not use on metal or zirconia substrates, as it may reduce bond strength on these surfaces.

POSSIBLE SIDE EFFECTS:

- Leaving the product on the dental surface for longer than indicated may cause excessive demineralization of the tissues;
- Contact of the product with soft tissues may cause irritation or chemical burns.

WARNING:

- Never use the product if it is past its expiration date;
- For dental use only, by licensed dental professionals;
- Handle in accordance with the instructions for use;

- It is the professional's responsibility to verify, prior to use, whether the product is compatible and suitable for the intended application;
- Do not use the product if the packaging is damaged or if the product shows any change in appearance; in such cases, contact the manufacturer immediately;
- Keep out of reach of children;
- The manufacturer is not liable for damage caused by improper use or incorrect handling;
- Do not reuse empty packaging for other purposes;
- For safe disposal of the medical device after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Step 1: Isolate the area to receive the product, preferably with rubber dam isolation. Clean the dental surface with a fluoride-free polishing paste, rinse, and dry.

Step 2: In deep cavities, protect the dentin-pulp complex with a lining base (e.g., glass ionomer cement) before applying the product.

Step 3: Apply the product over the entire area to be acid conditioned, observing the following application times according to substrate: enamel 30 seconds and dentin 15 seconds.

Step 4: Thoroughly rinse the conditioned area with water for a period equal to or longer than the application time, until the gel is completely removed.

Step 5: After complete removal of the acid, keep the substrate dry or moist according to the recommendations of the adhesive system to be used.

Note: For certain self-etching adhesives, application of the product to dentin is not recommended. Consult the adhesive system manufacturer's instructions for use.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0°C and 30°C (32°F and 86°F).

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5 °C e 50 °C (23 °F to 122 °F);
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: SEE PACKAGING.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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