Potenza

ESENTE

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PHS DO BRASIL LTDA

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0800-000-3780

www.phsgroup.com.br

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79018

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, N° 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ESENTE 2%.

Alternative Commercial Name: DESENSITIZING AGENT.

Technical Product Name: SUBSTANCE FOR THE TREATMENT OF DENTAL HYPERSENSITIVITY.

READ ALL THE INFORMATION IN THIS INSTRUCTION TO USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Potassium Nitrate at 6%, Sodium Fluoride at 2%, humectant (glycerol), neutralizers, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE 2% is a topical desensitizing gel. It has a double desensitizing action based on Potassium Nitrate (neural action) and Sodium Fluoride (obliterating agent). It can be used before or after tooth bleaching when the sensitivity is significant.

POTENZA ESENTE 2% was developed for exclusive use in a dental office. Due to the high concentration of Sodium Fluoride, we do not recommend home use.

PRODUCT INDICATION:

POTENZA ESENTE 2% is indicated for the treatment of teeth with low to moderate sensitivity. It was developed to relieve dental sensitivity triggered by thermal and chemical agents and also in the treatment of sensitivity caused by the treatment of tooth bleaching.

PRECAUTIONS:

- Always keep the product away from heat and/or sunlight;
- Do not use if allergic to any component of the product;
- Do not ingest the product;
- In case of allergic reaction, contact a doctor, pharmacist, or manufacturer, with this instruction for use;
- Product is not injectable;
- Do not reuse the packaging.

CONTRAINDICATIONS:

- POTENZA ESENTE 2% is contraindicated in individuals with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- Ingesting large amounts of the product can cause nausea.

WARNING:

- For dental use only, as indicated by the dentist;
- The product should be handled according to the instructions for use;
- It is the responsibility of the professional to verify, prior to use, whether the product

is compatible and suitable for its intended purpose;

- Keep out of reach of children;
- Before use, check the appearance of the product. If any abnormalities are noticed, contact the manufacturer immediately;
- The manufacturer is not responsible for any damage caused by improper use or incorrect handling;
- In case of a serious incident related to the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing desensitizer may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during prior use and grow under ideal storage conditions. This may pose a cross-contamination risk for the patient;
- Reusing the product may reduce the concentration of potassium nitrate and sodium fluoride in the solution, leading to a loss of treatment efficacy. This may make the desensitizer less effective in reducing dental sensitivity;
- With reuse, the quality of the potassium nitrate and sodium fluoride-based desensitizer gel may deteriorate, which can affect its ability to properly adhere to dental surfaces and provide sensitivity relief;
- Dispose of the DM after use in accordance with local regulations.

INSTRUCTIONS FOR USE:

PROTOCOL FOR DENTAL HYPERSENSITIVITY TREATMENT:

There will be 4 sessions for treatment with a recommended interval of 48 hours between them.

1st SESSION: Perform two applications of the POTENZA ESENTE 2% desensitizing gel (with the first application done using the gingival wire retractor).

1st STEP: Insert the lip retractor. Carry out prophylaxis with pumice stone and water on the dental surface, eliminating any deposits that may prevent contact of the product on the teeth.

2nd STEP: Insert the retractor wire into the gingival sulcus in the dental elements to be carried out, without adding hemostatic solution.

3rd STEP: Apply POTENZA ESENTE 2% desensitizing gel uniformly on the teeth using a disposable micro-applicator, and wait at least 5 minutes, gently rubbing every 2 minutes on groups of 3 or 4 teeth.

4th STEP: Remove the gel and the retractor wire. Apply the desensitizer again to the teeth. Wait for at least 5 minutes, gently rubbing every 2 minutes on groups of 3 or 4 teeth. Totaling 10 minutes of application of POTENZA ESENTE 2% at the same session. Allow 48 hours before the second session.

2nd SESSION: Repeat the 1st session steps.

3rd SESSION: Carry out the application of the Potenza Esente Glu-Hema obliterating desensitizer.

1st STEP: Insert the lip retractor. Carry out prophylaxis with pumice stone and water on the dental surface, eliminating any deposits that may prevent contact of the product on the teeth.

2nd STEP: Apply Potenza Esente Glu-Hema only on the exposed dentin, for 45 seconds, being careful not to rub on the gums and do not to exceed the recommended time, as it can cause irritation to the soft tissues. Rinse and dry.

4th SESSION: Repeat the same steps as the 3rd Session.

PRE-BLEACHING DESENSITIZATION PROTOCOL:

1st STEP: Carry out prophylaxis with pumice stone and water on the dental surface, eliminating any deposits that may prevent contact of the product on the teeth.

2nd STEP: Apply the POTENZA ESENTE 2% desensitizing gel evenly over the teeth using the applicator tip or disposable micro-applicator.

3rd STEP: Let the product act for 5 minutes, making rubbing movements with the aid of the disposable micro-applicator on groups of 3 or 4 teeth. Then, aspirate the desensitizing gel

from the dental surface with the aid of an aspirating cannula, rinse with water and suction the water with a saliva ejector. It is important to advise the patient not to swallow the product.

4th STEP: Apply Potenza Esente Glu-Hema only to the exposed dentin area, for 45 seconds, being careful not to rub on the gums and do not exceed the recommended time, as it can cause irritation to the soft tissues. Rinse and dry.

5th STEP: Begin the bleaching procedure after completing the protocol described above. Follow the manufacture's recommended steps.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store the product at temperatures from 0°C to 30°C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5°C to 50°C (23°F to 122°F).
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 3 YEARS AFTER THE MANUFACTURE DATE.

USE EXCLUSIVELY IN DENTAL OFFICES. USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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