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PHS DO BRASIL LTDA

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Potenza

ESENTE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79018

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ESENTE 2%.

Alternative Commercial Name: DESENSITIZING AGENT.

Technical Product Name: SUBSTANCE FOR THE TREATMENT OF DENTAL HYPERSENSITIVITY.

READ ALL THE INFORMATION IN THIS INSTRUCTION TO USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Potassium Nitrate at 6%, Sodium Fluoride at 2%, humectant (glycerol), neutralizers, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE 2% is a desensitizing gel based on 6% potassium nitrate and 2% sodium fluoride, for topical use in the dental office. It acts through a dual mechanism of action: potassium nitrate promotes neural action by increasing extracellular potassium concentration and depolarizing nerve fiber membranes, blocking the transmission of painful stimuli; sodium fluoride acts by occluding dentinal tubules, reducing dentin permeability and contributing to hypersensitivity relief.

PRODUCT INDICATION:

POTENZA ESENTE 2% is indicated for the treatment of teeth with low to moderate hypersensitivity. It was developed to attenuate dental hypersensitivity triggered by thermal and chemical agents, particularly during tooth whitening.

PRECAUTIONS:

- Keep away from heat and/or sunlight at all times;
- Do not use if allergic to any of the product's components;
- Do not ingest the product;
- In case of allergic reaction, contact a physician and the manufacturer with this instruction for use at hand;
- Non-injectable product;
- Do not reuse the packaging.

CONTRAINDICATIONS:

- POTENZA ESENTE 2% is contraindicated in individuals with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- Ingesting large amounts of the product can cause nausea.

WARNING:

- For dental use only, as indicated by the dentist;
- The product must be handled in accordance with the instructions for use;
- It is the professional's responsibility to verify, prior to use, whether the product is compatible and suitable for the intended application;
- Keep out of reach of children;
- Before use, inspect the product's appearance; if any anomaly is observed, contact the manufacturer

immediately;

- The manufacturer is not liable for damage caused by improper use or incorrect handling;
- In the event of serious incidents related to the use of this product, these must be reported to the manufacturer and to the competent authority;
- Do not reuse empty packaging for other purposes;
- Do not reuse the product. Reuse may cause cross-microbial contamination, loss of efficacy and deterioration of the formulation, posing a risk to patient safety;
- For safe disposal of the desensitizer after use, follow local regulations.

INSTRUCTIONS FOR USE:

PROTOCOL FOR DENTAL HYPERSENSITIVITY TREATMENT:

It is recommended to perform 4 sessions with a 48-hour interval between them:

1st Session: Perform 2 applications of the product: the first using a gingival retraction cord and the second without it.

1.1 Apply the lip retractor. Polish the dental surface with pumice stone and water to remove any deposits that may prevent contact of the product with the teeth.

1.2 Insert the gingival retraction cord (without hemostatic agent) into the gingival sulcus of the teeth to receive the desensitizer.

1.3 Using a disposable microapplicator, apply POTENZA ESENTE 2% desensitizing gel evenly over the teeth and wait at least 5 minutes, rubbing the tooth surfaces every 2 minutes.

1.4 Remove the gel and the retraction cord. Apply the desensitizer again over the teeth. Wait at least 5 minutes, rubbing the tooth surfaces every 2 minutes. Upon completion of this step, the desensitizing gel will have been in contact with the teeth for a minimum of 10 minutes.

2nd Session: This session should be scheduled 2 days after the previous one. Repeat the same steps as the 1st session.

3rd Session: Application of POTENZA ESENTE GLU-HEMA desensitizing solution.

3.1 Apply the lip retractor. Polish the dental surface with pumice stone and water to remove any deposits that may prevent contact of the product with the teeth.

3.2 Apply POTENZA ESENTE GLU-HEMA only to the exposed dentin area and allow the product to act for 45 seconds. Take care not to rub the product on the gingiva and do not exceed the specified time, as this may cause irritation to the soft tissues. After application, remove excess product with a cotton pellet. If necessary, rinse and dry the area.

4th Session: This session should be scheduled 2 days after the previous one. Repeat the same steps as the 3rd session.

PRE-BLEACHING DESENSITIZATION PROTOCOL:

Step 1: Apply the lip retractor. Polish the dental surface with pumice stone and water to remove any deposits that may prevent contact of the product with the teeth.

Step 2: Apply POTENZA ESENTE 2% desensitizing gel evenly over the teeth using the applicator tip or a disposable microapplicator.

Step 3: Allow the product to act for 5 minutes, performing rubbing motions with the disposable microapplicator. Then rinse and aspirate the contents. Take care to prevent the patient from ingesting the gel.

Step 4: As a complementary measure, to enhance the desensitizing effect and maximize patient comfort, it is suggested to combine the use of POTENZA ESENTE GLU-HEMA. Apply it only to the exposed dentin area and allow the product to act for 45 seconds. Take care not to rub the product on the gingiva and do not exceed the specified time, as this may cause irritation to the soft tissues. After application, remove excess product with a cotton pellet, rinse and dry the area.

Step 5: Begin whitening after the presented protocol, following the steps recommended by the whitening agent manufacturer.

Step 6: At the end of the whitening session, if the patient presents residual sensitivity or as a preventive measure, repeat steps 1 through 4. Post-whitening application contributes to immediate relief of discomfort and enhances the desensitizing effect.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store the product at temperatures from 0°C to 30°C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5°C to 50°C (23°F to 122°F).
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 3 YEARS AFTER THE MANUFACTURE DATE.

USE EXCLUSIVELY IN DENTAL OFFICES. USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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