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AL UNODUE

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**PHS DO BRASIL LTDA**

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**Versão: 09**

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AL UNODUE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

**PHS DO BRASIL LTDA**

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

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**Alternative Trade Name:** POLISHING PASTE

**Product Technical Name:** DENTAL PROPHYLAXIS AND POLISHING PASTES

**PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.**

**KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

**Technical Support**

**0800-000-3780**

[www.phsgroup.com.br](http://www.phsgroup.com.br)

**BASIC COMPOSITION:**

Aluminum oxide ALF 240 (44.5 µm for UNO), aluminum oxide ALF 600 (9.3 µm for DUE), lubricating vehicle, thickeners, and flavoring.

**PRODUCT DESCRIPTION:**

POTENZA SPECCHI AL UNO DUE are aluminum oxide-based polishing pastes available in two grit sizes, with UNO being coarser and DUE finer. The pastes have medium viscosity and a thixotropic effect, with a grayish color and mint flavor.

**PRODUCT INDICATION:**

POTENZA SPECCHI AL UNO is indicated for pre-polishing of resins and other restorative materials, while POTENZA SPECCHI AL DUE is recommended for final polishing. Both should be used in conjunction with felt discs at low speed.

**PRECAUTIONS:**

- To avoid excessive heating of the dental surface, apply light and intermittent pressure during polishing with the felt disc and paste;
- The use of safety glasses is recommended for both the professional and the patient during application;
- Do not use in patients allergic to any components of the formulation.

**CONTRAINDICATIONS:**

- POTENZA SPECCHI AL UNO DUE are contraindicated for individuals with allergy to any component of the formulation.

**POSSIBLE SIDE EFFECTS:**

- Ingestion of large amounts of the product may cause nausea;
- Discontinue use in case of allergic reactions;
- In case of allergic reaction, discontinue use and seek medical advice. Adverse events must be reported to the manufacturer and the competent authority.

**WARNINGS:**

- Observe the expiration date;
- Do not use after the expiration date;
- Keep out of reach of children;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;
- This product is intended exclusively for dental use and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, prior to use, that the product is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the

manufacturer and the competent authority;

- Do not reuse empty packaging for other purposes;
- This product is supplied in a multi-dose package. To prevent microbial contamination, infection control measures must be adopted during use, including the use of disposable accessories or applicators and the non-reuse of any dispensed material;
- Contact of the product or its packaging with potentially contaminated surfaces must be avoided in order to prevent cross-contamination between patients;
- Dental polishing pastes contain abrasive particles and other components that may undergo changes after opening, especially if improperly stored;
- Changes in the physical characteristics of the product, such as consistency, texture, or performance, may indicate compromised quality and may impair polishing effectiveness;
- For safe disposal of the medical device after use, follow applicable local regulations.

### **INSTRUCTIONS FOR USE:**

Polishing is the final step of a dental restoration. Before polishing, ensure that finishing and pre-polishing steps have already been completed using burs, abrasive discs, and polishing rubbers of different grits. The quality of the final gloss depends largely on the smoothness of the pre-polished surface.

POTENZA SPECCHI AL UNO DUE pastes must be used in association with a disposable felt disc attached to a low-speed handpiece. The use of thin and flexible discs is recommended.

**Step 1:** For pre-polishing, apply a small amount of POTENZA SPECCHI AL UNO paste onto the felt disc or directly onto the restoration surface;

**Step 2:** Using intermittent movements with light pressure, cover the entire restoration and tooth surface to be polished. Typically, 2 minutes are sufficient for thorough polishing;

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**Step 4:** Check whether the desired gloss level has been achieved. If necessary, repeat the pre-polishing and polishing process.

### **CONSERVATION AND STORAGE:**

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

### **TRANSPORT CONDITIONS:**

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

**MANUFACTURING DATE:** SEE PACKAGING.

**EXPIRATION DATE:** 2 YEARS AFTER THE MANUFACTURE DATE.

**PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.**

**ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.**

**TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.**

**KEEP THE PACKAGE CLOSED WHEN NOT IN USE.**

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