Potenza

ESENTE

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PHS DO BRASIL LTDA

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Potenza

ESENTE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79018

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, N° 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ESENTE 0,2%.

Alternative Commercial Name: DESENSITIZING AGENT.

Technical Product Name: SUBSTANCE FOR TREATMENT OF DENTIN HYPERSENSITIVITY.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Potassium nitrate at 6%, sodium fluoride 0.2%, humectant (glycerol), neutralizers, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE 0.2% is a desensitizing gel based on 6% potassium nitrate and 0.2% sodium fluoride for topical use, applied by the patient in individual trays at home under professional supervision.

PRODUCT INDICATION:

POTENZA ESENTE 0.2% is indicated for the treatment of teeth with low to moderate hypersensitivity. It was developed to attenuate dental hypersensitivity caused by thermal and chemical agents, especially during tooth bleaching.

PRECAUTIONS:

- Always keep away from heat and/or sunlight;
- Do not use if allergic to any of the product's components;
- Do not ingest the product;
- Non-injectable product;
- Do not reuse the packaging.

CONTRAINDICATIONS:

- POTENZA ESENTE 0.2% is contraindicated in individuals with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- Ingesting large quantities of the product can cause nausea;
- In case of an allergic reaction, contact a doctor, pharmacist, or manufacturer with this usage instruction.

WARNING:

- For dental use only, as recommended by the dentist;
- Never use the product after its expiration date;
- The product must be handled according to the instructions for use;
- It is the professional's responsibility to verify, before using the product, that it is compatible and suitable for the intended purpose;
- Keep out of reach of children;
- Before use, check the product's appearance, if you notice any anomalies, contact the manufacturer immediately;

- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- In case of a serious incident related to the product's use, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing the desensitizer may lead to microbial contamination. Organic particles and bacteria can be introduced during previous use and grow under ideal storage conditions. This could pose a risk of cross-contamination to the patient;
- With reuse, the concentration of potassium nitrate and sodium fluoride in the solution may decrease, leading to a loss of treatment efficacy. This may make the desensitizer less effective in reducing dental sensitivity;
- With reuse, the quality of the potassium nitrate and sodium fluoride-based desensitizer gel may deteriorate, affecting its ability to properly adhere to dental surfaces and provide relief from sensitivity;
- Dispose of the DM after use in accordance with local regulations.

INSTRUCTIONS FOR USE:

These instructions should be passed on to the patient.

1st STEP: Before performing dental bleaching, brush and floss, removing any dirt from the surface.

2nd STEP: Apply POTENZA ESENTE 0.2% desensitizing gel with the aid of the tip applicator in the dental bleaching tray. Apply a small portion to the space corresponding to the teeth that will be bleached. Adapt the tray to the dental arch and press lightly to wrap the surface with the gel.

3rd STEP: Allow the product sit for 15 minutes. Remove the tray and rinse with water to eliminate all desensitizing gel from the dental surface. It is important to warn the patient not to swallow the product.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C;
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 3 YEARS AFTER THE MANUFACTURE DATE.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

Publication Date: 09-16-2025

Code: IFU-007

Version: 07

Potenza

ESENTE

INSTRUCTIONS FOR USE FOR PATIENTES

PHS DO BRASIL LTDA

Street Guilherme Boldt, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil, - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number 8.11992.79018

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, N° 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ESENTE 0,2%.

Alternative Commercial Name: DESENSITIZING AGENT.

Technical Product Name: SUBSTANCE FOR TREATMENT OF DENTIN HYPERSENSITIVITY.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

Technical Support

0800-000-3780

www.phsgroup.com.br

CE

BASIC COMPOSITION:

Potassium nitrate at 6%, sodium fluoride 0.2%, humectant (glycerol), neutralizers, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE 0.2% is a desensitizing gel based on 6% potassium nitrate and 0.2% sodium fluoride for topical use, applied to individual trays by the patient at home under professional follow-up.

PRODUCT INDICATION:

POTENZA ESENTE 0.2% is indicated for the treatment of teeth with low to moderate hypersensitivity. It was developed to attenuate dental hypersensitivity caused by thermal and chemical agents, especially during tooth bleaching.

PRECAUTIONS:

- Always keep away from heat and/or sunlight;
- Do not use if allergic to any of the product's components;
- Do not ingest the product;
- Non-injectable product;
- Do not reuse the packaging.

CONTRAINDICATIONS:

- POTENZA ESENTE 0.2% is contraindicated in individuals with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- Ingesting large quantities of the product can cause nausea;
- In case of an allergic reaction, contact a doctor, pharmacist, or manufacturer with this instruction for use.

WARNING:

- For dental use only, as indicated by a dentist;
- Never use the product after its expiration date;
- The product must be handled according to the instructions for use;
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use;
- Keep out of reach of children;
- Before use, check the appearance of the product. If you notice any abnormalities, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or

incorrect handling;

- In case of serious incidents related to the use of the product, this must be reported to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing the desensitizer may lead to microbial contamination. Organic particles and bacteria may be introduced during prior use and grow under ideal storage conditions. This could pose a risk of cross-contamination to the patient;
- With reuse, the concentration of potassium nitrate and sodium fluoride in the solution may decrease, leading to a loss of treatment effectiveness. This may make the desensitizer less effective in reducing dental sensitivity;
- With reuse, the quality of the potassium nitrate and sodium fluoride-based desensitizer gel may deteriorate, affecting its ability to properly adhere to dental surfaces and provide relief from sensitivity;
- Dispose of the DM after use in accordance with local regulations.

INSTRUCTIONS FOR USE FOR PATIENTS:

1st STEP: Before performing dental bleaching, brush and floss, removing any dirt from the surface.

2nd STEP: Apply POTENZA ESENTE 0,2% desensitizing gel with the aid of the tip applicator in the dental bleaching tray. Apply a small portion to the space corresponding to the teeth that will be bleached. Adapt the tray to the dental arch and press lightly to wrap the surface with the gel.

3rd STEP: Let it sit for 15 minutes or for the time recommended by your dentist. Remove the tray and rinse with water to eliminate all desensitizing gel from the dental surface.

IMPORTANT: Avoid swallowing the gel and contact your dentist if you have any questions.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0°C to 30°C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5°C to 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 3 YEARS AFTER THE MANUFACTURE DATE.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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