# Potenza

## STAMPO

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**PHS DO BRASIL LTDA**

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# Potenza

## STAMPO

INSTRUCTIONS FOR USE FOR PROFESSIONALS

**PHS DO BRASIL LTDA**

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.70017

EU

REP

**CINTERQUAL – Commerce Solutions**

**International Lda**; Avenida Defensores de Chaves, N° 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

**Product Trade Name:** POTENZA STAMPO EVA

**Alternative Trade Name:** FLEXIBLE SHEETS FOR CUSTOM TRAYS

**Product Technical Name:** SHEET FOR CUSTOM TRAY FABRICATION

**PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.**

**KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

Technical Support

0800-000-3780

www.phsgroup.com.br

**BASIC COMPOSITION:**

Ethylene vinyl acetate copolymer.

**PRODUCT DESCRIPTION:**

POTENZA STAMPO EVA is a flexible, colorless, and transparent sheet made of EVA (ethylene vinyl acetate copolymer).

**PRODUCT INDICATION:**

POTENZA STAMPO EVA sheets (1 mm) are indicated for the fabrication of custom thermoformed trays, primarily intended for the application of at-home whitening agents. They may also be used for topical application of fluoride agents or other low-viscosity agents, as prescribed by a dental professional.

This product is intended for use exclusively by dental professionals, such as dentists and dental technicians.

**PRECAUTIONS:**

- In case of allergic reaction, discontinue use and seek medical advice;
- Carefully read the instructions for use before using the product;
- Do not perform adaptations or modifications not described in the instructions for use;
- Check the integrity of the packaging and the product before use. In case of damage or tampering, do not use;
- Keep the product in its original packaging until use;
- Single-use product, which may be thermoformed only once;
- The gypsum model must be intact and with well-defined structures for proper tray fabrication;
- Keep the product away from heat sources and flames.

**CONTRAINDICATIONS:**

- Do not use in patients with known sensitivity to any component of the formulation.

**WARNING:**

- Do not use the product after the expiration date;
- Keep out of reach of children;
- For dental use only. Must be used by qualified professionals according to the instructions for use;
- Before use, check the integrity and appearance of the product. If any abnormalities or damage are observed, do not use;
- Improper use, use outside the instructions, or use of a damaged product may compromise safety and clinical performance;
- Single-use product. Reuse may result in microbial contamination and risk of cross-contamination between patients;

- It is the responsibility of the professional to assess the suitability of the product for the intended use;
- In case of serious incidents related to the use of the product, they must be reported to the manufacturer and the competent authority;
- For safe disposal of the medical device after use, follow applicable local regulations.

## **INSTRUCTIONS FOR USE:**

**Step 1:** Obtain a dental gypsum model of the patient's dental arches through appropriate impression techniques;

**Step 2:** Fix the sheet in the vacuum thermoforming machine and position the gypsum model centrally beneath it. Activate heating until the sheet shows uniform sagging (approximately 2 cm), indicating proper softening. Then activate the lowering mechanism and vacuum, maintaining it for approximately 10 seconds until full adaptation to the model is achieved. Remove the assembly and allow it to cool naturally to room temperature;

**Step 3:** After cooling, remove the tray from the model and trim the edges, maintaining approximately 1 mm beyond the gingival margins. Ensure that no sharp edges or burrs remain;

**Step 4:** Keep the tray on the model until delivery to the patient;

**Step 5:** Instruct the patient regarding use, hygiene, storage, and maintenance of the tray. During use, do not eat or drink. After use, rinse with running water at room temperature, dry completely, and store in a clean, dry place. Avoid exposure to heat, which may cause deformation of the material.

## **CONSERVATION AND STORAGE:**

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C (32 °F to 86 °F).

## **TRANSPORT CONDITIONS:**

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

**MANUFACTURING DATE:** SEE PACKAGING.

**EXPIRATION DATE:** 5 YEARS AFTER THE MANUFACTURE DATE.

**PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.**

**ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.**

**TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.**

**KEEP THE PACKAGE CLOSED WHEN NOT IN USE.**

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