Potenza

ESENTE

GLU-HEMA

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PHS DO BRASIL LTDA

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ESENTE

GLU-HEMA

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.70012

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ESENTE GLU-HEMA.

Alternative Commercial Name: DESENSITIZING AGENT.

Technical Product Name: DENTIN DESENSITIZER BASED ON GLUTARALDEHYDE AND HEMA.

READ ALL THE INFORMATION IN THIS INSTRUCTION TO USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support
0800-000-3780
www.phsgroup.com.br

BASIC COMPOSITION:

2-hidroxietyl-methacrylate, glutaraldehyde, deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE GLU-HEMA is an aqueous solution, made with HEMA monomer and glutaraldehyde (disinfectant and desensitizing agent). Its formula is capable of reducing dentin permeability by precipitating proteins that occlude dentin tubule, besides blocking pain stimulation due to membrane depolarization of the nerve cell.

PRODUCT INDICATION:

POTENZA ESENTE GLU-HEMA is indicated for the treatment and control of dentin hypersensitivity when dentin is exposed to the oral environment and, therefore, susceptible to chemical, thermal, tactile or osmotic stimulation that may generate discomfort.

PRECAUTIONS:

- Always keep away from heat and/or sunlight;
- Avoid contact with the skin;
- It has the potential for irritation and toxic effects;
- Wear gloves when handling the product;
- Use only the minimum amount required;
- Immediately close the bottle after use;
- Before use, protect the patient's eyes and wear protective glasses;
- Advise the patient not to ingest the product;
- In case of contact with the eyes, rinse immediately and thoroughly with water and consult a specialist.

CONTRAINDICATIONS:

- POTENZA ESENTE GLU-HEMA is contraindicated in individuals with hypersensitivity to any of the components in the formulation;
- Do not use the product on teeth with pulpitis requiring endodontic treatment, in restorative cavities, or in areas with exposed pulp.

POSSIBLE SIDE EFFECTS:

- Ingesting large quantities of the product can cause nausea;
- The product poses a health risk if inhaled and can cause irritation to the respiratory tract;

- If it comes into contact with the skin, it may cause irritation, and if it contacts the eyes, it may result in serious injury;
- In case of an allergic reaction, contact a doctor, pharmacist, or manufacturer with this usage instruction.

WARNING:

- Observe the expiration date;
- Never use the product after its expiration date;
- Keep out of the reach of children;
- For dental use only;
- Before use, check the appearance of the product. If you notice any abnormalities, contact the manufacturer immediately;
- The manufacturer is not responsible for damages caused by improper use or incorrect handling;
- This product is for dental use only and should be handled according to the usage instructions;
- It is the responsibility of the professional to verify, prior to use, whether the product is compatible and suitable for its intended purpose;
- In case of serious incidents related to the use of the product, it must be reported to the manufacturer and the competent authority;
- Do not reuse the empty container for other purposes;
- Reusing the desensitizer may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during previous use and grow in ideal storage conditions. This may pose a cross-contamination risk for the patient;
- Glu-Hema is one of the active ingredients that helps reduce dental sensitivity. With reuse, the concentration of Glu-Hema in the solution may decrease, leading to a loss of treatment efficacy, making it less effective in reducing dental sensitivity;
- For safe disposal of the DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

PROTOCOL FOR TREATMENT OF DENTIN HYPERSENSITIVITY (in association with POTENZA ESENTE 2%):

Recommended: 4 sessions with interval of 48 hours.

1st SESSION: 2 applications of the desensitizing agent POTENZA ESENTE 2%. First application must be made with gingival retraction wire and second application without the gingival retraction wire.

1st STEP: Insert the lip retractor. Make prophylaxis with pumice stone and water to eliminate contamination from teeth surface.

2nd STEP: Insert retraction wire on gingival sulcus (without hemostatic).

3rd STEP: Apply POTENZA ESENTE 2% for 5 minutes, gently rubbing the gel on the buccal surface.

4th STEP: Remove the retraction wire and the gel. Reapply the desensitizer and wait for more 5 minutes, gently rubbing the gel on the buccal surface.

2nd SESSION: After 48 hours, repeat the 1st session steps.

3rd SESSION: Apply Glu-Hema:

1st STEP: Insert the lip retractor. Make prophylaxis with pumice stone and water.

2nd STEP: Apply the product only on the areas with exposed dentin, for 45 seconds, being careful not to rub the product on the gums and do not exceed the indicated time of application. Then, rinse and dry the application area.

4th SESSION: After 48h, repeat the 3rd session steps.

PRE-BLEACHING DESENSITIZING PROTOCOL:

1st STEP: Make prophylaxis with pumice stone and water to eliminate contamination from teeth surface.

2nd STEP: With a disposable applicator or disposable brush, apply desensitizing gel POTENZA ESENTE 2% on teeth to be bleached.

3rd STEP: Leave the product on the buccal surface for 5 minutes, gently rubbing it. Then, remove the gel with a fine saliva ejector and rinse off any residue. Avoid patient swallowing.

4th STEP: If any area presents exposed dentin, apply GLU-HEMA for 45 seconds, being careful not to rub the product on the gums and do not exceed the indicated time of application. Then, rinse and dry the application area.

5th STEP: Begin the bleaching procedure after this protocol. Follow the specific product instructions.

TREATMENT FOR PREPARED DENTIN OF TOOTH RECEIVING FIXED PROSTHETICS:

1st STEP: Protect and isolate mucosa with rubber dam.

2nd STEP: Gently reduce humidity from prepared dentin, with gentle air stream. If needed, anesthetize patient.

3rd STEP: Apply POTENZA ESENTE GLU-HEMA and leave it for 30 to 45 seconds. Apply small amount and ensure the product does not come into contact with any exposed soft tissues.

4th STEP: Then, dry gently with air and remove the product with gentle water spray.

5th STEP: Deep preparations or cavities should receive lining/base material.

CONSERVATION AND STORAGE:

- Maintain the product in its original package;
- Protect it from direct sunlight;
- Keep it in temperatures of 0°C to 30°C (32°F to 86°F).

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5°C to 50°C (23°F to 122°F).
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 3 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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