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ARGILLA PRO

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PHS DO BRASIL LTDA

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Versão: 09

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ARGILLA PRO

INSTRUCTIONS FOR USE FOR PROFESSIONALS |

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79020

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Trade Name: POTENZA SPECCHI ARGILLA PRO.

Alternative Trade Name: POLISHING PASTE.

Product Technical Name: DENTAL PROPHYLAXIS AND POLISHING PASTES

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Suporte Técnico

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

White clay M635 (22.5 µm), aluminum oxide ALF 800 (18 µm), lubricating vehicle, thickeners, and flavoring.

PRODUCT DESCRIPTION:

POTENZA SPECCHI ARGILLA PRO is a clay-based polishing paste composed of minerals with different hardness levels and extra-fine particle size. It has medium viscosity and a thixotropic effect, with a grayish color and mint flavor.

PRODUCT INDICATION:

POTENZA SPECCHI ARGILLA PRO is indicated for the final polishing of dental enamel, resins, ceramics, acrylics, metals, and other restorative materials. It should be used in conjunction with disposable felt discs at low speed.

PRECAUTIONS:

- To avoid excessive heating of the dental surface, apply light and intermittent pressure during polishing;
- The use of protective eyewear is recommended for both the professional and the patient during application;
- Avoid ingestion of the product and contact with the eyes;
- Do not use in patients with hypersensitivity to any component of the formulation.

CONTRAINDICATIONS:

- POTENZA SPECCHI ARGILLA PRO is contraindicated for individuals with allergies to any of the formulation.

POSSIBLE SIDE EFFECTS:

- Ingestion of large amounts of the product may cause nausea;
- In case of allergic reaction, discontinue use and seek medical advice.

WARNINGS:

- Do not use the product after the expiration date;
- Keep out of reach of children;
- For dental use only. Must be used according to the instructions for use;
- Before use, check the appearance of the product. In case of abnormalities, do not use;
- Improper use or use outside the instructions may compromise product safety and performance;
- It is the responsibility of the professional to evaluate the suitability of the product for the intended use;
- In case of serious incidents related to the use of the product, report to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;

- This product is supplied in a multi-dose package. Adopt contamination control measures during use, avoiding contact with contaminated surfaces and the reuse of dispensed material;
- Changes in the physical characteristics of the product, such as consistency, texture, or performance, may indicate compromised quality;
- For safe disposal of the medical device after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Polishing is the final step of a dental restoration. Before polishing, ensure that finishing and pre-polishing steps have already been completed using burs, abrasive discs, and polishing rubbers of different grits. The quality of the final gloss depends largely on the smoothness of the pre-polished surface.

POTENZA SPECCHI ARGILLA PRO paste must be used in association with a disposable felt disc attached to a low-speed handpiece. The use of thin and flexible discs is recommended.

Step 1: Apply a small amount of paste onto the felt disc or directly onto the restoration surface;

Step 2: Using intermittent movements with light pressure, cover the entire restoration and tooth surface to be polished. Typically, 2 minutes are sufficient for thorough polishing;

Step 3: Check whether the desired gloss level has been achieved. If necessary, repeat the pre-polishing and polishing process.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGE

SHELF LIFE: 2 YEARS AFTER THE MANUFACTURING DATE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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