Potenza

ATTACCO

HF

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PHS DO BRASIL LTDA

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ATTACCO

HF

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 81199270019

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, N° 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Trade Name: POTENZA ATTACCO HF.

Alternative Trade Name: ACID ETCHANT FOR CERAMICS.

Product Technical Name: ACID ETCHANTS.

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Hydrofluoric acid 5% or 10% (depending on the version), dye, thickeners, and deionized water.

PRODUCT DESCRIPTION:

POTENZA ATTACCO HF is an acid etchant based on hydrofluoric acid, available in 5% or 10% concentrations, depending on the version. It presents medium viscosity and thixotropic behavior, allowing controlled application restricted to the desired area, minimizing flow. The gel contains a dye to facilitate visualization during application and is highly water-soluble, allowing efficient removal after use. It acts by promoting chemical conditioning of ceramic and glass surfaces through selective dissolution of the glassy phase, enhancing micromechanical retention.

PRODUCT INDICATION:

Indicated for acid etching of dental porcelains to increase micromechanical retention and surface area for adhesives and cements, enhancing the bond strength of the restoration after cementation to the tooth.

PRECAUTIONS:

- Protect from direct sunlight exposure;
- Contact with skin and/or eyes may cause burns; avoid direct contact;
- Ingestion of the product may cause adverse effects such as nausea;
- Product intended exclusively for extraoral use on prosthetic pieces not yet cemented;
- Non-injectable product;
- Do not reuse the packaging.

CONTRAINDICATIONS:

- Contraindicated for individuals with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- Contact with soft tissues may cause irritation or chemical burns; accidental ingestion may result in fluoride toxicity;
- In case of allergic reaction, discontinue use and seek medical advice.

WARNINGS:

- Do not use the product after the expiration date;
- For dental use only. Must be used according to the instructions for use;
- Keep out of reach of children;
- Before use, check the appearance of the product. In case of abnormalities, do not use;
- Improper use or use outside the instructions may compromise product safety and performance;
- It is the responsibility of the professional to evaluate the suitability of the product for the intended use;
- In case of serious incidents related to the use of the product, report to the manufacturer and the competent authority;

- Do not reuse empty packaging for other purposes;
- Single-use product. Reuse may compromise physicochemical stability, altering viscosity, texture, and performance;
- For safe disposal of the medical device after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

The product must be applied only to the internal surface of the prosthetic restoration that will receive the cementing agent. The contact time varies according to the substrate type. It is recommended to follow the ceramic manufacturer’s instructions. The application times below are suggested:

SURFACE	CONDITIONING TIME FOR POTENZA ATTACCO HF 5%	CONDITIONING TIME FOR POTENZA ATTACCO HF 10%
Feldspathic Porcelain	-	60s – 120s
Glass Ceramics Reinforced with Leucite	-	60s
Lithium Disilicate Based Glass Ceramics	20s	-
Zirconia	NOT ETCHABLE	NOT ETCHABLE

Step 1: Apply POTENZA ATTACCO HF only to the internal surface of the ceramic restoration (including margins). Allow the product to act according to the recommended time;

Step 2: Remove the acid by rinsing thoroughly for 2 minutes. This can be done using an ultrasonic bath or air/water spray. Then dry the restoration and observe a whitish and opaque appearance;

Step 3: Apply 35% phosphoric acid etchant (e.g., POTENZA ATTACCO 35%) to the previously etched surface. Leave for 1 minute, then rinse thoroughly and dry;

Step 4: With a clean and dry surface, apply silane, primer, or universal adhesive as selected. Follow the adhesive protocol according to the manufacturers’ instructions.

CONSERVATION AND STORAGE:

- Keep the product in its original package;
- Protect from direct sunlight;
- Store at temperatures between 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C to 50 °C (23 °F to 122 °F).

- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGE.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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