Potenza

CLOUD FLÚOR

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PHS DO BRASIL LTDA

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Potenza

CLOUD FLÚOR

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 81199270029

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Trade Name: POTENZA CLOUD FLUORIDE

Alternative Trade Name: FLUORIDE FOAM

Product Technical Name: FLUORIDE-CONTAINING DENTAL FLUID

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Water, sodium fluoride, humectant, foam stabilizer, flavoring, and preservative.

PRODUCT DESCRIPTION:

POTENZA CLOUD FLUORIDE is a fluoride foam for topical application, with a remineralizing effect on dental enamel, promoting the incorporation of fluoride ions and favoring the formation of fluorapatite.

PRODUCT INDICATION:

POTENZA CLOUD FLUORIDE is indicated for topical fluoridation of dental enamel, especially in patients at high risk of caries, including patients with special needs, those undergoing orthodontic treatment, patients subjected to periodontal scaling, or those with dentin hypersensitivity.

PRECAUTIONS:

- Use the product according to the instructions for use;
- Check packaging integrity and expiration date before use. Do not use if damaged or expired;
- Avoid contact with eyes, skin, and hair. In case of contact, rinse thoroughly with water;
- Keep out of reach of children and unauthorized individuals;
- Use the product in an upright position, according to the valve system instructions, to ensure proper foam formation;
- After use, keep the container properly closed.

CONTRAINDICATIONS:

- Contraindicated in patients with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- When used according to the instructions, no significant adverse effects are expected;
- In case of adverse reaction, discontinue use and seek medical advice.

WARNINGS:

- Avoid contact with mucous membranes, skin, eyes, and hair;
- Store protected from direct sunlight and heat sources;
- Product for dental use only;
- In case of serious incidents related to the use of the product, report to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;
- This product is supplied in a multi-dose package. Adopt contamination control measures during use, avoiding contact with contaminated surfaces;
- Changes in product characteristics, such as consistency or foam formation, may indicate compromised quality;

- For safe disposal of the medical device after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Step 1: Retract soft tissues (e.g., mouth opener);

Step 2: Perform prior professional prophylaxis using pumice and water or prophylactic paste;

Step 3: Position the patient in a seated or semi-seated position, using suction to control saliva and reduce the risk of ingestion;

Step 4: Dry the teeth with air before application to avoid dilution of the product;

Step 5: Apply the product to the dental surfaces, preferably using a fitted tray, maintaining contact for 1 to 4 minutes according to the clinical protocol;

Step 6: Remove excess product and instruct the patient not to eat, drink, or rinse for at least 30 minutes after application.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGE.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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