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PRO H2O2

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PHS DO BRASIL LTDA

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Versão: 10

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PRO H2O2

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79002

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA BIANCO HYDROGEN PEROXIDE PRO 35%.

Alternative Commercial Name: POTENZA BIANCO PRO H2O2 DENTAL WHITENING FOR USE IN OFFICES.

Technical Product Name: DENTAL WHITENING AGENTS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Dental Whitening Gel: Hydrogen peroxide, thickening polymers, polyethylene glycol, silica, pH-adjusting agent, colorants, and visual indicator.

Neutralizing Solution: Ultrapure water, sodium bicarbonate, and preservative.

PRODUCT DESCRIPTION:

POTENZA BIANCO HYDROGEN PEROXIDE PRO 35% is a professional dental whitening agent intended for in-office dental whitening procedures, under the supervision and exclusive application of a licensed dentist.

The product is based on 35% hydrogen peroxide and, after mixing its components, forms a thixotropic gel suitable for application onto the tooth surface. The gel has a carmine-red color that gradually changes to light blue during the procedure, assisting the professional in visualizing the material.

The product is supplied in two bottles, one containing the hydrogen peroxide phase and the other containing the thickening phase. The contents must be mixed according to the ratio indicated in the instructions for use immediately before application, forming the whitening gel ready for clinical use.

PRODUCT' INDICATION:

POTENZA BIANCO HYDROGEN PEROXIDE PRO 35% is indicated for dental whitening procedures performed in a dental office, under the supervision and exclusive application of a licensed dentist.

The product is indicated for whitening vital and non-vital teeth using the external whitening technique, in cases of tooth discoloration.

It may also be used as an adjunctive treatment for mild to moderate dental staining caused by fluorosis or the use of tetracycline and its derivatives.

PRECAUTIONS:

- Before performing the procedure, evaluate the presence of pre-existing dentin hypersensitivity. If the patient reports sensitivity during routine activities, such as consuming cold foods or beverages, exposure to cold air, or mechanical stimuli, a prior desensitizing protocol using Potenza Esente 2% and Potenza Esente GLU HEMA is recommended, according to the dentist's clinical judgment.
- In cases of significant gingival recession or root exposure that cannot be treated beforehand, it is recommended to adequately protect susceptible areas before starting the procedure, using Potenza Blocco gingival barrier or another isolation method deemed appropriate by the professional.
- Tooth surfaces that will receive the whitening gel must be clean and free from bacterial plaque, calculus, or other deposits. When necessary, prophylaxis or scaling procedures should be performed before whitening, preferably at least 5 days in advance, according to the professional's clinical evaluation.

- It is essential to avoid contact between the whitening gel and soft tissues during the procedure, since hydrogen peroxide may cause irritation or local injuries. Ensure proper application of the gingival barrier (e.g., Potenza Blocco) and use an appropriate lip retractor throughout the procedure.
- In case of accidental contact of the whitening gel with soft tissues, immediately stop the application and use the neutralizing solution supplied with the product, according to the instructions for use.
- If the patient experiences significant sensitivity during the whitening session, consider temporary or permanent interruption of the procedure, according to the clinical judgment of the responsible professional. When applicable, application of a desensitizing agent (e.g., Potenza Esente 2%) for 10 minutes is recommended before continuing or ending the session.
- Some patients may not be suitable candidates for in-office whitening with high-concentration hydrogen peroxide, and alternative supervised whitening protocols using lower-concentration products may be more appropriate, according to the dentist's evaluation.

IMPORTANT:

- Dental whitening treatment must be performed only under the supervision of a dentist;
- The efficacy and safety of the treatment depend on the correct use of the product;
- Whitening efficacy varies greatly from patient to patient according to individual factors (e.g., type of pigment present in the teeth, degree of tissue permeability, physiological intrinsic coloration of dentin, etc.). In some cases, it is suggested to add in-office whitening sessions or combine treatment with supervised at-home whitening gel.

CONTRAINDICATIONS:

Before starting whitening treatment, a detailed anamnesis and complete clinical examination of the patient must be performed to identify conditions that may compromise the safety, efficacy, or predictability of the procedure results.

A) TEMPORARY CONTRAINDICATIONS – Must be resolved before starting treatment:

- Presence of active caries;
- Infiltrated or defective restorations;
- Dental cracks or fractures;
- Periodontal diseases, including gingivitis and periodontitis;
- Presence of bacterial plaque or dental calculus;
- Teeth with doubtful pulp vitality;
- Untreated endodontic problems;
- Extensive gingival recession with significant root exposure;
- Uncontrolled dentin hypersensitivity.

B) ABSOLUTE CONTRAINDICATIONS – Treatment must not be performed in the following situations:

- Teeth affected by amelogenesis imperfecta or dentinogenesis imperfecta;
- Severe cases of dental fluorosis or staining caused by tetracycline and its derivatives;
- Teeth with marked mobility or significant periodontal bone loss;
- Teeth with incomplete root formation;
- Patients under 15 years of age;
- Pregnant women;
- Breastfeeding women;
- Patients with known hypersensitivity to hydrogen peroxide or to any component of the formulation;
- Patients with systemic conditions that, at the discretion of the responsible dentist, contraindicate elective whitening procedures.

The indication for treatment, as well as the evaluation of the patient's eligibility for the whitening procedure, is the responsibility of the dentist.

POSSIBLE SIDE EFFECTS:

- It is common for the patient to experience some degree of tooth sensitivity during the whitening session or in the first few days after the procedure. This condition is related to the mechanism of action of hydrogen peroxide and is usually transient. The intensity of discomfort may vary among patients. The use of desensitizing agents, such as Potenza Esente 2% and Potenza Esente GLU-HEMA, is recommended according to the dentist's clinical judgment. In cases of intense or persistent sensitivity, consider interrupting or suspending the treatment.
- The whitening procedure may reveal areas of hypocalcification, hypoplasia, or other pre-existing enamel changes due to temporary dehydration of dental tissues or increased visual contrast after whitening. In most cases, these changes become less noticeable after natural rehydration of the teeth or physiological remineralization processes. If such changes are observed, it is recommended to reassess the patient before continuing treatment. In persistent situations, the dentist should discuss with the patient the advisability of continuing treatment and the possible need for complementary restorative procedures.
- Some patients may show a non-uniform whitening response among different teeth or among different thirds of the same tooth. These variations are related to individual anatomical and structural characteristics of the teeth, including differences in the thickness and composition of dental tissues, which may limit the uniformity of the obtained result.
- Temporary irritation of the soft tissues of the oral cavity may occur in cases of accidental contact of the whitening gel with the gingiva, labial mucosa, or buccal mucosa. Proper isolation of soft tissues and use of the neutralizing solution supplied with the product help minimize this risk.

WARNING:

- Respect the product expiration date. Do not use the product after expiration.

- Keep the product out of reach of children.
- Product intended exclusively for dental use and for use by a licensed dentist.
- Before use, check the integrity of the packaging and the appearance of the components. If changes in color, consistency, leaks, or any other abnormality are observed, do not use the product and contact the manufacturer.
- The bottle containing the thickening phase must be shaken before use.
- Strictly follow the mixing ratio and homogenization time indicated in the instructions for use. If the proper gel consistency is not obtained after preparation, do not use the product and contact the manufacturer.
- Due to the presence of hydrogen peroxide, an increase in internal pressure in the containers may occur during storage. Keep the bottles properly closed and stored under the conditions recommended by the manufacturer.
- Do not use the product differently from what is indicated in these instructions for use.
- It is the responsibility of the dentist to assess the patient's clinical conditions and verify the suitability of the product for the intended purpose before use.
- Do not reuse empty packaging for any purpose.
- After use, disposal of the product, its residues, and packaging must be carried out in accordance with current local regulations.
- The product contains hydrogen peroxide and must be handled with care, avoiding contact with eyes, skin, mucous membranes, and unprotected soft tissues.
- Use appropriate personal protective equipment (PPE) during product preparation and application.

INSTRUCTIONS FOR USE:

Before using the product, read these instructions for use in full. Perform a detailed anamnesis and complete clinical examination of the patient, resolving beforehand any conditions that may contraindicate or compromise the treatment (see Contraindications and Precautions).

Use appropriate personal protective equipment (PPE) throughout the procedure. During the session, if accidental contact of the whitening gel with soft tissues occurs or if the patient presents excessive hypersensitivity, immediately interrupt the treatment and take the measures recommended in these instructions for use.

Step 1: Select and record the initial shade of the patient's teeth using an appropriate shade guide. Photographic record is recommended for monitoring clinical treatment progress.

Step 2: Apply Potenza Idrata moisturizing base to the patient's lips, especially at the lip commissures, providing hydration, protection, and greater comfort during the procedure. Then, position an appropriate lip retractor to facilitate access to the teeth.

Step 3: When applicable, perform a desensitizing protocol before whitening.

3.1. Apply Potenza Esente 2% to the buccal surface of the teeth to be whitened and leave the product in place for 10 minutes. During this period, perform friction movements using a disposable microapplicator. At the end, rinse thoroughly with water and air jets, using suction to avoid swallowing the product, and dry the teeth.

3.2. If areas with exposed root dentin are present, apply Potenza Esente GLU-HEMA with a disposable brush onto the exposed root surface and leave the product in place for 45 seconds, avoiding contact with the gingival margin. Then rinse and properly dry the area.

Step 4: Carefully dry the gingival tissue and apply Potenza Blocco gingival barrier onto the marginal gingiva of the teeth to be whitened. The layer should be approximately 1 mm thick and 5 mm wide, completely covering the gingival papillae. Also apply approximately 0.5 mm over the cervical enamel region. Light-cure the barrier for 20 to 30 seconds for each group of three teeth.

Check that the barrier is completely polymerized, rigid, and firmly adhered to the tissues before proceeding.

IMPORTANT: Using a dental mirror, observe the cervical region of the teeth to confirm that there are no exposed gingival areas. If necessary, supplement the gingival barrier application before starting whitening.

Step 5: To form the whitening gel, mix the hydrogen peroxide with the thickener at a 3:1 ratio. Normally, 18 drops of hydrogen peroxide to 6 drops of thickener are sufficient for application on the upper and lower arches, from premolar to premolar.

First dispense the hydrogen peroxide drops into the mixing container supplied with the kit. Then add the thickener drops and mix vigorously for up to 40 seconds, until a homogeneous, viscous, carmine-red gel is obtained.

IMPORTANT: The mixing ratio between the phases must always be maintained at 3 parts hydrogen peroxide to 1 part thickener, regardless of the amount prepared for the procedure.

NOTE: Before dispensing the thickener, shake the bottle to ensure complete homogenization.

Step 6: Apply the gel onto the buccal surface of the teeth using an appropriate spatula or brush, using sweeping movements. Completely cover the tooth surface, including the interproximal regions, extending the application slightly to the incisal or occlusal surfaces. The gel layer should be approximately 1 mm thick.

Step 7: Allow the gel to act for 15 consecutive minutes. During this period, the gel will gradually change color from carmine red to light blue, indicating the progression of the chemical reaction and assisting visual monitoring of the procedure.

At the end of the 15 minutes, carefully aspirate all the gel using a surgical or endodontic suction cannula. Remove remaining residues using gauze or cotton. Do not use water jets directly on the gel before initial suction.

Prepare a new mixture as described in Step 5 and repeat the application two more times, totaling three 15-minute applications in the same session.

The total gel contact time with the teeth will be 45 minutes (3 applications of 15 minutes), except in cases where the professional considers interruption of the procedure necessary due to sensitivity or other clinical conditions.

Step 8: After the third application, completely aspirate the whitening gel and remove any residues with gauze. Then rinse the teeth thoroughly and aspirate all contents.

Step 9: Remove the Potenza Blocco gingival barrier by carefully detaching it with the aid of an explorer probe. Rinse the teeth again with water and air jets, using suction simultaneously to avoid ingestion of whitening gel residues.

Step 10: Perform final polishing using felt discs associated with Potenza Specchi PRO diamond polishing paste.

The final evaluation of the whitening result should preferably be performed after complete rehydration of the dental tissues, usually observed after approximately 7 days. In general, 2 to 3 clinical sessions are required to obtain the expected result, with an interval of approximately one week between them. The protocol may be adjusted according to each patient's individual needs.

USE IN DARKENED NON-VITAL TEETH

POTENZA BIANCO HYDROGEN PEROXIDE PRO 35% may also be used for internal whitening of endodontically treated non-vital teeth.

The selected teeth must be free from clinically detectable cracks, signs of root resorption, relevant trauma history, or active endodontic problems.

Isolate the tooth with a rubber dam. Perform coronal access and remove approximately 2 mm of the filling material beyond the cemento-enamel junction. Properly seal the region using glass ionomer cement or flowable resin.

Apply the whitening gel into the pulp chamber and leave it in place for 15 minutes. Repeat the procedure two more times, totaling three 15-minute applications during the same session.

At the end of the session, completely aspirate the gel, rinse the cavity thoroughly, and dry it properly. Place a temporary restoration with glass ionomer cement and instruct the patient regarding chewing precautions necessary to avoid dental fractures.

After completion of the whitening treatment, perform the definitive restoration of the tooth. On average, approximately three clinical sessions are required to obtain the desired result, respecting a one-week interval between sessions.

IMPORTANT: Non-vital teeth have lower whitening response predictability when compared with vital teeth. During the procedure, continuously monitor the patient. If painful symptoms occur, immediately interrupt the treatment and evaluate the integrity of the internal seal, since pain may indicate diffusion of the whitening gel into the periodontal tissues.

GENERAL SAFETY INSTRUCTIONS:

- Product handling and application must be performed exclusively in a dental office by a licensed dentist.
- The professional must use appropriate personal protective equipment (PPE) for dental procedures, including mandatory gloves, mask, and protective eyewear.
- It is recommended that the patient wear protective eyewear, a disposable cap, and a protective drape over clothing throughout the procedure.
- The bottle containing hydrogen peroxide may present increased internal pressure due to inadequate storage conditions. Before opening the package, use the recommended PPE and check for signs of leakage, damage, or changes in the package.
- Avoid contact of the product with skin, eyes, mucous membranes, and clothing.
- Instruct the patient not to swallow the product or residues from the whitening procedure.
- In case of accidental contact of the whitening gel with skin or oral mucosa, immediately stop the application, rinse the affected area thoroughly with water, and apply the neutralizing solution supplied with the product as needed.
- In case of accidental contact with the eyes, rinse immediately with plenty of water for several minutes and seek medical attention, taking these instructions for use with you.
- If the patient shows signs of hypersensitivity or allergic reaction to any component of the formulation, immediately interrupt the treatment and refer the patient for medical evaluation.
- Do not reuse leftover or residual whitening gel prepared for the procedure.
- Do not reuse containers, accessories, or empty packaging for any purpose.
- Disposal of the product, its residues, and packaging must be carried out in accordance with current local regulations.
- Keep the product out of reach of children and unauthorized persons.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;

- Protect from direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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