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PF

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PHS DO BRASIL LTDA

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Versão: 11

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PF

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79017

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA BIANCO PF 35%

Alternative Commercial Name: POTENZA BIANCO PF DENTAL WHITENING FOR USE IN OFFICES.

Technical Product Name: DENTAL WHITENING AGENTS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

35% carbamide peroxide, carbomer, potassium nitrate, sodium fluoride, humectant (glycerol), neutralizing agents, and ultrapure water.

PRODUCT DESCRIPTION:

POTENZA BIANCO PF 35% is a dental whitening agent intended exclusively for professional use in dental offices, based on 35% Carbamide Peroxide. The product is presented as a colorless thixotropic gel, packaged in an applicator syringe, ready for immediate use without the need for mixing, activation, or prior preparation. Its formulation contains potassium nitrate and sodium fluoride as desensitizing agents, which help reduce dental sensitivity. In addition, its thixotropic characteristic reduces the risk of flow toward adjacent soft tissues during the procedure, while its humectant capacity improves wettability over the dental surface.

PRODUCT INDICATION:

POTENZA BIANCO PF 35% is indicated for professional in-office dental whitening procedures. The product may be used on vital and non-vital teeth presenting discoloration, including darkening caused by intrinsic or extrinsic factors. It is also indicated as an adjunctive treatment for mild cases of dental staining caused by fluorosis, tetracycline and its derivatives, as well as chromatic alterations associated with dental aging, trauma, or endodontic treatment

PRECAUTIONS:

- The dentist and dental assistant must wear personal protective equipment (gloves, safety glasses, mask, and gown) throughout the entire procedure. The patient must also be protected with safety glasses and an appropriate lip retractor to avoid accidental contact of the gel with the eyes, skin, and clothing;
- If the patient reports hypersensitivity during daily activities (e.g., intake of cold liquids, exposure to cold air, or pressure) without any identifiable cause, perform the desensitizing protocol (Potenza Esente 2% + Potenza Esente GLU-HEMA) prior to whitening;
- In cases of significant gingival recession that cannot be previously treated, isolate the area with a gingival barrier before starting the whitening procedure;
- The dental surfaces receiving the whitening gel must be clean and free of calculus and bacterial plaque. If necessary, prophylaxis and/or scaling procedures should be performed at least 5 days before whitening;
- It is essential that the whitening gel does not come into contact with soft tissues during the procedure, as high-concentration carbamide peroxide may cause transient or permanent tissue damage. Ensure proper application of the gingival barrier and use an efficient lip retractor throughout the procedure. In case of accidental contact of the gel with soft tissue, immediately

apply the Neutralizing solution to the affected area and rinse thoroughly with water. In severe cases, contact the manufacturer (see contact information on the product packaging);

- Do not use the product while the patient is under local anesthesia. The patient's perception of sensitivity during the procedure is an important safety parameter and must be monitored throughout the session;
- If the patient presents a high level of hypersensitivity during the session, suspend the procedure and apply Potenza Esente 2% desensitizing agent for 10 minutes;
- Instruct the patient to avoid acidic or strongly pigmented foods and beverages (e.g., coffee, tea, red wine, sauces) for at least 24 hours after each session in order to preserve the whitening result and reduce the risk of sensitivity;
- Some patients may not be suitable candidates for in-office whitening and may require treatment with lower-concentration gels for home use under professional supervision.

CONTRAINDICATIONS:

Before initiating whitening treatment, perform a detailed patient anamnesis, identifying habits, general health conditions, chronic and acute diseases. During the clinical examination, any transient condition (A) must be resolved before treatment begins. Certain conditions (B) absolutely contraindicate whitening treatment.

A) TRANSIENT CONTRAINDICATIONS - The following should be treated before starting treatment: Active caries, infiltrated restorations, dental cracks or fractures, periodontal diseases (gingivitis or periodontitis), presence of bacterial plaque or calculus, teeth with questionable vitality or untreated endodontic problems, teeth with extensive gingival recession.

B) ABSOLUTE CONTRAINDICATIONS - Treatment should not be performed in the following cases: Teeth with amelogenesis imperfecta or dentinogenesis imperfecta; teeth affected by severe fluorosis or severe tetracycline staining; teeth with mobility or significant bone loss; teeth with incomplete root formation; patients under 15 years of age; breastfeeding women; pregnant women; patients undergoing treatment for severe diseases or with severe uncontrolled diseases; patients allergic to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- It is common for patients to experience some degree of dental hypersensitivity during the session or up to 48 hours afterward due to the product's mechanism of action. Sensitivity may occur in response to cold, heat, or spontaneously, and is generally transient;
- The degree of discomfort varies significantly among patients. The associated use of desensitizing agents (Potenza Esente 2% + Potenza Esente GLU-HEMA) is recommended for greater comfort and safety during treatment;

- In cases of severe hypersensitivity that do not respond to desensitizing measures, discontinue treatment;
- Whitening may reveal pre-existing hypocalcification areas (enamel hypoplasia), which appear as opaque white spots due to temporary tooth dehydration or more intense whitening in these regions. In most cases, these spots disappear after tooth rehydration and remineralization, which occurs within hours or days after the session. Whenever this effect is identified, suspend treatment until the spots disappear. If persistent, discuss continuation of treatment with the patient, informing them about the possible need for restorative intervention;
- Some patients may present non-uniform whitening among different tooth thirds. This occurs for physiological reasons: the cervical region has greater dentin thickness, a structure naturally more pigmented than enamel predominant in the remaining thirds, which may limit the whitening effect in that region.

WARNING:

- Observe the expiration date. Never use the product after its expiration date; Keep out of reach of children;
- Before use, inspect the product appearance. If any abnormality is identified (change in color, consistency, odor, or leakage), do not use and contact the manufacturer immediately;
- The manufacturer is not responsible for damages resulting from improper use, inappropriate storage, or incorrect handling of the product;
- This product was manufactured exclusively for dental use and must be used according to these instructions for use. It is the professional's responsibility to verify, prior to use, whether the product is compatible and suitable for the intended purpose;
- Do not reuse empty packaging for other purposes;
- For disposal of the product, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Before using the product, read all sections of this document. Perform a detailed anamnesis and clinical examination of the patient and resolve any identified problems before starting treatment (see Contraindications and Precautions). The professional and assistant must wear appropriate PPE (gloves, safety glasses, mask, and gown) throughout the procedure. During the session, if accidental contact of the whitening gel with soft tissues or excessive dental hypersensitivity occurs, interrupt the session immediately and take the appropriate measures as instructed in this document.

Step 1: Select and record the patient's tooth shade using a shade guide, preferably with photographic documentation for comparison of results after treatment.

Step 2: Apply Potenza Idrata moisturizing base to the patient's lips, especially at the lip commissures, to hydrate, protect, and provide greater comfort during the procedure. Then position the lip retractor to facilitate access to the teeth and protect the patient's eyes with safety glasses.

Step 3: Perform the pre-whitening desensitization protocol:

3.1 Apply Potenza Esente 2% desensitizer to the buccal surface of all teeth to be whitened and allow it to act for 10 minutes, rubbing the surface with a disposable micro-applicator during this period. Then rinse thoroughly with water/air spray using suction to avoid swallowing, and dry the teeth.

3.2 If exposed dentin is present, apply Potenza Esente GLU-HEMA with a disposable brush to the exposed root surface, allowing it to act for 45 seconds. Avoid contact of the product with the gingival margin. Then rinse and dry the surfaces.

Step 4: Thoroughly dry the gingival tissue and apply Potenza Blocco gingival barrier to the marginal gingiva of all teeth to be whitened. The layer should be approximately 1 mm thick and 5 mm wide, fully covering the gingival papillae and approximately 0.5 mm of the cervical enamel region. Light-cure the barrier for 20 to 30 seconds for each group of 3 teeth. Once completed, verify that the barrier is rigid and firmly adhered to the gingiva before proceeding.

IMPORTANT: Using a clinical mirror, inspect from incisal to cervical areas for any exposed gingival tissue and correct if necessary. This step is essential to prevent contact of the whitening gel with the gingiva.

Step 5: The product is supplied in a ready-to-use syringe, requiring no mixing or prior preparation. Attach the applicator tip to the syringe before application.

Step 6: Apply the gel to the dental surface, fully covering the buccal surface of all teeth to be whitened, including interproximal areas. Extend the gel slightly to the incisal and occlusal surfaces. The layer should be approximately 1 mm thick. Use a disposable micro-applicator or explorer probe to help evenly distribute the gel over the dental surface.

Step 7: Allow the gel to act for 35 to 45 consecutive minutes. During this period, monitor the patient and, if necessary, redistribute the gel to ensure uniform coverage. At the end of the application time, aspirate all gel using a surgical or endodontic cannula and remove remaining residues with cotton or gauze. Rinse thoroughly and aspirate the contents while instructing the patient not to swallow.

Step 8: Remove the gingival barrier by lifting it anteriorly with the aid of an explorer probe. Rinse the dental surface again with air/water spray while simultaneously using suction, and instruct the patient not to swallow water containing whitening gel residues.

Step 9: Polish the teeth using felt discs associated with Potenza Specchi PRO diamond polishing paste to complete the session. Instruct the patient to avoid acidic or strongly pigmented foods and beverages for at least 24 hours.

The whitening result will be better visualized after 7 days. Typically, 2 to 3 sessions with a minimum interval of 1 week between them are required for complete treatment. The number of sessions may be adjusted according to each patient's needs and clinical response.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Keep the product in its original packaging.
- Store the product at temperatures from 0 °C to 30 °C (32 °F to 86 °F.).

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F).
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

SHELF LIFE: 2 YEARS AFTER THE MANUFACTURING DATE.

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 923 OF 2024).

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

MAKE SURE THAT THE INSTRUCTION FOR USE IS IN ACCORDANCE WITH THE PRODUCT TO BE USED.

TO ACCESS THE PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

IMPORTANT: DISCONTINUE USE OF THE PRODUCT IN CASE OF ALLERGIC REACTIONS

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