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PHS DO BRASIL LTDA

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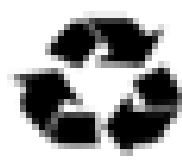
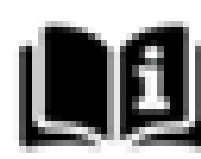
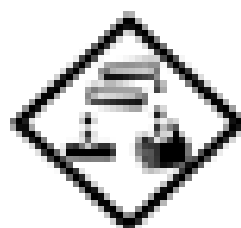
CORROSIVO

IRRITANTE

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Street Ottokar Doerffel, 1112, District: Atiradores, City: Joinville,

State: Santa Catarina, Country: Brasil Zip Code: 89203-212

CNPJ: 17.910.389/0001-05

BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)

Registry at ANVISA number 8.11992.79014

Technical Manager: ÉLEN ALVES LOOS - CRQ/SC N. 13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product's Name: POTENZA ATTACCO 35%

Product's technical name: ACID ETCHANT OF PHOSPHORIC ACID AT 35%

READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.

KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Phosphoric acid at 35%, dye, thickeners and deionized water.

PRODUCT'S DESCRIPTION:

POTENZA ATTACCO is a thixotropic gel based on phosphoric acid at 35%. It is indicated for acid etching of enamel and/or dentin, increasing bond strength of resin materials to the tooth's surface. Its proper viscosity allows easier handling as the gel does not run off from the area of application. The product's blue color helps identifying the application area. Potenza Attacco 35% provides an etching depth of about 2,0µm.

PRODUCT'S INDICATION:

Indicated for etching of enamel and/or dentin prior to adhesive application.

PRECAUTIONS:

- Keep away from sunlight.
- The contact of the product with eyes and/or skin can cause burning. Ingestion of big quantities of the product can cause nausea.
- The professional should instruct the patient not to swallow the product. If the patient swallows a big quantity of it and present any side effect, the professional should take him/her to get medical assistance, along with this instructions manual.
- This product is not for injection. Do not reuse the package.

CONTRAINDICATIONS:

POTENZA ATTACCO is contraindicated for individuals with history of allergic reactions to any of the components of the formula.

SIDE EFFECTS:

If the product remains on the dental tissues for longer periods than recommended, excessive demineralization may occur.

WARNINGS:

Do not use the product after its expiry date. This product is exclusive for dental use according to this document. It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use.

Keep out of children's reach.

Before using the product, observe the aspect of it. If any variation is noticed, contact the manufacturer immediately. The manufacturer does not take responsibility for eventual damages caused by inadequate use or incorrect manipulation. Do not reuse the empty package. For proper product disposal, follow local legislation.

INSTRUCTIONS FOR USE:

1st STEP: Clean and dry the dental surface to be etched. Control the area with relative or total dam isolation.

2nd STEP: Apply Potenza Attacco on the area to be etched. Leave it for 30 seconds on enamel and 15 seconds on dentin. Attention: the pulp wall of deep cavity preparations must be protected with GIC or calcium hydroxide before Potenza Attacco application.

3rd STEP: Rinse the etched area abundantly.

4th STEP: Dry enamel and leave dentin moist (to avoid collapse of the collagen fibers).

Note: Depending on the chosen adhesive system (i.e. self etching adhesives), Potenza Attacco 35% must not be applied to dentin.

CONSERVATION AND STORAGE:

Keep the product in its original package. Protect it from direct sunlight. Keep it in temperatures between 0°C to 30°C (32o F to 86o F).

TRANSPORTATION CONDITIONS:

Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F). Protect it from direct sunlight.

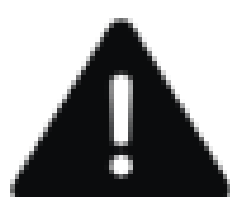
DATE OF MANUFACTURE: see package

EXPIRY DATE: 2 YEARS AFTER DATE OF MANUFACTURE

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.


WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.


**FOLLOW INSTRUCTIONS
FOR USE CORRECTLY**

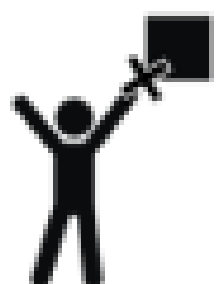**INTENDED FOR USE
ONLY BY PRESCRIPTION**

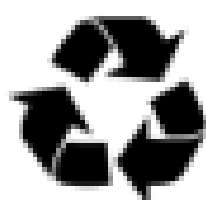**KEEP THE PRODUCT
OUT OF THE REACH OF
CHILDREN**

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