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PHS DO BRASIL LTDA

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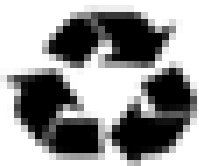
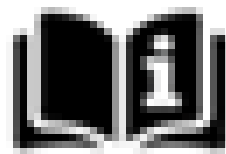
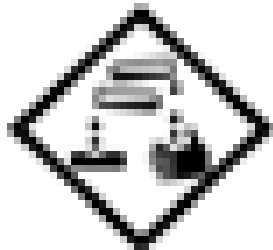
CORROSIVO

IRRITANTE

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Street Ottokar Doerffel, 1112, District: Atiradores,
City: Joinville, State: Santa Catarina, Country: Brasil Zip Code: 89203-212
CNPJ: 17.910.389/0001-05
BRAZILIAN INDUSTRY
E-MAIL: contato@phsdobrasil.com.br
Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)
Registry at ANVISA number 8.11992.70019
Technical Manager: ÉLEN ALVES LOOS - CRQ/SC N. 13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product's Name: POTENZA ATTACCO FLUORIDIC ACID
CONDITIONER

Product's technical name: FLUORIDIC ACID CONDITIONER

**READ ALL INFORMATION CAREFULLY BEFORE USING THE
PRODUCT.**

**KEEP THIS DOCUMENT UNTIL THERE IS NO MORE
INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

BASIC COMPOSITION:

Fluoridic Acid at 5 and 10%, Dye, Thickeners and Deionized Water.

PRODUCT'S DESCRIPTION:

POTENZA ATTACCO FLUORIDIC ACID CONDITIONER is found in concentrations of 5 and 10%. It is a thixotropic gel used for surface conditioning of dental porcelains. The conditioning allows the adhesion of feldspar porcelain, ceramic glazing and fixation of orthodontic brackets and repairs of porcelain with composites.

PRODUCT'S INDICATION:

Indicated for conditioning of dental porcelain, treatment carried out before silanization and application of adhesives. It aims to increase adhesion in restoration processes. Conditioning is essential for an optimal interaction between resin cement and ceramic restoration to occur, thus favoring greater retention and resistance.

PRECAUTIONS:

Keep away from sunlight. The contact of the product with eyes and/or skin can cause burning. Ingestion of big quantities of the product can cause nausea. The professional should instruct the patient not to swallow the product. If the patient swallows a big quantity of it and presents any side effect, the professional should take him/her to get medical assistance, along with this instructions manual. This product is not for injection. Do not reuse the package.

CONTRAINDICATIONS:

POTENZA ATTACCO FLUORIDIC ACID CONDITIONER is contraindicated for individuals with history of allergic reactions to any of the components of the formula.

SIDE EFFECTS:

Contact with soft tissues can cause irritation and accidental ingestion of the product can cause fluoride poisoning.

WARNINGS:

Do not use the product after its expiry date. This product is exclusive for dental use according to this document. It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use.

Keep out of children's reach.

Before using the product, observe the aspect of it. If any variation is noticed, contact the manufacturer immediately. The manufacturer does not take responsibility for eventual damages caused by inadequate use or incorrect manipulation.

Do not reuse the empty package.

For proper product disposal, follow local legislation.

INSTRUCTIONS FOR USE:

The treatment of the internal surface of the prosthetic parts must follow the indication according to the instructions of the porcelain manufacturer.

SURFACE	Conditioning time for Potenza Attacco Hydrofluoric Acid 5%	Conditioning time for Potenza Attacco Hydrofluoric Acid 10%
	FELDSPATHIC PORCELAIN	-
GLASS CERAMICS REINFORCED WITH LEUCITE	-	60s
LITHIUM DISSILICATE- BASED GLASS CERAMICS	20s	-
ZIRCONIA	UNCONDITIONABLE	UNCONDITIONABLE

After conditioning with Hydrofluoric Acid, proceed as follows:

1st step: Wash with plenty of water until the product is completely removed and dry well.

2nd step: Proceed with silanization according to the manufacturer's instructions.

3rd step: Apply the adhesive system.

Observation: Conduct the dental substrate with **POTENZA ATTACCO 35% PHOSPHORIC ACID CONDITIONER**. The conditioning of the dental substrate can vary according to the type of adhesive system, selective enamel conditioning can be performed, in the case of self-etching adhesive system. The choice of composite (in the case of repair) or resin cement is also the professional's choice.

CONSERVATION AND STORAGE:

Keep the product in its original package. Protect it from direct sunlight. Keep it in temperatures between 0°C to 30°C (32o F to 86o F).

TRANSPORTATION CONDITIONS:

Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F). Protect it from direct sunlight.

DATE OF MANUFACTURE: SEE PACKAGE.

EXPIRY DATE: 2 YEARS AFTER DATE OF MANUFACTURE.

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.

WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.

**FOLLOW INSTRUCTIONS
FOR USE CORRECTLY**

**RECYCLABLE
PACKING**

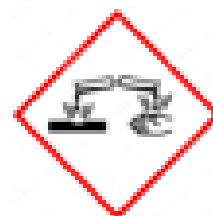
**SEE INSTRUCTION
MANUAL**

**INTENDED FOR USE
ONLY BY PRESCRIPTION**

**KEEP THE PRODUCT
OUT OF THE REACH OF
CHILDREN**

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