# POTENZA BIANCO **PRO** 38%

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### PHS DO BRASIL LTDA

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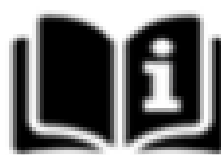
**IRRITANTE**

# POTENZA BIANCO **PRO** 38%

## INSTRUCTIONS FOR PROFESSIONAL USE

### PHS DO BRASIL LTDA

Street Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, State: Santa Catarina, Country: Brasil Zip Code: 89203-212

CNPJ: 17.910.389/0001-05

BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)

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**Product's name:** POTENZA BIANCO PRO SS 38%

**Product's Technical name:** In office dental bleaching agent

**READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.**

**KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

### **BASIC COMPOSITION:**

Hydrogen Peroxide at 38%, thickeners, neutralizers, dyes, glycols and deionized water.

### **PRODUCT'S DESCRIPTION:**

POTENZA BIANCO PRO SS 38% is a professional bleaching system based on hydrogen peroxide at 38%. The product is indicated exclusively for in office bleaching of vital teeth. It can be used isolatedly or in association with home bleaching (i.e. Potenza Bianco PF 10%, 16% or 22%, PHS)

to shorten the total treatment time. The bleaching gel should not contact soft tissues so it is mandatory to use an efficient gingival barrier (i.e. Potenza Blocco, PHS) during application. After mixing both phases (by connecting the syringes), a gel of Hydrogen Peroxide at 38% is obtained, which has a controlled level of pH. The activity of bleaching agents is associated to its penetration in the dental structure (enamel and dentin), reacting to the pigment's molecule and oxidizing it, transforming it into a less complex and smaller molecule. Thus, pigments incorporated to the tissue are totally or partially eliminated from dental structure.

The mixing system provided by the syringes ensures proper concentration of the hydrogen peroxide as well as its activity. Once phases A and B are mixed, the product is ready to use.

### **PRODUCT'S INDICATION:**

**POTENZA BIANCO PRO SS 38%** is indicated for in office bleaching of vital teeth with extrinsic shade alterations of chronic characteristic that are not removed by conventional procedures (i.e. dental prophylaxis) and also for teeth showing intrinsic less severe shade alteration (physiological darkening/ageing). For teeth showing staining by tetracycline or fluorosis, results may be effective but less predictable. The use of bleaching agents is contraindicated for cases of amelogenesis or dentinogenesis imperfecta. POTENZA BIANCO PRO SS 38% can also be indicated as a complimentary treatment for non-vital teeth, associated to internal bleaching.

### **PRECAUTIONS:**

Before bleaching: make a detailed anamnesis of the patient. In case he/she reports spontaneous teeth hypersensitivity, treat it before bleaching by using the desensitization protocol of POTENZA ESENTE 2% (potassium nitrate + sodium fluoride) associated to POTENZA GLU-HEMA (agent for dentin tubule occlusion). In cases where there is gingival recession, the area must be isolated with POTENZA BLOCCO (gingival barrier). Teeth surfaces must be free of contaminants and calculus, and the gingival tissue must be healthy without inflammation. Calculus removal must be performed in anticipation of at least 2 days before bleaching. Make sure teeth do not present caries or restorations with marginal leakage.

If little fractures or incisal wear is identified, those areas must also be isolated with the gingival barrier. Gingival margins must be dried before application of **POTENZA BLOCCO**.

**During the bleaching session:** Do not let the gel contact soft tissues. If it happens, apply NEUTRALIZING solution on the area. If the patient shows hypersensitivity (depending on the level), professional must stop the bleaching session and start the desensitizing protocol (combining Esente 2% with GLU-HEMA).

### **IMPORTANT:**

Dental bleaching must only be performed under professional 's supervision. It is a safe and effective treatment; however, it must be held properly. Dental hypersensitivity can occur as a side effect during the treatment, and if it is persistent, the professional must use desensitizing products to provide more comfort to the patient.

### **CONTRAINDICATIONS:**

Pregnant women, lactating women, patients under 15 years old, patients showing periodontal disease, patients with incomplete root formation and also those allergic to the components of the formula.

### **GENERAL SAFETY INSTRUCTIONS:**

Manipulation of the product must only be performed in a dental office. Use protective gloves during procedure. Use protective goggles for both professional and patient. The hydrogen peroxide present in the syringe can be under pressure or even leak due to adverse conservation or storage. Before opening the package, protect hands and eyes and observe if there is any spilled liquid. Avoid the contact of the product with the skin, mucosa or clothes. Instruct the patient not to swallow the product. In case of undue contact of the gel to the skin or mucosa, the area must be washed in abundance and the NEUTRALIZING solution must be applied.

If the bleaching gel contacts the eyes, wash immediately with water and seek medical attention. If patient present allergic reactions, discontinue the treatment. For those cases, patient must be conducted to medical assistance along with this product's instructions for use. Material's residue must not be used and should be properly disposed.

**SIDE EFFECTS:**

Some patient may present hypersensitivity during or after bleaching. If patient feels it during the treatment (depending on the level) discontinue treatment and use POTENZA ESENTE 2% (desensitizing agent). If hypersensitivity persists and its not possible to continue bleaching, schedule new appointment with a minimum interval of 7 days. Evaluate if there is no anomaly on teeth that can be corrected to eliminate the discomfort. There are cases where the hypersensitivity will only be noticed after the bleaching session (1 or 2 hours); for those cases, if the hypersensitivity is to high, it is recommended to use analgesics and sometimes anti-inflammatory drugs, according to each case. Bleaching can highlight areas of hypocalcification (white spots).

**WARNINGS:**

Respect the expiry date of the product and do not use it after its expiry date. Maintain the product out of children's reach. Only for professional use. Before using, observe the aspect of the product, if any anomaly is noticed, contact the manufacturer immediately. Manufacturer does not take responsibility for damage caused by inadequate use or incorrect manipulation. This product is intended for dental use only and should be used strictly according to this instructions document. It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use. Do not reuse the empty package. For proper product disposal, follow local legislation.

**ATTENTION:**

Recommend patient to avoid heavy dyes from diet such as coffee, wine, soda, chocolate, sauces, etc.

**INSTRUCTIONS FOR USE:**

**1st STEP:** Select and register the shade of the patient's teeth using a shade guide. Photographs are also useful for proper follow up.

**2nd STEP:** Apply hydrating gel POTENZA IDRATTA on lips to avoid dehydration and increase patient's comfort. Use lip retractor to facilitate products application.

**3rd STEP:** Perform the desensitizing protocol before bleaching. Apply POTENZA ESENTE 2% for at least 5min, disturbing the gel frequently in each group of 3 to 4 teeth. Rinse it with water/air along with the saliva ejector (don't let the patient swallow big amounts of the gel) and dry the area. If there is exposed dentin, apply POTENZA GLU-HEMA on the area for 45 seconds (avoid contact to the gingival tissue), then wash and dry the area.

**4th STEP:** make proper relative isolation with POTENZA BLOCCO (light-curing gingival barrier), covering marginal gingiva with a coat of approximately 3 to 5mm of width and 1mm thick. Dental area must also be covered in 0,5 to 1mm.

Note: use a mirror to see from incisal to cervical and observe if there is any uncovered area. Add material if needed. This step is important to avoid bleaching gel to contact soft tissue. Light-cure the gingival barrier for 20-30 seconds for each group of 3-4 teeth, using a light-curing unit of blue light. The barrier becomes rigid, preventing contact of the bleaching gel to soft tissues.

**5th STEP:** Remove cap from syringe A (Hydrogen Peroxide) and syringe B (thickener). The syringes present threaded ends, so connect both by twisting until they are firmly attached. Press the plungers alternatively to transfer the content from one syringe to the other, mixing phases. Ten movements in each direction are enough for proper mixing. Each movement must be slow to obtain a stable mixing. Once the mixing is obtained, transfer the content to syringe A and unscrew syringes. Attach the application tip to syringe A and apply the gel directly on the teeth, in screening movements. Distribute the gel all over the buccal surface of teeth to be bleached. Air voids must be eliminated with disposable microapplicator or explorer.

**6th STEP:** with the aid of a brush or spatula, cover all buccal surface including proximal areas. Overlap incisal and occlusal areas. The gel layer must present thickness of 0,5 to 1mm.

**7th STEP:** leave the gel on teeth for 25 to 30 minutes. POTENZA BIANCO PRO SS 38% does not demand changing the gel during the session. There is also no need for light application.

**8th STEP:** after the recommended time, remove gel from teeth with saliva ejector and clean it with a gauze.

**9th STEP:** remove the gingival barrier detaching it forward with an explorer. Rinse the surface well with air/water jet, always aided by the saliva ejector (prevent the patient from swallowing gel residue).

**10th STEP:** polish teeth with POTENZA SPECCHI PRO (diamond polishing paste) and felt disks. Register shade at the end of the session.

**NOTE 1:** monitor patient during all bleaching treatment. In case excessive hypersensitivity is noticed, discontinue process as previously stated in this document.

**NOTE 2:** If the result achieved do not correspond to the expectations and the patient do not present hypersensitivity or other contraindication, product reapplication can be performed for 2 more times with a minimum interval of 7 days between sessions.

**NOTE 3:** Bleaching can highlight pre-existing areas of hypocalcification due to an uneven bleaching of those areas. In most cases, the white spots disappear after teeth hydration. Whenever noticing this situation, suspend bleaching process and evaluate if the spots disappear in about 1 or 2 days. If not, tell the patient and evaluate if it is plausible to proceed with the treatment or not.

### **CONSERVATION AND STORAGE:**

Maintain the product in its original package. Protect it from direct sunlight. Keep it in temperatures of 0°C a 30°C (32o F to 86o F).

### **TRANSPORTATION CONDITIONS:**

Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F). Protect it from direct sunlight.

### **DATE OF MANUFACTURE:**

See package

### **EXPIRY DATE:**

2 YEARS AFTER DATE OF MANUFACTURE

**SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).**

**PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.**

**WHEN NOT IN USE, KEEP THE PRODUCT´S PACKAGE CLOSED.**

**FOLLOW INSTRUCTIONS  
FOR USE CORRECTLY**

**INTENDED FOR USE  
ONLY BY PRESCRIPTION**

**KEEP THE PRODUCT  
OUT OF THE REACH OF  
CHILDREN**

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