POTENZA ESENTE 0,2%

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PHS DO BRASIL LTDA

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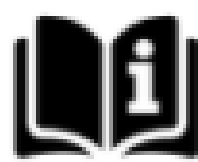
POTENZA ESENTE 0,2%

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, Estado: SC,

País: Brazil CEP [Zip Code]: 89203-212

CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279018

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC

[Chemist License] #13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product Trade Name: POTENZA ESENTE 0,2%

Product Technical Name: SUBSTANCE FOR TREATMENT OF DENTIN HYPERSENSITIVITY

READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Potassium nitrate at 6%, sodium fluoride 0.2%, wetting (glycerol), neutralizers, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE 0,2% is a desensitizing gel based on 6% potassium nitrate and 0.2% sodium fluoride for topical use, applied to individual trays by the patient at home under professional follow-up.

PRODUCT INDICATION:

POTENZA ESENTE 0,2% is indicated for the treatment of teeth with low to moderate hypersensitivity. It was developed to attenuate dental hypersensitivity caused by thermal and chemical agents, especially during tooth bleaching.

PRECAUTIONS AND CONTRAINDICATIONS:

- Always keep away from heat and/or sunlight.
- Do not use if you are allergic to the components of the product.
- Do not ingest the product.
- In case of allergic reaction, refer the patient to emergency medical service with this document at hand.
- Non-injectable product.
- Do not reuse the packaging.

SIDE EFFECTS:

- Ingestion of large amounts of the product may cause nausea.

WARNING:

- Dental use only and as indicated by the dentist.
- Never use the product if it has expired.
- The product must be handled according to the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use.
- Keep out of reach of children.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.

- The manufacturer is not responsible for damage caused by improper use or improper handling.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

INSTRUCTIONS FOR USE:

These instructions should be passed on to the patient.

Step 1: Before performing dental bleaching, brush and floss, removing any dirt from the surface.

Step 2: With the aid of the tip applicator, apply the **POTENZA ESENTE 0,2%** desensitizing gel in the dental bleaching tray. Apply a small portion to the space corresponding to the teeth that will be bleached. Adapt the tray to the dental arch and press lightly to wrap the surface with the gel.

Step 3: Leave to act for 15 minutes. Remove the tray and rinse with water to remove all desensitizing gel from the dental surface. It is important to instruct the patient not to swallow the product.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE:

SEE PACKAGING.

SHELF LIFE:

TWO YEARS AFTER THE MANUFACTURING DATE.

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

**FOLLOW THE INSTRUCTIONS
FOR USE CORRECTLY**

**INTENDED FOR
PRESCRIPTION USE
ONLY**

**KEEP OUT OF
REACH OF
CHILDREN**

**RECYCLABLE
PACKAGING**

**SEE INSTRUCTION
MANUAL**

MI0017 - REV. 03

POTENZA ESENTE **0,2%**

INSTRUCTIONS FOR USE FOR PATIENTS

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, Estado: SC,

País: Brazil CEP [Zip Code]: 89203-212

CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279018

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC

[Chemist License] #13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product Trade Name: POTENZA ESENTE 0,2%

Product Technical Name: SUBSTANCE FOR TREATMENT OF DENTIN HYPERSENSITIVITY

READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Potassium nitrate at 6%, sodium fluoride 0.2%, wetting (glycerol), neutralizers, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ESENTE 0,2% is a desensitizing gel based on 6% potassium nitrate and 0.2% sodium fluoride for topical use, applied to individual trays by the patient at home under professional follow-up.

PRODUCT INDICATION:

POTENZA ESENTE 0,2% is indicated for the treatment of teeth with low to moderate hypersensitivity. It was developed to attenuate dental hypersensitivity caused by thermal and chemical agents, especially during tooth bleaching.

PRECAUTIONS AND CONTRAINDICATIONS:

- Always keep away from heat and/or sunlight.
- Do not use if you are allergic to the components of the product.
- Do not ingest the product.
- In case of allergic reaction, refer the patient to emergency medical service with this document at hand.
- Non-injectable product.
- Do not reuse the packaging.

SIDE EFFECTS:

- Ingestion of large amounts of the product may cause nausea.

WARNING:

- Dental use only and as indicated by the dentist.
- Never use the product if it has expired.
- The product must be handled according to the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use.
- Keep out of reach of children.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.

- The manufacturer is not responsible for damage caused by improper use or improper handling.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

INSTRUCTIONS FOR USE FOR PATIENTS:

Step 1: Before performing dental bleaching, brush and floss, removing any dirt from the surface.

Step 2: With the aid of the tip applicator, apply the POTENZA ESENTE 0,2% desensitizing gel in the dental bleaching tray. Apply a small portion to the space corresponding to the teeth that will be bleached. Adapt the tray to the dental arch and press lightly to wrap the surface with the gel.

Step 3: Leave to act for 15 minutes or for the time recommended by your dentist. Remove the tray and rinse with water to remove all desensitizing gel from the dental surface.

IMPORTANT: Avoid swallowing the gel and contact your dentist if you have any questions.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE:

SEE PACKAGING.

SHELF LIFE:

TWO YEARS AFTER THE MANUFACTURING DATE.

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

**FOLLOW THE INSTRUCTIONS
FOR USE CORRECTLY**

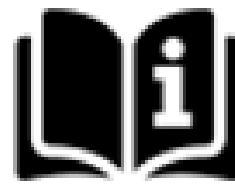
**INTENDED FOR
PRESCRIPTION USE
ONLY**

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