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PHS DO BRASIL LTDA

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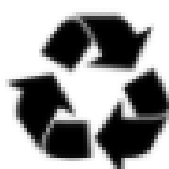
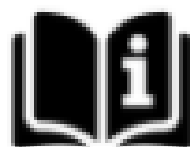
MI0010 – REV. 04

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INSTRUCTIONS FOR PROFESSIONAL USE

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville,
State: SC, Country: Brazil CEP [Zip Code]: 89203-212
CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279028

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC [Chemist License] #13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product's name: POTENZA ESENTE 2%

Product's Technical name: SUBSTANCE FOR THE TREATMENT OF DENTAL HYPERSENSITIVITY

READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.

KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Potassium Nitrate at 6%, Sodium Fluoride at 2%, humectant (glycerol), neutralizers, thickeners and deionized water.

PRODUCT'S DESCRIPTION:

POTENZA ESENTE 2% is a topical desensitizing gel, of low viscosity and neutral pH. It has a double desensitizing action based on Potassium Nitrate (neural action) and Sodium Fluoride (obliterating agent). It can be used before or after tooth whitening when the sensitivity is significant.

POTENZA ESENTE 2% was developed for exclusive use in a dental office. Due to the high concentration of Sodium Fluoride, we do not recommend home use.

PRODUCT'S INDICATION:

POTENZA ESENTE 2% is indicated for the treatment of teeth with low to moderate sensitivity. It was developed to relieve dental sensitivity triggered by thermal and chemical agents and also in the treatment of sensitivity caused by the treatment of tooth whitening.

PRECAUTIONS:

Always keep away from heat and/or sunlight. Do not use if you are allergic to product components. Do not ingest the product. In case of allergic reactions, advise the patient that he must inform his dentist, get the instruction manual and go directly to the Emergency Department. Product is not injectable. Do not reuse the packaging. POTENZA ESENTE 2% is contraindicated for people with a report of allergy to any of the components of the formula.

SIDE EFFECTS:

Ingesting large amounts of the product can cause nausea.

WARNINGS:

For dental use only and indicated by the dentist. Never use the product if it has expired. Exclusive for dental use and must be handled according to the instructions for use. It is the professional's responsibility to check before using the product that it is compatible and suitable for the intended use. Keep out of the reach of children. Before using, observe the appearance of the product, if you notice any anomaly, contact

the manufacturer immediately. The manufacturer is not responsible for damage caused by improper use or incorrect handling. Do not reuse the empty packaging for other purposes. To dispose of the product, follow local regulations.

INSTRUCTIONS FOR USE:

PROTOCOL FOR TREATING DENTAL HYPERSENSITIVITY:

There will be 4 sessions for treatment with a recommended interval of 48 hours between them.

1st Session: Perform 2 applications (the first application with the gingival retriever uncle and the second without the use of the gingival retriever uncle) of the POTENZA ESENTE 2% desensitizing gel.

1st Step: Insert the lip retractor. Carry out prophylaxis with pumice and water on the dental surface, eliminating any deposits that may prevent contact of the product on the teeth.

2nd Step: Insert the retractor wire inside the groove in the dental elements to be carried out (dry, without adding hemostatic solution, the wire having an exclusively mechanical action moving the gingival margin).

3rd Step: Apply the POTENZA ESENTE 2% desensitizing gel 2% uniformly on the teeth with a disposable micro-applicator and wait a minimum time of 5 minutes, rubbing every 2 minutes with each group of 3 or 4 teeth.

4th Step: Remove the gel and the retractor wire. Apply the desensitizer again to the teeth. Wait a minimum of 5 minutes, rubbing every 2 minutes with each group of 3 or 4 teeth. Totaling 10 minutes of application of POTENZA ESENTE 2% in the same session. Allow 48 hours for the second session.

2nd Session: Repeat the same steps as in the 1st Session.

3rd Session: Carry out the application of the Potenza Esente Glu-Hema obliterating desensitizer.

1st Step: Insert the lip retractor. Carry out prophylaxis with pumice and water on the dental surface, eliminating any deposits that may prevent contact of the product on the teeth.

2nd Step: Apply only on the exposed dentin area the Potenza Esente Glu-Hema for 45 seconds being careful not to rub on the gums and do not exceed the determined time, as it can cause irritation to the soft tissues. Wash and dry.

4th Session: Repeat the same steps as the 3rd Session.

PRE-WHITENING DESENSITIZATION PROTOCOL:

1st Step: Carry out prophylaxis with pumice and water on the dental surface, eliminating any deposits that may prevent the product from contacting the teeth.

2nd Step: Apply the POTENZA ESENTE 2% desensitizing gel 2% evenly over the teeth using the applicator tip or disposable microapplicator.

3rd Step: Let it act for 5 minutes, perform rubbing movements with the aid of the disposable microapplicator to each group of 3 or 4 teeth. Afterwards, aspirate the desensitizing gel from the dental surface with the aid of an aspirating cannula, wash with plenty of water and aspirate the water with a suction cup. It is important to advise the patient not to swallow the product.

4th Step: Only apply Potenza Esente Glu-Hema to the exposed dentin area for 45 seconds, taking care not to rub on the gums and do not exceed the determined time, as it can cause irritation to the soft tissues. Wash and dry.

5th Step: Start the clearing after the presented protocol. Follow the steps recommended by the manufacturer.

CONSERVATION AND STORAGE:

Keep the product in its original packaging. Protect from direct sunlight. Store the product at temperatures from 0° C to 30° C (32o F to 86o F).

TRANSPORTATION CONDITIONS:

The product can be transported at temperatures between -5° C and 50° C (23o F to 122o F). Protect from direct sunlight.

DATE OF MANUFACTURE:

SEE PACKAGE

EXPIRATION DATE: 2 YEARS AFTER MANUFACTURING DATE. SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.

WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.

**FOLLOW THE INSTRUCTIONS
FOR USE CORRECTLY**

**INTENDED FOR
PRESCRIPTION USE
ONLY**

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