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ARGILLA PRO

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PHS DO BRASIL LTDA

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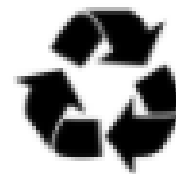
ARGILLA PRO

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, State: SC,

Country: Brazil CEP [Zip Code]: 89203-212

CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199279020

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC

[Chemist License] #13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product Trade Name: POTENZA SPECCHI ARGILLA PRO

Product Technical Name: PASTES FOR PROPHYLAXIS AND
DENTAL POLISHING

**READ CAREFULLY ALL THE INFORMATION IN THIS
INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.**

**KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER
INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

BASIC COMPOSITION:

White clay M635 (20 microns), aluminum oxide ALF 800 (18 microns), vehicle lubricant, thickeners and aroma.

PRODUCT DESCRIPTION:

POTENZA SPECCHI ARGILLA PRO is a clay-based polishing paste, a material that combines different minerals of different hardness and with extra fine granulation. It has medium viscosity and thixotropic effect, grayish color and mentholated aroma.

PRODUCT INDICATION:

POTENZA SPECCHI ARGILLA PRO is indicated for the final polishing of dental enamel, resins, ceramics, acrylics, metals and other restorative materials. It should be used in conjunction with felt discs in low rotation.

PRECAUTIONS:

- To avoid excessive heating of the dental surface, use light and intermittent pressure during polishing with the felt disc and paste.
- During application of the product, it is recommended that the professional and the patient wear safety goggles.
- Do not use in patients with allergy to the components of the formula.

CONTRAINDICATIONS:

POTENZA SPECCHI ARGILLA PRO is contraindicated for people with a report of allergy to any of the components of the formula.

INSTRUCTIONS FOR USE:

Polishing is the last step of dental restoration. Before polishing, make sure that the finishing and pre-polishing step has already been completed with the use of drills, sandpaper discs and polishing rubbers of different granulations. The quality of the restoration's brightness depends largely on the smoothness of the pre-polishing surface.

The **POTENZA SPECCHI ARGILLA PRO** paste must be used in combination with a disposable felt disc coupled with low rotation. It is recommended to use thin and flexible discs.

Step 1: Apply a small portion of the paste on the felt disc or directly on the surface of the restoration.

Step 2: In intermittent movements of slight pressure, cover the entire area of the restoration and the tooth to be polished. Usually 2 minutes is enough for a thorough polishing.

Step 3: Make sure the brightness level is satisfactory. If not, consider repeating the pre-polishing process and then repeat the polishing maneuver.

SIDE EFFECTS:

- Ingestion of large amounts of the product can cause nausea.
- Discontinue use of the product in case of allergic reactions.
- In case of allergic reaction, refer the patient to emergency medical service with these instructions for use of the product at hand.

WARNING:

- Respect the expiry date.
- Never use the product if it has expired.
- Keep out of reach of children.
- Dental use only.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.
- The manufacturer is not responsible for damage caused by improper use or improper handling.
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

The product can be transported at temperatures between -5 °C and 50 °C. Protect from the incidence of direct sunlight.

MANUFACTURING DATE:

SEE PACKAGING.

SHELF LIFE:

TWO YEARS AFTER THE MANUFACTURING DATE.

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

FOLLOW THE INSTRUCTIONS FOR USE CORRECTLY

INTENDED FOR PRESCRIPTION USE ONLY

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