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PHS DO BRASIL LTDA

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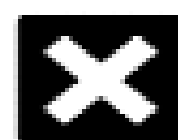
IRRITANTE

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Rua Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, State: SC,

Country: Brazil CEP [Zip Code]: 89203-212

CNPJ [Corporate Taxpayer's ID]: 17.910.389/0001-05

BRAZILIAN INDUSTRY

EMAIL: vendas@phsdobrasil.com.br

Operating License MS: X401M5619X7L(8.11.992-7)

ANVISA Registration #81199270017

Technical Officer: ÉLEN ALVES LOOS - CRQ/SC N. 13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

Product Trade Name: POTENZA STAMPO EVA

Product Technical Name: PLATE FOR MAKING TRAYS

READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Vinyl ethylene-acetate copolymer.

PRODUCT DESCRIPTION:

POTENZA STAMPO EVA is a plate made of EVA (vinyl ethylene acetate copolymer) of flexible, colorless and transparent appearance. The plate is 1.25mm thick and is plasticized, having an approximate thickness of 1mm.

PRODUCT INDICATION:

POTENZA STAMPO plates are indicated for the manufacture of individual trays, usually used for the application of home-use dental bleaching and drip for fluoridation.

The product is restricted to health professionals in the dental field (dentists and dental technicians).

PRECAUTIONS AND CONTRAINDICATIONS:

- Do not use in patients with any sensitivity to the components of the formulation.
- Discontinue use of the product in case of allergic reactions.
- If the patient has allergic reactions, the professional should refer them to emergency medical service with the instructions for use of the product at hand.
- Refer to the necessary instructions for use before carrying out the necessary operations.
- Avoid any irregular adaptations or improvisations.
- Check that the packaging has not been tampered with. If it has been, return it immediately to the manufacturer.
- Keep the product in its original packaging.
- SINGLE-USE product. It can be thermoplasticized only once.
- The plaster model to be used for the manufacture of the individual tray must be intact and with all well-defined structures.
- Do not use the product if it is not intact.
- Keep the product away from heat and flames.

WARNING:

- Respect the expiry date.
- Never use the product if it has expired.
- Keep out of reach of children.
- Dental use only.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.
- The manufacturer is not responsible for damage caused by improper use or improper handling.
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether it is compatible and suitable for the intended use.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

INSTRUCTIONS FOR USE:

Step 1: Obtain the plaster model of the patient's dental arches through alginate molding and common plaster.

Step 2: Attach the plate to the thermoplasticizer machine and center the plaster model below it. With the machine on, wait for the plate to heat until it has a bulging of approximately 2cm, evidencing its plasticization. Then lower the bulkhead and plate against the model by leaving the vacuum triggered for about 10sec. After the plate adjusts to the model format, the assembly should be removed from the machine and let cool naturally reaching room temperature.

Step 3: After cooling the tray, remove it from the model. Cut out its edges leaving approximately 1mm of coverage on the gingival margins. Make sure there are no sharp corners or burrs.

Step 4: Store the tray over the model until delivery to the patient and instruct them on the need to sanitize it before use.

Step 5: Instruct the patient on the use, cleaning, conservation and storage of the tray. During mouth use, instruct the patient not to eat or drink. After use, rinse under running water (at room temperature) until the gel is fully removed, then dry and store.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE:

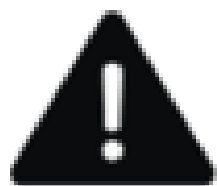
SEE PACKAGING.

SHELF LIFE:

FIVE YEARS AFTER THE MANUFACTURING DATE.

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.


**FOLLOW THE INSTRUCTIONS
FOR USE CORRECTLY**

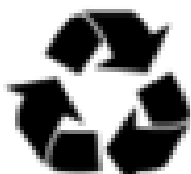**INTENDED FOR
PRESCRIPTION USE
ONLY**

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