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ÁCIDO FOSFÓRICO 37% FLOW

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PHS DO BRASIL LTDA

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ÁCIDO FOSFÓRICO 37% FLOW

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79036

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ATTACCO ACID CONDITIONER 37%.

Alternative Commercial Name: POTENZA ATTACCO ACID CONDITIONER FOR ENAMEL AND DENTIN.

Technical Product Name: ACID CONDITIONER.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

37% phosphoric acid, colorant, thickeners, humectants, and deionized water.

PRODUCT DESCRIPTION:

POTENZA ATTACCO 37% is an acid conditioner based on 37% phosphoric acid, available in the form of an intense blue gel. It comes in two versions: Selective and Flow.

In the **Selective** presentation, the gel is restricted only to the place where it is applied - without leaking to adjacent areas. Its very striking color allows easy visualization of the application site.

In the **Flow** presentation, due to the low viscosity, it allows a precise placement without draining to the adjacent areas. Its very striking color allows easy visualization of the application site.

PRODUCT INDICATION:

POTENZA ATTACCO 37% is indicated for acid etching of enamel and dentin, a step that precedes the application of adhesives. Its objective is to increase the microporosities in the tissues, consequently maximizing the mechanical retention of the adhesive.

PRECAUTIONS:

- Keep away from sunlight;
- Contact of the product with eyes and/or skin can cause burns;
- Ingestion of large amounts of the product may cause nausea;
- The professional should instruct the patient not to ingest the product. If the patient accidentally swallows a large volume of the product, the professional must take it to the Emergency Room along with the instructions for using the product;
- Non-injectable product;
- Do not reuse the packaging;
- At the slightest sign of allergy, suspend use immediately.

CONTRAINDICATIONS:

- POTENZA ATTACCO 37% is contraindicated for people with reported allergies to any of the components of the formula.

POSSIBLE SIDE EFFECTS:

- The permanence of the product on the tooth surface for longer than indicated may cause excessive tissue demineralization.

WARNING:

- Never use the product if it has expired;
- Exclusive for dental use;
- It must be handled in accordance with the instructions for use;
- It is the responsibility of the professional to check, before using the product, whether it is compatible and suitable for the intended use;
- Keep out of the reach of children;
- Before using, observe the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by inappropriate use or incorrect handling;
- Do not reuse the empty packaging for other purposes;
- To dispose of the product, follow the local legislation.

INSTRUCTIONS FOR USE:

1st STEP: Isolate the region to receive the product, preferably with absolute isolation. The substrate must be clean and dry.

2nd STEP: Check acid flow prior to intraoral application. Apply the product to the entire area where acid etching is desired. On enamel, let the product act for 30 seconds. On dentin, let the product act for 15 seconds.

Note: The pulp wall of deep cavity preparations must be protected with a liner base (eg: glass ionomer cement) before applying the product.

3rd STEP: Thoroughly wash the conditioned area with deionized water, at least for the same time as the application.

4th STEP: After complete removal of the acid, keep the substrate dry or moist as recommended by the adhesive system to be used.

Note: For certain self-etching adhesives, it is not recommended to apply the product to dentin.

CONSERVATION AND STORAGE:

- Keep the product in the original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures between 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5 °C e 50 °C (23 °F to 122 °F);
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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