Potenza

BLOCCO

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PHS DO BRASIL LTDA

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Código: IFU-003

Versão: 08

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

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Alternative Commercial Name: LIGHT-CURING GINGIVAL BARRIER.

Technical Product Name: LIGHT-CURING GINGIVAL BARRIER.

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Dimethacrylate groups, thickeners, antioxidants, humectants, catalysts, and pigments.

PRODUCT DESCRIPTION:

POTENZA BLOCCO is a light-curable resin used to protect soft tissues adjacent to the tooth, acting as a physical barrier that prevents contact of substances with soft tissue, thereby protecting it. Before polymerization, the product has the appearance of a flowable paste with static behavior. After curing, it becomes insoluble and rigid, while maintaining high resistance. The product may be available in blue or green color.

PRODUCT INDICATION:

It is indicated as a gingival barrier during in-office whitening procedures. It may also be used as a matrix stabilizer in restorations and as an aid in absolute isolation by stabilizing clamps and sealing margins.

PRECAUTIONS:

- Avoid applying the product on gingiva that has recently undergone surgical procedures or presents signs of inflammation or irritation;
- Dry the gingiva before application. Any moisture present on the surface may impair proper adhesion of the material;
- Before proceeding with whitening treatment, ensure that the product is fully cured and firmly adhered to the gingival tissue;
- Before light curing, the product may be irritating and may cause mild gingival irritation;
- Monitor for exaggerated responses and consider the possibility of an allergic reaction.

CONTRAINDICATIONS:

- Pregnant or breastfeeding patients, or patients under 15 years of age;
- Patients with periodontal diseases;
- Patients with incomplete root formation;
- Patients with known allergy to any component of the formula.

POSSIBLE SIDE EFFECTS:

- The product contains methacrylate monomers, which may cause allergic reactions or mild irritation in sensitive patients;
- Discontinue use in case of allergic reactions;
- In case of allergic reaction, discontinue use and seek medical advice.

WARNING:

- Observe the expiration date. Never use the product if it has expired;
- Keep out of reach of children;
- For dental use only;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;

- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is intended exclusively for dental use and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, prior to use, that the product is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;
- Reuse of gingival barriers may result in microbial contamination, as organic particles, saliva, blood, and other oral fluids may remain on the barrier after previous use. This contamination may pose a risk of cross-infection between patients;
- Light-curable gingival barriers contain materials designed to harden when exposed to light;
- After opening, the product must be stored according to the recommended conditions, keeping the packaging properly closed;
- During use, disposable applicator tips must be used, with mandatory replacement for each use;
- Contact of the syringe with potentially contaminated surfaces must be avoided;
Improper handling after opening may compromise the quality and performance of the product;
- For safe disposal after use, follow local regulations.

INSTRUCTIONS FOR USE:

Step 1: Dry the gingiva and the teeth corresponding to the area where the product will be applied;

Step 2: Apply the product directly from the syringe using the applicator tip. Cover the entire gingival margin, paying special attention to interpapillary areas. The barrier should have approximately 1 mm in height and 1 mm in thickness, slightly covering the cervical enamel margin of each tooth. It is recommended to divide the application into segments of 3 teeth;

Step 3: For an area corresponding to 3 teeth, light-cure the product for 30 seconds using sweeping motion, keeping the curing light tip* as close as possible to the material;

*The curing light must have a minimum power of 800 mW/cm² and a wavelength of 400 nm.

Step 4: Check whether the product covers the entire soft tissue surface, especially the interdental area. Uncovered areas must be corrected. Also ensure that the material is properly cured (rigid) and adhered to the tissue;

Step 5: Repeat the application (steps 2 to 4) on the gingival margins of the remaining teeth to be whitened;

Step 6: Proceed with whitening according to the specific whitening product instructions. At the end of the session, remove the gingival barrier using an explorer probe, gently displacing it until it detaches in large pieces or entirely.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 ° F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

ENSURE THAT THE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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