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PHS DO BRASIL LTDA

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Código: IFU-008

Versão: 09

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 81199279024

EU REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Trade Name: POTENZA IDRATA

Alternative Trade Name: ORAL MOISTURIZER

Product Technical Name: PROTECTIVE AGENTS

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Liquid petrolatum and polyethylene.

PRODUCT DESCRIPTION:

POTENZA IDRATA is a lubricating and moisturizing gel for topical application (skin or mucosa). The gel protects the application site against dryness, which may cause fissures in soft tissues. In addition, the product creates a protective film, preventing irritating or abrasive chemical agents from coming into direct contact with the skin/mucosa. POTENZA IDRATA does not contain medicinal substances in its composition.

PRODUCT INDICATION:

POTENZA IDRATA is indicated for topical application (skin and mucosa) in patients undergoing dental procedures (preventing lip dryness and providing patient comfort), such as when a lip retractor is used (photographic protocol, dental whitening, etc.) or in situations where the area may be exposed to irritating chemical agents.

POTENZA IDRATA may also be used with rubber dam to facilitate placement of absolute isolation on teeth.

PRECAUTIONS:

- Do not use in patients allergic to any components of the formulation;
- In case of suspected allergy, discontinue use immediately and refer the patient for medical care;
- Avoid contact with eyes.

CONTRAINDICATIONS:

- POTENZA IDRATA is contraindicated for individuals with allergy to any component of the formulation.

WARNINGS:

- Observe the expiration date;
- Do not use after the expiration date;
- Keep out of reach of children;
- For dental use only;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is intended exclusively for dental use and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, prior to use, that the product is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;
- For safe disposal of the product after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Step 1: Dispense the required amount of POTENZA IDRATA into a sterile dappen dish. Using cotton, a swab, or a gloved finger, take a small amount of gel, apply and spread it over the desired area until a thin film is formed;

Step 2: The treated area will acquire a smooth and shiny (moist) appearance. For long procedures, reapplication may be necessary to maintain the moisturizing effect;

Step 3: If removal of the gel is necessary, use gauze or cotton, gently rubbing the area.

STORAGE AND HANDLING:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGE

SHELF LIFE: 2 YEARS AFTER THE MANUFACTURING DATE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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