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AL

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PHS DO BRASIL LTDA

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Versão: 09

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AL

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79026

EU

REP

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA SPECCHI AL.

Alternative Commercial Name: POLISHING PASTE.

Technical Product Name: DENTAL PROPHYLAXIS AND POLISHING PASTES.

**PLEASE READ ALL THE INFORMATION IN THIS INSTRUCTIONS FOR USE CAREFULLY
BEFORE APPLYING THE PRODUCT.**

**KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE
PRODUCT WITH YOUR PATIENT.**

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Aluminum oxide ALF 800 (6.5 µm), lubricating vehicle, thickeners, and flavoring.

PRODUCT DESCRIPTION:

POTENZA SPECCHI AL is a polishing paste based on extra-fine aluminum oxide. It has medium viscosity and a thixotropic effect, with a grayish color and mint flavor.

PRODUCT INDICATION:

POTENZA SPECCHI AL is indicated for the final polishing of resins and other restorative materials. It should be used in conjunction with felt discs at low speed.

PRECAUTIONS:

- To avoid excessive heating of the dental surface, use light and intermittent pressure during polishing with the felt disc and paste;
- The use of safety glasses is recommended for both the professional and the patient during application;
- Do not use in patients allergic to any components of the formula.

CONTRAINDICATIONS:

- POTENZA SPECCHI AL is contraindicated for individuals allergic to any of the ingredients in the formula.

POSSIBLE SIDE EFFECTS:

- Ingesting large amounts of the product may cause nausea;
- Discontinue use of the product if allergic reactions;
- In case of allergic reaction, contact a physician, pharmacist, or the manufacturer, with this Instructions for Use available.

WARNING:

- Observe the expiration date;
- Never use the product if the expiration date has passed;
- Keep out of reach of children;
- For dental use only;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;

- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is intended solely for dental use and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, prior to use, that the product is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;
- Reuse of polishing pastes may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during prior use and grow under ideal storage conditions. This may pose a risk of cross-contamination to the patient;
- Dental polishing pastes are formulated with abrasive particles and other active ingredients that may deteriorate after use;
- Repeated reuse of polishing pastes may lead to loss of proper consistency and texture, which may impair the product's polishing performance.
- For safe disposal of the medical device after use, follow local regulations.

INSTRUCTIONS FOR USE:

Polishing is the final step of a dental restoration. Before polishing, ensure that finishing and pre-polishing steps have already been completed using burs, abrasive discs, and polishing rubbers of different grits. The quality of the final gloss depends largely on the smoothness of the pre-polished surface.

POTENZA SPECCHI AL paste must be used in association with a disposable felt disc attached to a low-speed handpiece. The use of thin and flexible discs is recommended.

Step 1: Apply a small amount of paste onto the felt disc or directly onto the restoration surface;

Step 2: Using intermittent movements with light pressure, cover the entire restoration and tooth surface to be polished. Typically, 2 minutes are sufficient for thorough polishing;

Step 3: Check whether the desired gloss level has been achieved. If necessary, repeat the pre-polishing and polishing process.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C;
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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