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AL UNODUE

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PHS DO BRASIL LTDA

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Versão: 09

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AL UNODUE

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

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Alternative Trade Name: POLISHING PASTE

Product Technical Name: DENTAL PROPHYLAXIS AND POLISHING PASTES

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Aluminum oxide ALF 240 (44.5 µm for UNO), aluminum oxide ALF 600 (9.3 µm for DUE), lubricating vehicle, thickeners, and flavoring.

PRODUCT DESCRIPTION:

POTENZA SPECCHI AL UNO DUE are aluminum oxide-based polishing pastes available in two grit sizes, with UNO being coarser and DUE finer. The pastes have medium viscosity and a thixotropic effect, with a grayish color and mint flavor.

PRODUCT INDICATION:

POTENZA SPECCHI AL UNO is indicated for pre-polishing of resins and other restorative materials, while POTENZA SPECCHI AL DUE is recommended for final polishing. Both should be used in conjunction with felt discs at low speed.

PRECAUTIONS:

- To avoid excessive heating of the dental surface, apply light and intermittent pressure during polishing with the felt disc and paste;
- The use of safety glasses is recommended for both the professional and the patient during application;
- Do not use in patients allergic to any components of the formulation.

CONTRAINDICATIONS:

- POTENZA SPECCHI AL UNO DUE are contraindicated for individuals with allergy to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- Ingestion of large amounts of the product may cause nausea;
- Discontinue use in case of allergic reactions;
- In case of allergic reaction, discontinue use and seek medical advice. Adverse events must be reported to the manufacturer and the competent authority.

WARNINGS:

- Observe the expiration date;
- Do not use after the expiration date;
- Keep out of reach of children;
- Before use, inspect the product. If any abnormality is observed, contact the manufacturer immediately;
- This product is intended exclusively for dental use and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, prior to use, that the product is compatible and suitable for the intended application;
- In case of serious incidents related to the use of the product, they must be reported to the

manufacturer and the competent authority;

- Do not reuse empty packaging for other purposes;
- This product is supplied in a multi-dose package. To prevent microbial contamination, infection control measures must be adopted during use, including the use of disposable accessories or applicators and the non-reuse of any dispensed material;
- Contact of the product or its packaging with potentially contaminated surfaces must be avoided in order to prevent cross-contamination between patients;
- Dental polishing pastes contain abrasive particles and other components that may undergo changes after opening, especially if improperly stored;
- Changes in the physical characteristics of the product, such as consistency, texture, or performance, may indicate compromised quality and may impair polishing effectiveness;
- For safe disposal of the medical device after use, follow applicable local regulations.

INSTRUCTIONS FOR USE:

Polishing is the final step of a dental restoration. Before polishing, ensure that finishing and pre-polishing steps have already been completed using burs, abrasive discs, and polishing rubbers of different grits. The quality of the final gloss depends largely on the smoothness of the pre-polished surface.

POTENZA SPECCHI AL UNO DUE pastes must be used in association with a disposable felt disc attached to a low-speed handpiece. The use of thin and flexible discs is recommended.

Step 1: For pre-polishing, apply a small amount of POTENZA SPECCHI AL UNO paste onto the felt disc or directly onto the restoration surface;

Step 2: Using intermittent movements with light pressure, cover the entire restoration and tooth surface to be polished. Typically, 2 minutes are sufficient for thorough polishing;

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Step 4: Check whether the desired gloss level has been achieved. If necessary, repeat the pre-polishing and polishing process.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from direct sunlight;
- Store at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between -5 °C and 50 °C (23 °F to 122 °F);
- Protect from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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