Potenza

ATTACCO

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PHS DO BRASIL LTDA

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Potenza

ATTACCO

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

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Alternative Commercial Name: POTENZA ATTACCO ACID CONDITIONER FOR ENAMEL AND DENTIN.

Technical Product Name: ACID ETCHANT OF PHOSPHORIC ACID AT 35%.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Phosphoric acid at 35%, dye, thickeners and deionized water.

PRODUCT DESCRIPTION:

POTENZA ATTACCO is a thixotropic gel based on phosphoric acid at 35%. It is indicated for acid etching of enamel and/or dentin, increasing bond strength of resin materials to the tooth's surface. Its proper viscosity allows easier handling as the gel does not run off from the area of application. The product's blue color helps identifying the application area. Potenza Attacco 35% provides an etching depth of about 2,0µm.

PRODUCT' INDICATION:

POTENZA ATTACCO is indicated for etching of enamel and/or dentin prior to adhesive application.

PRECAUTIONS:

- Keep away from sunlight;
- The contact of the product with eyes and/or skin can cause burning;
- Ingestion of big quantities of the product can cause nausea. The professional should instruct the patient not to swallow the product. If the patient swallows a big quantity of it and present any side effect, the professional should take him/her to get medical assistance, along with this instruction's manual;
- Do not reuse the packaging;
- At the slightest sign of allergy, discontinue use immediately.

CONTRAINDICATIONS:

- POTENZA ATTACCO 35% is contraindicated in patients allergic to any of the components of the formula.

POSSIBLE SIDE EFFECTS:

- Remaining the product on the tooth surface for longer than indicated may cause excessive tissue demineralization.

WARNING:

- Never use the product if it has expired;
- Exclusive for dental use;
- It must be handled in accordance with the instructions for use;

- It is the professional's responsibility to check, before using the product, whether it is compatible and suitable for the intended use;
- Keep out of reach of children;
- Before using, observe the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by inappropriate use or incorrect handling;
- Do not reuse empty packaging for other purposes;
- To safely dispose of DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

1st STEP: Clean and dry the dental surface to be etched. Control the area with relative or total dam isolation;

2nd STEP: Apply the product to the entire area where acid conditioning is desired. Leave it for 30 seconds on enamel and 15 seconds on dentin.;

Note: The pulp wall of deep cavity preparations must be protected with GIC or calcium hydroxide before Potenza Attacco application.

3rd STEP: Rinse the etched area abundantly.

4th STEP: Dry enamel and leave dentin moist (to avoid collapse of the collagen fibers).

Note: Depending on the chosen adhesive system (i.e. self-etching adhesives), Potenza Attacco 35% must not be applied to dentin.

CONSERVATION AND STORAGE:

- Keep the product in its original package;
- Protect it from direct sunlight;
- Keep it in temperatures between 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5 °C e 50 °C (23 °F to 122 °F);
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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