Potenza

ATTACCO

HF

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PHS DO BRASIL LTDA

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Versão: 06

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HF

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.70019

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA ATTACCO HF.

Alternative Commercial Name: ACID ETCHANT FOR CERAMICS.

Technical Product Name: ACID CONDITIONERS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Fluoridic Acid at 5 and 10%, Dye, Thickeners and Deionized Water.

PRODUCT DESCRIPTION:

POTENZA ATTACCO FLUORIDIC ACID CONDITIONER is found in concentrations of 5 and 10%. It is a thixotropic gel used for surface conditioning of dental porcelains. The conditioning allows the adhesion of feldspar porcelain, ceramic glazing and fixation of orthodontic brackets and repairs of porcelain with composites.

PRODUCT INDICATION:

Indicated for conditioning of dental porcelain, treatment carried out before silanization and application of adhesives. It aims to increase adhesion in restoration processes. Conditioning is essential for an optimal interaction between resin cement and ceramic restoration to occur, thus favoring greater retention and resistance.

PRECAUTIONS:

- Keep away from sunlight;
- The contact of the product with eyes and/or skin can cause burning;
- Ingestion of big quantities of the product can cause nausea;
- This product should only be used extraorally, on prosthetic pieces that are not yet cemented to the tooth;
- Non-injectable product;
- Do not reuse the packaging.

CONTRAINDICATIONS:

- POTENZA ATTACCO FLUORIDIC ACID CONDITIONER is contraindicated for individuals with allergic to any of the components of the formula.

POSSIBLE SIDE EFFECTS:

- Contact with soft tissues may cause irritation, and accidental ingestion of the product may lead to fluoride poisoning;
- In case of an allergic reaction, contact a doctor, pharmacist, or manufacturer, providing this instruction for use.

WARNING:

- Never use the product if it is past its expiration date;
- For dental use only, and must be handled according to the instructions for use;
- It is the responsibility of the professional to verify, before use, if the product is compatible and suitable for the desired application;
- Keep out of reach of children;

- Before use, check the appearance of the product. If you notice any abnormalities, contact the manufacturer immediately;
- The manufacturer is not responsible for any damage caused by improper use or incorrect handling;
- In case of serious incidents related to the use of the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- The quality and physical stability of the hydrofluoric acid-based conditioner may deteriorate with reuse, leading to changes in viscosity and texture of the product;
- For the safe disposal of DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

The treatment of the internal surface of the prosthetic parts must follow the indication according to the instructions of the porcelain manufacturer.

SURFACE	CONDITIONING TIME FOR POTENZA ATTACCO HF 5%	CONDITIONING TIME FOR POTENZA ATTACCO HF 10%
Feldspathic Porcelain	-	60s – 120s
Glass Ceramics Reinforced with Leucite	-	60s
Lithium Disilicate Based Glass Ceramics	20s	-
Zirconia	UNCONDITIONABLE	UNCONDITIONABLE

After conditioning with Hydrofluoric Acid, proceed as follows:

1st STEP: Wash with plenty of water until the product is completely removed and dry well.

2nd STEP: Proceed with silanization according to the manufacturer's instructions.

3rd STEP: Apply the adhesive system.

Observation: Conduct the dental substrate with POTENZA ATTACCO 35% PHOSPHORICACID CONDITIONER. The conditioning of the dental substrate can vary according to the type of adhesive system, selective enamel conditioning can be performed, in the case of self-etching adhesive system. The choice of composite (in the case of repair) or resin cement is also the professional's choice.

CONSERVATION AND STORAGE:

- Keep the product in its original package;
- Protect it from direct sunlight;
- Keep it in temperatures between 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5 °C to 50 °C (23 °F to 122 °F).
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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