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PF

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PHS DO BRASIL LTDA

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Versão: 08

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PF

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79017

Product Commercial Name: POTENZA BIANCO CARBAMIDE PEROXIDE PF 35%.

Alternative Commercial Name: POTENZA BIANCO PF DENTAL WHITENING FOR USE IN OFFICES.

Technical Product Name: DENTAL WHITENING AGENTS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

35% carbamide peroxide, carbomer, potassium nitrate, sodium fluoride, wetting (glycerol), neutralizers and deionized water.

PRODUCT DESCRIPTION:

POTENZA BIANCO PF 35% is a professional-use bleaching agent for office application based on 35% Carbamide Peroxide. The product looks like thixotropic and colorless gel, ready for use directly from the syringe.

PRODUCT INDICATION:

POTENZA BIANCO PF 35% is indicated for dental bleaching in the office. It can be used in vital and non-vital teeth that show color changes. The product is also indicated as an adjunct in the treatment of mild cases of staining due to fluorosis or to the use of tetracycline and its derivatives.

PRECAUTIONS:

- If the patient reports previous hypersensitivity during the daily routine (e.g., ingestion of cold liquids, cold air or pressure, etc.) and there is no apparent cause that leads to discomfort, perform the desensitizing protocol (Potenza Esente 2% + Potenza Esente GLU-HEMA) prior to bleaching;
- In cases of major gingival recession that cannot be treated previously with temporary restoration, isolate the area with the gingival barrier before starting bleaching;
- Dental surfaces receiving the bleaching gel must be clean (free of calculation or plate). If necessary, prophylaxis/scraping should be performed at least 5 days prior to bleaching;
- It is essential that the bleaching gel does not contact soft tissues during bleaching, as this may result in transient or even permanent damage. Therefore, ensure the correct application of the gingival barrier and use an efficient lip reformer during the procedure. If there is still contact of the gel with soft tissue, apply the neutralizing solution immediately. In severe cases, contact the manufacturer of the product for assistance (see "Contact" on the product packaging);
- If the patient has a high level of hypersensitivity during the session, consider discontinuing it and apply the desensitizing agent Potenza Esente 2% for 10 minutes;
- Some patients may not be eligible for in-office bleaching and will require treatment with less concentrated home-use gel.

CONTRAINDICATIONS:

Before starting bleaching, take detailed case history of the patient. Identify habits, general conditions, chronic and acute diseases. On clinical examination, any transient change (A) should be treated prior to initiation of treatment. Some conditions (B) make bleaching absolutely impossible.

A) TRANSIENT CONTRAINDICATIONS - The following should be treated before starting treatment: Caries, infiltrated restorations, dental cracks/fractures, periodontal problems (e.g., gingivitis or periodontitis), presence of plaque/calculus, teeth with dubious vitality or untreated endodontic problems, teeth with large gingival recessions.

B) ABSOLUTE CONTRAINDICATIONS - Treatment should not be performed in the following cases: Teeth presenting amelogenesis or dentinogenesis imperfecta, teeth affected by severe fluorosis or tetracycline staining, teeth with relevant mobility or bone loss, teeth with incomplete rhizogenesis, individual under 15 years of age, lactating, pregnant women, patients undergoing treatment of serious diseases or patients with serious decompensated diseases, allergic to any of the components of the formulation.

POSSIBLE SIDE EFFECTS:

- It is common for the patient to experience a certain degree of hypersensitivity during the session or up to 2 days after the session. This is due to the product's own mechanism of action. The level of discomfort varies greatly from one patient to another. Desensitizing agents (e.g., Potenza Esente 2% + Potenza Esente GLU-HEMA) should be associated with treatment for greater patient comfort. In extreme cases, it is suggested to discontinue treatment;
- Bleaching may highlight areas of pre-existing hypocalcification (hypoplasia's) due to dehydration or even more intense bleaching of these areas. In most cases, these spots disappear after rehydration or remineralization of the region, which occurs within a few hours or days after the bleaching session. Whenever you identify the spots, suspend treatment until they are gone. If the stains are persistent, discuss the continuity or not of treatment with the patient, informing them of the possible need for restorative intervention.
- Some patients may have non-uniform bleaching in the different thirds of the teeth. This occurs for a physiological issue and will hardly be changed, because in the cervical region of the teeth there is greater thickness of dentin, which is naturally more stained than the predominant enamel in the other thirds.

WARNING:

- Respect the expiry date. Never use the product if it has expired;
- Keep out of reach of children;
- Dental use Only;
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or improper handling;
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use. It is the responsibility of the professional to prove, before the use of the product, whether this material is compatible and suitable for the intended use;
- Do not reuse the empty packaging for other purposes;
- To dispose of the product, please follow local legislation.

INSTRUCTIONS FOR USE:

Before using the product, read all the items in this document. Take detailed case history and make clinical examination of the patient and resolve any current problems before starting treatment (see contraindications and precautions).

During the session, if you notice undue contact of the soft tissue bleaching gel or excessive dental hypersensitivity, stop the session and take appropriate action as previously recommended in this document.

1st STEP: Select and record the color of the patient's teeth using a colors cale. Photographic record is suggested.

2nd STEP: Apply the Potenza Idrata moisturizing base to the patient's lips (especially in commissures) to moisturize, protect and provide greater comfort. Then position a lip repressor to facilitate access to teeth.

3rd STEP: Perform the pre-bleaching desensitization protocol:

3.1. Apply the Potenza Esente 2% desensitizer on the vestibular face of the teeth to be bleached, leaving it for 10 minutes. During this period, perform frictional movements with the aid of a disposable microapplicator. Then wash with jets of water/air (always with the aid of the sucker to avoid swallowing) and dry your teeth. If the teeth have exposed dentin, apply Potenza Esente GLU-HEMA with the aid of a disposable brush on the exposed root and let the product act for 45seconds, taking care to avoid contact of the product with the gingival margin. Then wash and dry the surfaces.

4th STEP: Dry the gingival tissue appropriately and apply the Potenza Blocco gingival barrier to the marginal gum of all teeth to be bleached. The layer should be approximately 1 mm thick and 5 mm wide, completely covering the gingival papillae. Also cover about 0.5 mm of the cervical tooth (enamel). Then, light cure the gingival barrier for 20 to 30 seconds for each group of 3 teeth. Finally, check if the barrier became rigid and is firmly adhered to the gum before proceeding.

IMPORTANT: With a clinical mirror, check from incisal to cervical if there is uncovered gingival tissue. Make correction, if necessary.

5th STEP: The product is presented in a ready-to-use syringe, without the need for mixing. Just attach the applicator tip to the syringe.

6th STEP: Apply a small portion of the gel on the dental surface. Fully cover the vestibular surface of the teeth to be bleached, including the interproximal ones. Extend slightly to the incisal/occlusal faces. The gel layer should be approximately 1mm thick. Use a disposable microapplicator or exploitative probe to facilitate the distribution of the gel on the dental surface.

7th STEP: Let the gel act for 35-45 consecutive minutes on the teeth. During this period, if necessary, redistribute the gel over the surface. Then vacuum the entire gel contents of the teeth with a surgical or endodontic cannula and remove as much gel remnant as possible with cotton or gauze. Wash appropriately and vacuum the content.

8th STEP: Remove the gingival barrier by separating it forward with the aid of an exploitative probe. Rewash the dental surface with air/waterjets, always using the sucker concomitantly (instruct the patient not to ingest the water with the bleaching gel residue).

9th STEP: To complete the session, polish the teeth with felt discs associated with the Potenza Specchi PRO paste (diamond polishing paste). The bleaching result of the session will be better viewed after 1week. Usually 2 to 3 sessions (spaced in 1 week each) are required for a complete treatment, but this can be made more flexible according to each patient.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging.
- Keep the product in its original packaging.
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C and 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

SHELF LIFE: 2 YEARS AFTER THE MANUFACTURING DATE.

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.

MAKE SURE THAT THE INSTRUCTION FOR USE IS IN ACCORDANCE WITH THE PRODUCT TO BE USED.

TO ACCESS THE PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

IMPORTANT: DISCONTINUE USE OF THE PRODUCT IN CASE OF ALLERGIC REACTIONS.

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