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H2O2

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PHS DO BRASIL LTDA

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Versão: 08

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H2O2

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.70022

Product Commercial Name: POTENZA BIANCO HYDROGEN PEROXIDE 3%, 6%, 7.5%, 9.5%.

Alternative Commercial Name: POTENZA BIANCO H2O2 DENTAL WHITENING FOR HOME USE.

Technical Product Name: DENTAL WHITENING AGENTS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

**BASIC COMPOSITION:**

Hydrogen peroxide, thickeners, deionized water, desensitizers and humectant.

PRODUCT DESCRIPTION:

POTENZA BIANCO H₂O₂ is a bleaching agent for home use, supervised by the dentist, based on hydrogen peroxide at 3%, 6%, 7.5% or 9.5%. The product has an aspect of thixotropic colorless gel and is presented in syringes that are ready to use. Its formulation contents potassium nitrate and sodium fluoride that act dually desensitizing the teeth, providing more comfort to the patient. Besides, the product has high content of water and has the humectant Glycerol, that prevents dental dehydration.

PRODUCT INDICATION:

POTENZA BIANCO H₂O₂ is indicated for home bleaching, supervised by the dentist. It can be used for vital and non-vital teeth (externally) that present shade alterations. The product is also indicated for mild cases of fluorosis or tetracycline staining.

PRECAUTIONS:

- Keep away from sunlight;
- The contact of the product to eyes and/or skin can cause burning;
- Ingestion of high quantities of the product can cause nausea. Professional must tell the patient not to swallow product. If patient accidentally swallows high volume of the product, professional must take him to medical support with this instructions document;
- This product is not for injection;
- Do not reuse the package;
- If there is any suspect of allergy, discontinue using the product immediately;
- If the patient reports pre-existing hypersensitivity during his daily routine (i.e. consumption of cold liquids, cold air or pressure, etc.) and there is no obvious cause that leads the patient to this discomfort, apply the desensitizing (Potenza Esente 0,2% + Potenza Esente GLU-HEMA) previously to bleaching;

- In cases of important gingival recession, restore the region with provisional material (i.e. GIC) before using bleaching agent;
- Dental surfaces that will receive the bleaching gel must be clean (absence of calculus or plaque). If necessary, dental prophylaxis must be performed with minimum of 5 days before bleaching;
- It is essential for the bleaching gel to contact soft tissue as minimum as possible, because it can lead to irritation in different magnitudes according to period of exposure, concentration of the product and gingival phenotype of the patient. Instruct patient to apply small amount of gel (half of a rice grain) on the space corresponding to each tooth. This will avoid excessive overflow through the edges of the custom tray;
- If the patient present high level of hypersensitivity while using the product, reduce bleaching gel concentration to 10% and instruct the patient to use desensitizing agent (POTENZA ESENTE 0,2%) for 10 minutes, before and after bleaching applications daily.

IMPORTANT:

- Dental bleaching must be supervised by a dentist;
- The efficacy of the treatment depends on the correct usage of the product;
- The efficacy of the treatment varies from one patient to another according to individual factors (i.e. type of stains, level of tissue permeability, intrinsic color of teeth, etc.).
- In some cases, it is suggested to combine bleaching approaches (home and office use).

CONTRAINDICATIONS:

Before initiating bleaching, perform detailed anamnesis of the patient. Identify habits, general conditions, chronical and acute diseases. In the clinical exam, any transitory (A) alterations must be treated previously bleaching. Some conditions (B) contraindicate bleaching in an absolute way.

A) TRANSITORY CONTRAINDICATIONS – solve before bleaching: Caries, leaked restorations, dental cracks or fractures, periodontal diseases (i.e. gingivitis or periodontitis),

presence of plaque or calculus, teeth with doubtful pulp vitality or showing non treated endodontic problems, teeth with big gingival recession.

B) ABSOLUTE CONTRAINDICATIONS – do not perform bleaching: Teeth showing amelogenesis or dentinogenesis imperfecta, teeth stricken by severe fluorosis or tetracycline staining, teeth with relevant bone loss or mobility, teeth with incomplete rhizogenesis, individuals under 15 years old, pregnant or breastfeeding women, patient treating severe disease or those with unbalanced diseases, allergic to any of the formula contents.

POSSIBLE SIDE EFFECTS:

- It is common for the patient to present a certain degree of hypersensitivity during or after the application of the whitening gel. This is due to the product's own mechanism of action. The level of discomfort varies greatly from patient to patient. The desensitizing agent Potenza Esente 0.2% should be associated with the treatment for greater patient comfort. In extreme cases, it is suggested to stop the treatment;
- The bleaching can highlight pre-existing areas of hypocalcification (hypoplasia) due to dehydration or even more intense bleaching of these areas. In most cases, these spots disappear after rehydration or remineralization of the region, which occurs within a few hours or days after using the bleaching agent. Whenever you identify the stains, suspend the treatment until they have disappeared. If the stains are persistent, discuss the continuation or not of the treatment with the patient, informing him of any need for restorative intervention;
- Some patients may have non-uniform whitening in different thirds of the teeth. This is due to a physiological issue and will hardly be changed, because in the cervical region of the teeth there is a greater thickness of dentin, which is naturally more stained than the predominant enamel in the other thirds.

WARNING:

- Respect the expiration date. Never use the product if it has expired;
- Keep out of the reach of children;
- Only for dental use;
- Before using, observe the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;

- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use. It is the professional's responsibility to check before using the product if this product is compatible and suitable for the intended use;
- Do not reuse empty packaging for other purposes; To carry out the disposal of the product, follow the local legislation.

INSTRUCTIONS FOR USE:

Before using the product, read all items in this document. Make a detailed history and clinical examination of the patient and solve any existing problems before starting treatment (see contraindications and precautions).

SUGGESTION OF USAGE TIME ACCORDING TO GEL CONCENTRATION HYDROGEN PEROXIDE / DAILY USE TIME

6%	2 HOURS
7,5%	1,5 HOURS
9,5%	1 HOUR

Attention: The dentist must monitor the patient throughout the treatment carried out at home. Weekly return visits for clinical follow-up are suggested.

1st STEP: Select and register the shade of the patient's teeth using a shade guide. Photographs are suggested;

2nd STEP: obtain the patient's plaster models through impressions with alginate and common plaster. Cut out the models in a "horseshoe" shape, removing the palatal area (upper model) and the tongue area (lower model);

3rd STEP: With the model already cut out, submit it to a vacuum forming machine. In this machine, place a 1.25mm EVA plate (Potenza Stampo) and turn on the heating. When you

notice a 2cm bulging (plastification), lower the plate against the model to activate the vacuum and conform the tray;

4th STEP: Allow the plate to cool down until it reaches room temperature. Then, separate the model from the tray and cut out the individual tray as desired, with small scissors. It is suggested for the tray edges to cover up to 1mm of the gingival margin. Remove any sharp edges that might bother gingiva or lips;

5th STEP: Instruct the patient to use the tray. The whitening gel must be applied directly from the syringe, with the aid of the applicator tip, in the spaces of each tooth to be whitened (usually from 2nd premolar to another).

The inner wall of the tray that corresponds to the buccal surface of the tooth must receive an amount of gel with a volume proportional to half of a grain of rice; deposited in the center of the space;

6th STEP: Instruct the patient to fit the loaded tray to the dental arch. Any excess that overflows the edges must be removed with moistened fingers;

7th STEP: After the recommended use time, instruct the patient to remove and wash the tray under running water at room temperature. After removing all the whitening gel and saliva residues, the tray should be dried with a paper towel and stored in an appropriate case, in a place free from humidity, heat and direct sunlight.

NOTE: If the patient reports that they are swallowing gel during use or their gingiva show signs of inflammation, consider to cut lower the tray edges and instruct them to apply a smaller amount of gel in the tray.

GENERAL SAFETY INSTRUCTIONS:

- Product handling must be performed exclusively in a dental office;
- The professional must use all the protection equipment's during session, most importantly protective goggles and gloves;
- The patient must wear disposable cap, protective goggles and cloth covering his clothes;

- The carbamide peroxide inside the syringe may be under pressure if stored or transported in adverse conditions. Before opening the package, protect hands and eyes and observe if there is any leakage;
- Avoid contact of the product to skin, mucosa or clothes;
- The carbamide peroxide inside the syringe may be under pressure if stored or transported in adverse conditions. Before opening the package, protect hands and eyes and observe if there is any leakage;
- Avoid contact of the product to skin, mucosa or clothes;
- Instruct the patient not to swallow the product. While using the tray with the gel, instruct him not to drink or eat;
- If the gel contacts skin or mucosa intraorally, clean the area abundantly with water;
- In case the gel contacts the eye, rinse abundantly with water and lead the patient to medical assistance with these instructions;
- If patient present allergic reactions, the treatment must be suspended immediately. Lead the patient to medical assistance with these instructions;
- Expired leftovers of gel must not be used. Instruct the patient to follow correctly local legislation on how to discard it properly;

CONSERVATION AND STORAGE:

- Keep the product in its original package;
- Protect the product from direct sunlight.
- Keep the product in temperatures between 0 °C a 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product must be transported in temperatures between -5 °C a 50 °C (23 °F to 122 °F).
- Protect the product from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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