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PHS DO BRASIL LTDA

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Código: IFU-003

Versão: 06

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

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Alternative Commercial Name: LIGHT-CURING GINGIVAL BARRIER.

Technical Product Name: LIGHT-CURING GINGIVAL BARRIER.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Dimethacrylate groups, BHT, catalizers, pigments and dyes.

PRODUCT DESCRIPTION:

POTENZA BLOCCO is a light-curing resin indicated to protect gingival margins during in office bleaching, once hydrogen peroxide may injury soft tissues in a situation of direct contact. POTENZA BLOCCO is also indicated to improve sealing of total dam, mainly for cases of complex restorations/cementations and endodontic treatments.

PRODUCT INDICATION:

Indicated to protect gingival tissue from direct contact to harmful substances (i.e. hydrogen peroxide) and also to improve sealing of total dam isolation.

PRECAUTIONS:

- Avoid applying the product to the gum tissue if it has recently undergone surgical procedures, shows signs of inflammation, or irritation;
- Dry the gums before applying the product. Any moisture on the surface may prevent proper adhesion of the material;
- Before proceeding with a tooth whitening procedure, ensure that the product has fully cured and is firmly adhered to the gum tissue;
- Before light curing, the product may cause mild irritation and can irritate the gums. Be alert to exaggerated reactions and consider the possibility of an allergic process in such cases.

CONTRAINDICATIONS:

- Pregnant, lactating patients or patients under 15 years old;
- Patients with periodontal pathologies;
- Patients with incomplete rhizogenesis;
- Patients with allergic reaction to the components of the formula.

POSSIBLE SIDE EFFECTS:

- The product contains methacrylate monomers that may cause allergic reactions or mild irritations in patients sensitive to these substances;
- Discontinue use if allergic reactions occur;
- In case of an allergic reaction, contact a doctor, pharmacist, or manufacturer, along

with this user instruction.

WARNING:

- Respect the expiration date. Never use the product if the expiration date has passed;
- Keep out of reach of children;
- For dental use only;
- Before using, inspect the product. If you notice any abnormalities, contact the manufacturer immediately;
- The manufacturer is not responsible for damages caused by improper use or incorrect handling;
- This product is manufactured solely for dental use and should be handled according to the instructions for use;
- It is the professional's responsibility to verify that the product is suitable and compatible for the intended use before application;
- In case of a serious incident related to the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing gingival barriers may result in microbial contamination, as organic particles, saliva, blood, and other oral fluids can remain on the barrier after previous use. This contamination could pose a risk of cross-infection between patients;
- Photopolymerizable gingival barriers contain materials designed to harden when exposed to light. With reuse, the effectiveness of the barrier's hardening may decrease, making it less effective in providing an appropriate seal. If a reused gingival barrier does not provide effective isolation, there may be an increased risk of chemicals used in the dental procedure coming into contact with the gums;
- To dispose of the DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

1st STEP: Dry the gums and the teeth corresponding to the area where the product will be applied.

2nd STEP: Apply the product directly from the syringe using the applicator tip. Cover the entire gingival margin, paying special attention to the interpapillary areas. The barrier should be approximately 1 mm in height and 1 mm in thickness, slightly covering the cervical enamel

margin of each tooth. It is recommended to divide the application into segments of 3 teeth at a time.

3rd STEP: For an area corresponding to 3 teeth, light cure the product for 30 seconds in a sweeping motion, keeping the light tip of the curing light* as close as possible to the product.

*The curing light must have a minimum power of 800mW/cm² and a wavelength of 400nm.

4th STEP: Check that the product covers the entire surface of the soft tissue, especially the interdental area. Any uncovered areas should be repaired. Also, ensure that the material is properly cured (rigid) and adhered to the tissue.

5th STEP: Repeat the application (steps 2 to 4) on the gingival margin of the other teeth to be whitened.

6th STEP: Proceed with the whitening procedure according to the specific instructions for the whitening agent. Once the session is complete, remove the gingival barrier with the aid of an explorer probe, gently moving it until it detaches in large pieces or entirely.

CONSERVATION AND STORAGE:

- Maintain the product in its original package;
- Protect it from direct sunlight;
- Keep it in temperatures of 0 °C to 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- Product must be transported in temperatures between -5 °C to 50 °C (23 °F to 122 °F);
- Protect it from direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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