Potenza

IDRATA

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PHS DO BRASIL LTDA

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Código: IFU-008

Versão: 07

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79024

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA IDRATA.

Alternative Commercial Name: MOUTH MOISTURIZER.

Technical Product Name: PROTECTORS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Liquid petrolatum and polyethylene.

PRODUCT DESCRIPTION:

POTENZA IDRATA is a lubricating gel for topical application (skin or mucosa). The gel protects the application site against dryness that may cause soft tissue cracks. In addition, the product also creates an insulating film, preventing irritating or abrasive chemical agents from coming into direct contact with the skin/mucosa. POTENZA IDRATA has no medicine in its composition.

PRODUCT INDICATION:

POTENZA IDRATA is indicated for topical application (skin and mucosa) in patients undergoing dental procedures (avoiding lip dryness and providing comfort to the patient), as in cases where the lip retractor will be employed (photographic protocol, dental bleaching, etc.) or on occasions where the site is subject to eventual contact of irritating chemical agents.

POTENZA IDRATA can also be used in a rubber dam to facilitate the installation of absolute insulation on teeth.

PRECAUTIONS:

- Do not use in patients allergic to any of the components of the formulation;
- If an allergy is suspected, discontinue use immediately and refer the patient for medical attention;
- Do not reuse the packaging;
- Avoid contact with eyes.

CONTRAINDICATIONS:

- POTENZA IDRATA is contraindicated for individuals allergic to any of the ingredients in the formula.

WARNINGS:

- Observe the expiration date;
- Never use the product if it is past its expiration date;
- Keep out of reach of children;
- For dental use only;
- Before use, check the appearance of the product, if you notice any abnormalities, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;

- This product is manufactured for dental use only and should be handled according to the instructions for use;
- It is the responsibility of the professional to verify before use whether the product is compatible and suitable for the intended application;
- In case of a serious incident related to the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- For safe disposal of the DM after use, follow local regulations.

INSTRUCTIONS FOR USE:

1st STEP: Dispense the required amount of POTENZA IDRATA in a sterile Dappen pot. With the aid of a cotton ball, cotton swab or even with the gloved finger, capture a small amount of the gel, apply and spread on the desired area until it forms a thin film.

2nd STEP: The region that received the product will adopt smooth and bright (damp) appearance. For long-term procedures, it may be necessary to reapply the cream to maintain the moisturizing effect.

3rd STEP: If it is necessary to remove the gel from the applied area, make it with gauze or cotton rubbing gently.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORTATION CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C;
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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