Potenza SPECCHI ARGILLA

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PHS DO BRASIL LTDA

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Versão: 07

Potenza SPECCHI ARGILLA

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.79020

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, N° 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA SPECCHI ARGILLA.

Alternative Commercial Name: POLISHING PASTE.

Technical Product Name: PASTES FOR PROPHYLAXIS AND DENTAL POLISHING.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

White clay M635 (22,5 µm), aluminum oxide ALF 800 (18 µm), vehicle lubricant, thickeners and aroma.

PRODUCT DESCRIPTION:

POTENZA SPECCHI ARGILLA is a clay-based polishing paste, a material that combines different minerals of different hardness and with extra fine granulation. It has medium viscosity and thixotropic effect, grayish color and mentholated aroma.

PRODUCT INDICATION:

POTENZA SPECCHI ARGILLA is indicated for the final polishing of dental enamel, resins, ceramics, acrylics, metals and other restorative materials. It should be used in conjunction with felt discs in low rotation.

PRECAUTIONS:

- To avoid excessive heating of the dental surface, use light and intermittent pressure during polishing with the felt disc and paste;
- During application of the product, it is recommended that the professional and the patient wear safety goggles;
- Do not use in patients with allergy to any components of the formula.

CONTRAINDICATIONS:

- POTENZA SPECCHI ARGILLA is contraindicated for individuals with allergies to any of the components of the formula.

POSSIBLE SIDE EFFECTS:

- Ingestion of large amounts of the product can cause nausea;
- Discontinue use of the product in case of allergic reactions;
- In case of an allergic reaction, contact the doctor or pharmacist or manufacturer, in possession of this instruction for use.

WARNING:

- Respect the expiry date;
- Never use the product if it has expired;
- Keep out of reach of children;
- Dental use only;
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or improper handling;

- This product is intended for dental use only and should be handled according to the instructions for use;
- It is the professional's responsibility to ensure that the product is compatible and suitable for the desired use before use;
- In case of a serious incident related to the product, report it to the manufacturer and the competent authority;
- Do not reuse the empty packaging for other purposes;
- Reusing polishing pastes may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during previous use and grow under ideal storage conditions. This can present a risk of cross-contamination for the patient;
- Dental polishing pastes are formulated with abrasive particles and other active ingredients that may deteriorate after use;
- Repeated reuse of polishing pastes may cause a loss of proper consistency and texture, impairing the product's ability to perform effective polishing;
- To dispose of the DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

Polishing is the last step of dental restoration. Before polishing, make sure that the finishing and pre-polishing step has already been completed with the use of drills, sandpaper discs and polishing rubbers of different granulations. The quality of the restoration's brightness depends largely on the smoothness of the pre-polishing surface.

The POTENZA SPECCHI ARGILLA paste must be used in combination with a disposable felt disc coupled with low rotation. It is recommended to use thin and flexible discs.

1st STEP: Apply a small portion of the paste on the felt disc or directly on the surface of the restoration.

2nd STEP: In intermittent movements of slight pressure, cover the entire area of the restoration and the tooth to be polished. Usually, 2 minutes is enough for a thorough polishing.

3rd STEP: Make sure the brightness level is satisfactory. If not, consider repeating the pre-polishing process and then repeat the polishing maneuver.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C;
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 2 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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