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PHS DO BRASIL LTDA

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Código: IFU-009

Versão: 06

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number: 8.11992.70017

CINTERQUAL – Commerce Solutions

International Lda; Avenida Defensores de Chaves, Nº 4

Idea Spaces Office – 1000-117

Lisboa – Portugal +351 967 417 654

Product Commercial Name: POTENZA STAMPO EVA.

Alternative Commercial Name: FLEXIBLE PLATES FOR INDIVIDUAL TRAYS.

Technical Product Name: PLATE FOR MAKING TRAYS.

READ ALL THE INFORMATION IN THIS INSTRUCTION FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Vinyl ethylene acetate copolymer.

PRODUCT DESCRIPTION:

POTENZA STAMPO EVA is a plate made of EVA (vinyl ethylene acetate copolymer) of flexible, colorless and transparent appearance. The plate is 1.25 mm thick and is plasticized, having an approximate thickness of 1 mm.

PRODUCT INDICATION:

POTENZA STAMPO plates are indicated for the manufacture of individual trays, usually used for the application of home-use dental bleaching and drip for fluoridation.

The product is restricted to health professionals in the dental field (dentists and dental technicians).

PRECAUTIONS:

- Suspend use of the product in case of allergic reactions;
- Pay attention to the usage instructions before proceeding with the necessary operations;
- Avoid any irregular adaptations or improvisations;
- Check if the packaging has been tampered with. If it has been, return it immediately to the manufacturer;
- Keep the product in its original packaging;
- SINGLE-USE product, may be thermoplastified only once;
- The plaster model to be used for making the individual tray should be intact and have all structures well defined;
- Do not use the product if it is not intact;
- Keep the product away from heat and flames;
- In case of allergic reaction, contact a doctor, pharmacist, or manufacturer with this usage instruction.

CONTRAINDICATIONS:

- Do not use in patients with any sensitivity to the components of the formulation;

WARNING:

- Respect the expiry date;
- Never use the product if it has expired;
- Keep out of reach of children;

- Dental use only;
- Before use, check the appearance of the product. If you notice any anomalies, contact the manufacturer immediately;
- The manufacturer is not responsible for damage caused by improper use or incorrect handling;
- In case of a serious incident related to the product, report it to the manufacturer and the competent authority;
- This product was manufactured for dental use only and must be handled according to the instructions for use;
- It is the professional's responsibility to verify that the product is compatible and suitable for the intended use before use;
- Do not reuse the empty packaging for other purposes;
- Reusing the tray may lead to microbial contamination. Organic particles and bacteria may be introduced into the product during previous use and grow under ideal storage conditions. This can pose a risk of cross-contamination for the patient;
- To safely dispose of the DM after use, follow local legislation.

INSTRUCTIONS FOR USE:

1st STEP: Obtain the plaster model of the patient's dental arches through alginate molding and common plaster.

2nd STEP: Attach the plate to the thermoplasticizer machine and center the plaster model below it. With the machine on, wait for the plate to heat until it has a bulging of approximately 2 cm, evidencing its plasticization. Then lower the bulkhead and plate against the model by leaving the vacuum triggered for about 10 sec. After the plate adjusts to the model format, the assembly should be removed from the machine and let cool naturally reaching room temperature.

3rd STEP: After cooling the tray, remove it from the model. Cut out its edges leaving approximately 1 mm of coverage on the gingival margins. Make sure there are no sharp corners or burrs.

4th STEP: Store the tray over the model until delivery to the patient and instruct them on the need to sanitize it before use.

5th STEP: Instruct the patient on the use, cleaning, conservation and storage of the tray. During mouth use, instruct the patient not to eat or drink. After use, rinse under running water (at room temperature) until the gel is fully removed, then dry and store.

CONSERVATION AND STORAGE:

- Keep the product in its original packaging;
- Protect from the incidence of direct sunlight;
- Store the product at temperatures from 0 °C to 30 °C.

TRANSPORT CONDITIONS:

- The product can be transported at temperatures between -5 °C to 50 °C.
- Protect from the incidence of direct sunlight.

MANUFACTURING DATE: SEE PACKAGING.

EXPIRATION DATE: 5 YEARS AFTER THE MANUFACTURE DATE.

EXCLUSIVELY FOR USE IN DENTAL OFFICES. FOR USE ONLY BY DENTAL PROFESSIONALS.

MAKE SURE THE INSTRUCTION FOR USE MATCHES THE PRODUCT TO BE USED.

TO ACCESS A PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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