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PHS DO BRASIL LTDA

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Potenza

ATTACCO

INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Street Ottokar Doerffel, 1112, District: Atiradores, City: Joinville,

State: SC, Country: Brazil CEP Zip Code: 89203-212

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

EMAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)

Notification at ANVISA number 8.11992.70019

Product's Name: POTENZA ATTACCO HF.

Product's technical name: ACID CONDITIONERS.

READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.

**KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF
THE PRODUCT WITH YOUR PATIENT.**

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

Fluoridic Acid at 5 and 10%, Dye, Thickeners and Deionized Water.

PRODUCT'S DESCRIPTION:

POTENZA ATTACCO FLUORIDIC ACID CONDITIONER is found in concentrations of 5 and 10%. It is a thixotropic gel used for surface conditioning of dental porcelains. The conditioning allows the adhesion of Feldspar porcelain, ceramic glazing and fixation of orthodontic brackets and repairs of porcelain with composites.

PRODUCT'S INDICATION:

Indicated for conditioning of dental porcelain, treatment carried out before silanization and application of adhesives. It aims to increase adhesion in restoration processes. Conditioning is essential for an optimal interaction between resin cement and ceramic restoration to occur, thus favoring greater retention and resistance.

PRECAUTIONS:

- Keep away from sunlight;
- The contact of the product with eyes and/or skin can cause burning;
- Ingestion of big quantities of the product can cause nausea;
- The professional should instruct the patient not to swallow the product;
- If the patient swallows a big quantity of it and presents any side effect, the professional should take him/her to get medical assistance, along with this instruction's manual;
- This product is not for injection;
- Do not reuse the package.

CONTRAINDICATIONS:

POTENZA ATTACCO FLUORIDIC ACID CONDITIONER is contraindicated for individuals with history of allergic reactions to any of the components of the formula.

SIDE EFFECTS:

Contact with soft tissues can cause irritation and accidental ingestion of the product can cause fluoride poisoning.

WARNINGS:

- Do not use the product after its expiry date;
- This product is exclusive for dental use according to this document;
- It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use;
- Keep out of children’s reach;
- Before using the product, observe the aspect of it. If any variation is noticed, contact the manufacturer immediately;
- The manufacturer does not take responsibility for eventual damages caused by inadequate use or incorrect manipulation;
- Do not reuse the empty package;
- For proper product disposal, follow local legislation.

INSTRUCTIONS FOR USE:

The product must only be applied to the internal area of the prosthetic part, which will receive the cementing agent. The contact time of the product with the part varies depending on the type of substrate. Suggest following the ceramic manufacturer's guidelines for correct direction. The application time table shown in this document consists of a suggestion:

1st STEP: Apply POTENZA ATACCO HF only to the internal face of the ceramic piece (including edges) which will receive the cementing agent. Let the product act according to the recommended time;

SURFACE	Conditioning time for Potenza Attacco Hydrofluoric Acid 5%	Conditioning time for Potenza Attacco Hydrofluoric Acid 10%
	FELDSPATHIC PORCELAIN	-
GLASS CERAMICS REINFORCED WITH LEUCITE	-	60s
LITHIUM DISSILICATE- BASED GLASS CERAMICS	20s	-
ZIRCONIA	UNCONDITIONABLE	UNCONDITIONABLE

2nd STEP: Remove the acid by washing thoroughly for 2 minutes. This washing can be done in an ultrasonic tank or using an air/water spray. Then, dry the piece and observe a whitish and opaque appearance;

3rd STEP: Apply 35% phosphoric acid conditioner (ex: POTENZA ATTACCO 35%) to the entire surface that was previously conditioned with the hydrofluoric acid conditioner. Leave it to act for 1 minute and then wash copiously and dry the surface;

4th STEP: With the surface dry and clean, apply the silane agent, primer or universal adhesive, depending on your choice. Follow the adhesive steps for the part and teeth in accordance with the manufacturer's instructions for the respective materials.

CONSERVATION AND STORAGE:

- Keep the product in its original package;
- Protect it from direct sunlight;
- Keep it in temperatures between 0°C to 30°C (32o F to 86o F).

TRANSPORTATION CONDITIONS:

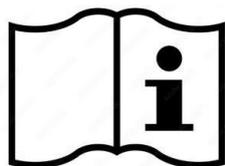
- Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F).
- Protect it from direct sunlight.

DATE OF MANUFACTURE: SEE PACKAGE.

EXPIRY DATE: 2 YEARS AFTER DATE OF MANUFACTURE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.

WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.

CONSULT THE INSTRUCTIONS
FOR USE

KEEP THE PRODUCT OUT OF THE
REACH OF CHILDREN

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