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**PHS DO BRASIL LTDA**

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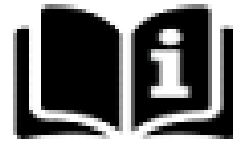
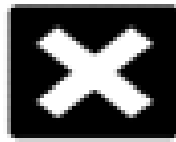
MI0013 – REV. 04

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**UNO & DUE**

**INSTRUCTIONS FOR USE FOR PROFESSIONALS**

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**Product Technical Name:** DENTAL POLISHING PASTES

**READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.**

**KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

### **BASIC COMPOSITION:**

Aluminum oxide ALF 240 (63 microns for UNO), aluminum oxide ALF 600 (23 microns for DUE), lubricating vehicle, thickeners and aroma.

## **PRECAUTIONS:**

- To avoid excessive heating of the dental surface, use light and intermittent pressure during polishing with the felt disc and paste.
- During application of the product, it is recommended that the professional and the patient wear safety goggles.
- Do not use in patients with allergy to the components of the formula.

## **CONTRAINDICATIONS**

POTENZA SPECCHI AL UNO DUE is contraindicated for people with a report of allergy to any of the components of the formula.

## **INSTRUCTIONS FOR USE:**

Polishing is the last step of dental restoration. Before polishing, make sure that the finishing and pre-polishing step has already been completed with the use of drills, sandpaper discs and polishing rubbers of different granulations. The quality of the restoration's brightness depends largely on the smoothness of the pre-polishing surface.

The **POTENZA SPECCHI AL UNO DUE** pastes must be used in combination with a disposable felt disc coupled with low rotation. It is recommended to use thin and flexible discs.

**Step 1:** For pre-polishing, apply a small portion of the POTENZA SPECCHI AL UNO paste on the felt disc or directly on the surface of the restoration.

**Step 2:** In intermittent movements of slight pressure, cover the entire area of the restoration and the tooth to be polished. Usually 2 minutes is enough for a thorough polishing.

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**Step 4:** Make sure the brightness level is satisfactory. If not, consider repeating the pre-polishing process and then repeat the polishing maneuver.

## **SIDE EFFECTS:**

- Ingestion of large amounts of the product can cause nausea.
- Discontinue use of the product in case of allergic reactions.
- In case of allergic reaction, refer the patient to emergency medical service with these Instructions for Use of the product at hand.

## **WARNING:**

- Respect the expiry date.
- Never use the product if it has expired.
- Keep out of reach of children.
- Dental use only.
- Before using, notice the appearance of the product, if you notice any anomaly, contact the manufacturer immediately.
- The manufacturer is not responsible for damage caused by improper use or improper handling.
- This product is manufactured for dental use only and must be handled in accordance with the instructions for use.
- It is the responsibility of the professional to prove, before the use of the product, whether this material is compatible and suitable for the intended use.
- Do not reuse the empty packaging for other purposes.
- To dispose of the product, please follow local legislation.

## **CONSERVATION AND STORAGE:**

- Keep the product in its original packaging.
- Protect from the incidence of direct sunlight.
- Store the product at temperatures from 0 °C to 30 °C.

## **TRANSPORT CONDITIONS:**

The product can be transported at temperatures between -5 °C and 50 °C. Protect from the incidence of direct sunlight.

## **MANUFACTURING DATE:**

SEE PACKAGING.

## **SHELF LIFE:**

TWO YEARS AFTER THE MANUFACTURING DATE.

**PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.**

**KEEP THE PACKAGING CLOSED WHEN NOT IN USE.**

**FOLLOW THE INSTRUCTIONS  
FOR USE CORRECTLY**

**KEEP OUT OF  
REACH OF  
CHILDREN**

**INTENDED FOR  
PRESCRIPTION USE  
ONLY**

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