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PHS DO BRASIL LTDA

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IRRITANTE

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INSTRUCTIONS FOR PROFESSIONAL USE

PHS DO BRASIL LTDA

Street Ottokar Doerffel, 1112, District: Atiradores,
City: Joinville, State: Santa Catarina, Country: Brasil,

Zip Code: 89203-212

CNPJ: 17.910.389/0001-05

BRAZILIAN INDUSTRY

E-MAIL: contato@phsdobrasil.com.br

Health Ministry's Operating Licence: X401M5619X7L(8.11992.7)

REGISTRY AT ANVISE NUMBER 8.11992.70023

TECHNICAL MANAGER: ÉLEN ALVES LOOS - CRQ/SC N. 13303733

Technical Support

0800-000-3780

www.phsdobrasil.com.br

PRODUCT'S NAME: POTENZA SPECCHI PRO

PRODUCT'S TECHNICAL NAME: POLISHING PASTES

READ ALL INFORMATION CAREFULLY BEFORE USING THE PRODUCT.

KEEP THIS DOCUMENT UNTIL THERE IS NO MORE INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

BASIC COMPOSITION:

Micronized diamond powder, lubricant, thickeners and aroma.

PRODUCT'S DESCRIPTION:

POTENZA SPECCHI PRO is a polishing paste produced with micronized diamond. It is available in extra-fine (0,5 to 1,5 microns), fine (2 to 4 microns) or medium (5 to 10 microns) granulation.

POTENZA SPECCHI PRO provides easy lubrication during polishing, has medium viscosity and does not alter its viscosity when heated. Combine lubricant properties with hard abrasives for enhanced polishing.

PRODUCT'S INDICATION:

POTENZA SPECCHI PRO is indicated for polishing of enamel, porcelain, composites and acrylic resins. Must be used in association with felt disks.

PRECAUTIONS

To avoid excessive heating of teeth, use the paste on a felt disk with low speed piece under gentle pressure under intermittent circular movements.

During application it is recommended to use security goggles for both professional and patient. Do not use the product in patients with allergy to any of the formula components.

CONTRAINDICATIONS:

POTENZA SPECCHI PRO is contraindicated for people with known allergy to any of the formula components.

SIDE EFFECTS:

Ingestion of big quantities of the product can cause nausea. Suspend its use in case of allergic reactions and take the patient to medical appointment along with the product's instructions for use.

WARNINGS:

Respect the expiry date of the product and do not use it after its expiry date. Maintain the product out of children's reach. Only for professional use. Before using, observe the aspect of the product, if any anomaly is noticed, contact the manufacturer immediately.

Manufacturer does not take responsibility for damage caused by inadequate use or incorrect manipulation. This product is intended for dental use only and should be used strictly according to this instructions document. It is of full responsibility of the professional to make sure, before using the product, if it is compatible to the desired use. Do not reuse the empty package. For proper product disposal, follow local legislation.

INSTRUCTIONS FOR USE:

1ST STEP: Prepare the surface to be polished. To obtain the best result, use finishing disks from coarse to extra-fine prior to polishing.

2ND STEP: Apply POTENZA SPECCHI PRO paste on the felt disk or directly on the surface to be polished. Use low speed with gentle and intermittent pressure to polish surfaces.

3RD STEP: Repeat procedure until obtaining a satisfying result.

CONSERVATION AND STORAGE:

Maintain the product in its original package. Protect it from direct sunlight. Keep it in temperatures of 0°C a 30°C (32o F to 86o F).

TRANSPORTATION CONDITIONS:

Product must be transported in temperatures between -5°C e 50°C (23o F to 122o F). Protect it from direct sunlight.

DATE OF MANUFACTURE See package

EXPIRY DATE: 2 YEARS AFTER DATE OF MANUFACTURE

SALES UNDER PRESCRIPTION (ACCORDING TO RDC N° 6 OF 2015).

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICE AND ONLY BY DENTAL PROFESSIONALS.

WHEN NOT IN USE, KEEP THE PRODUCT'S PACKAGE CLOSED.

**FOLLOW INSTRUCTIONS
FOR USE CORRECTLY**

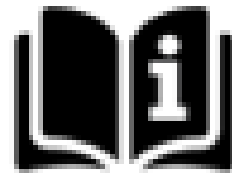
**INTENDED FOR USE
ONLY BY PRESCRIPTION**

**KEEP THE PRODUCT
OUT OF THE REACH OF
CHILDREN**

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