Potenza

CLOUD CLEAN

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Potenza

CLOUD CLEAN

INSTRUCTIONS FOR USE FOR PROFESSIONALS

Manufacturer and registration holder: PHS DO BRASIL LTDA

Ottokar Doerffel street, 1112, Atiradores, Joinville/SC, Brazil, ZIP CODE:

89203-212 | CNPJ: 17.910.389/0001-05

BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Operating permit MS: X401M5619X7L (8.11.992-7)

Notification at ANVISA nº 81199279035

Technical manager: ÉLEN LOOS LEHM - CRQ/SC NR. 13303733

Technical Support

0800-000-3780

www.phsgroup.com.br

Product trade name: POTENZA CLOUD CLEAN

Product Technical Name: Material for cleaning removable orthodontic appliances

READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTIONS FOR USE BEFORE APPLYING THE PRODUCT.

KEEP FOR CONSULTATION UNTIL THE PRODUCT IS COMPLETELY CONSUMED.

MAKE SURE THAT THE INSTRUCTIONS FOR USE AGREE WITH THE PRODUCT TO BE USED.

TO ACCESS THE PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

BASIC COMPOSITION:

Water, hydrogen peroxide, thickener, foamer, aroma.

PRODUCT DESCRIPTION:

POTENZA CLOUD CLEAN is a cleaning foam for removable oral devices (aligners, orthodontic retainers and mouthguards). Cleaning of mobile intraoral devices is done through the action of surfactant (sodium lauryl sulfate) and oxidative (hydrogen peroxide) agents.

PRODUCT INDICATION:

POTENZA CLOUD CLEAN is intended for cleaning removable intraoral devices, such as: invisible aligners, orthodontic retainers and mouthguards.

CONTRAINDICATIONS:

The product must not be used in the mouth.

Do not use the product on patients who are allergic to any of the components of the formula.

PRECAUTIONS:

Follow the instructions for use;

Do not use if the packaging is damaged;

Do not use after the expiration date;

Adult supervision is required for children under 12 years of age;

Avoid contact with eyes, skin or hair. If it occurs, wash abundantly and monitor until remission of any signs or symptoms;

Do not leave the product within the reach of children or people not qualified for its use;

Right after using the product, keep its packaging properly closed;

Before reusing the removable intraoral device, make sure to remove the entire content of the product by rinsing it under running water.

SIDE EFFECTS:

If the product is used as directed, no side effects are expected.

WARNINGS:

Avoid contact of the product with the mucosa, skin, eyes and hair; Store the product away from sunlight and heat sources; Do not leave the product within the reach of children or people not qualified for its use.

INSTRUCTIONS FOR USE:

To obtain the desired effects, place the intraoral device in a container (eg case, container) and apply the foam over it in order to cover it for 15 minutes. After this period of time, rinse the intraoral device under running water.

SIDE EFFECTS:

If the product is used as directed, no side effects are expected.

CONSERVATION AND STORAGE:

Store the product at room temperature (0°C to 30°C), protected from moisture and direct sunlight.

TRANSPORT CONDITIONS:

The product can be transported at temperatures between -5°C and 50°C.

MANUFACTURING DATE:

SEE PACKAGING.

EXPIRATION DATE:

24 (twenty-four) months after the manufacturing date.

KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

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