Potenza

CLOUD FLÚOR

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Potenza

CLOUD FLÚOR

INSTRUCTIONS FOR USE FOR PROFESSIONALS

Manufacturer and registration holder: PHS DO BRASIL LTDA

Ottokar Doerffel Street, 1112, Atiradores, Joinville/SC, Brazil,

CIP CODE: 89203-212

CNPJ: 17.910.389/0001-05 BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Operating permit MS: X401M5619X7L(8.11.992-7)

Notification at ANVISA nº81199270029

Technical manager: ÉLEN LOOS LEHM - CRQ/SC NR. 13303733

Technical Support

0800-000-3780

www.phsgroup.com.br

Product trade name: POTENZA CLOUD FLÚOR

Product Technical Name: Fluid with Fluoride for Dental Use

READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTIONS FOR USE BEFORE APPLYING THE PRODUCT.

KEEP FOR CONSULTATION UNTIL THE PRODUCT IS COMPLETELY CONSUMED.

KEEP THIS INSTRUCTION UNTIL THE PRODUCT HAS NO MORE INTERACTION WITH YOUR PATIENT.

MAKE SURE THAT THE INSTRUCTIONS FOR USE AGREE WITH THE PRODUCT TO BE USED.

TO ACCESS THE PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

BASIC COMPOSITION:

Water, sodium fluoride, thickener, foam stabilizer, flavor and preservative.

PRODUCT DESCRIPTION:

POTENZA CLOUD FLUOR is a fluoridated foam that contains fluoride as an active ingredient, acts as a remineralizing agent for dental enamel, binding to the calcium and phosphate naturally present in the tooth and forming fluorapatite, a material that is resistant to caries and helps prevent it.

PRODUCT INDICATION:

POTENZA CLOUD FLUOR is indicated for enamel fluoridation of patients at high risk of caries, including people with special needs and orthodontic patients, patients undergoing periodontal scaling and cases of dentin hypersensitivity.

CONTRAINDICATIONS:

Do not use the product on patients who are allergic to any of the components of the formula.

PRECAUTIONS:

Follow the instructions for use;
Do not use if the packaging is damaged;
Do not use if the validity of the product has expired;
Avoid contact with eyes, skin or hair. If it occurs, wash abundantly and monitor until remission of any signs or symptoms;
Do not leave the product within the reach of children or people not qualified for its use;
Right after using the product, keep its packaging properly closed.

SIDE EFFECTS:

If the product is used according to this IFU, no side effects are expected.

WARNINGS:

Avoid contact of the product to the mucosa, skin, eyes and hair; Store the product away from sunlight and heat sources; Do not leave the product within the reach of children or people not qualified for its use.

INSTRUCTIONS FOR USE:

1. Place a mouth opener to retract lips;
2. Perform teeth prophylaxis (pumice stone and water or prophylactic paste).
3. Position the patient in an upright position to reduce the risk of swallowing and use the saliva ejector to remove excess saliva.
4. Air dry teeth before application to avoid product dilution.
5. Apply the product on the teeth preferably with the aid of an adjustable tray for 1 to 4 minutes, depending on the swallowing control.
6. Instruct the patient not to eat, drink or rinse his mouth during the first 30 minutes after application.

CONSERVATION AND STORAGE:

Store the product at room temperature (0°C to 30°C), protected from moisture and direct sunlight.

CONDITIONS OF TRANSPORT:

The product can be transported at temperatures between -5°C and 50°C.

MANUFACTURING DATE:

SEE PACKAGING.

EXPIRATION DATE:

24 (twenty-four) months after the manufacturing date.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. ONLY FOR USE BY PROFESSIONALS IN THE DENTAL AREA. KEEP THE PACKAGING CLOSED WHEN NOT IN USE.

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