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PHS DO BRASIL LTDA

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Código: IFU-024

Versão: 05

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

PHS DO BRASIL LTDA

Guilherme Boldt Street, 210, District: Zona Industrial Norte, City: Joinville,

State: Santa Catarina, Country: Brazil - Zip Code: 89239-260

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: 8.11992.7 (X401M5619X7L)

Notification at ANVISA number: 8.11992.79037

Product Trade Name: POTENZA SILANE

Alternative Trade Name: BONDING AGENT

Product Technical Name: ADHESION AGENT / SILANIZATION AGENT

PLEASE READ ALL THE INFORMATION IN THESE INSTRUCTIONS FOR USE CAREFULLY BEFORE APPLYING THE PRODUCT.

KEEP THESE INSTRUCTIONS UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.

Technical Support

0800-000-3780

www.phsgroup.com.br

BASIC COMPOSITION:

3-Methacryloxypropyltrimethoxysilane, ethanol and acetic acid.

PRODUCT DESCRIPTION:

Silane is a coupling agent with a bifunctional molecule, used as an adhesion promoter between inorganic substrates and resin-based materials. It acts by forming chemical bonds between surfaces containing a glass phase and adhesive systems based on methacrylate monomers.

PRODUCT INDICATION:

Silane is indicated for the treatment of surfaces of inorganic materials, such as ceramics with a glass phase, ceromers, composite resins, and fiberglass posts, with the aim of increasing bond strength with adhesives and resin cements.

PRECAUTIONS:

- Close the bottle immediately after use to prevent evaporation;
- Flammable product contains ethanol;
- Keep out of the reach of children.

CONTRAINDICATIONS:

- POTENZA SILANE is contraindicated in patients with hypersensitivity to any component of the formulation.

POSSIBLE SIDE EFFECTS:

- When the steps in these instructions for use are followed in full, no adverse effects are expected with the use of this product;
- In case of adverse reaction, discontinue use and seek medical advice.

WARNING:

- Do not use the product after the expiration date;
- Keep out of reach of children;
- Product for dental use only, to be used by qualified professionals according to the instructions for use;
- Before use, check the appearance of the product. In case of abnormalities, do not use;
- Improper use or use outside the instructions may compromise product safety and performance;
- It is the responsibility of the professional to evaluate the suitability of the product for the intended use;
- In case of serious incidents related to the use of the product, report to the manufacturer and the competent authority;
- Do not reuse empty packaging for other purposes;

- Keep the packaging properly closed when not in use;
- Single-use product;
- In case of allergic reaction, discontinue use.

INSTRUCTIONS FOR USE:

A. Application on glass-phase ceramics (e.g., porcelain, lithium disilicate, ceromers)

Step 1: Condition the internal (bonding) surface of the restoration with hydrofluoric acid according to the time recommended by the material manufacturer (e.g., POTENZA ATTACCO HF 5% or 10%);

Step 2: Rinse thoroughly and dry with an air stream;

Step 3: If necessary, perform additional surface cleaning with phosphoric acid (e.g., POTENZA ATTACCO 35%) or by ultrasonic cleaning. Rinse and dry completely;

Step 4: Apply a thin layer of POTENZA SILANE over the entire bonding surface using a disposable microapplicator. Wait approximately 60 seconds;

Step 5: Promote solvent evaporation with a gentle air stream;

Note: After silane application, the surface should present a glossy appearance, indicating proper silanization;

Step 6: Proceed with the application of the adhesive system and resin cement according to the manufacturers' instructions and perform restoration cementation.

B. Application on composite resins and ceromers (repair or cementation)

Step 1: Perform mechanical surface preparation (e.g., sandblasting or roughening) according to the material manufacturer's recommendations;

Step 2: Clean the surface (e.g., water, alcohol, or ultrasonic bath) and dry completely;

Step 3: Apply a thin layer of POTENZA SILANE using a disposable microapplicator. Wait approximately 60 seconds;

Step 4: Promote solvent evaporation with a gentle air stream;

Step 5: Proceed with adhesive system and resin cement according to the manufacturers' instructions.

C. Application on fiberglass posts

Step 1: Clean the post according to the manufacturer's recommendations;

Step 2: Apply a thin layer of POTENZA SILANE over the entire post surface using a disposable microapplicator;

Step 3: Wait approximately 60 seconds and promote solvent evaporation with a gentle air stream free of oil and water;

Step 4: Proceed with cementation according to the adhesive system and resin cement instructions.

CONSERVATION AND STORAGE:

- Keep the product in the original packaging;
- Protect from direct sunlight exposure;
- Store the product at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

TRANSPORT CONDITIONS:

- The product may be transported at temperatures between 0 °C and 30 °C (32 °F to 86 °F).

MANUFACTURING DATE: SEE PACKAGE.

SHELF LIFE: 2 YEARS AFTER THE MANUFACTURING DATE.

PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES. FOR PROFESSIONAL DENTAL USE ONLY.

ENSURE THAT THESE INSTRUCTIONS FOR USE CORRESPOND TO THE PRODUCT BEING USED.

TO OBTAIN A PRINTED VERSION OF THE INSTRUCTIONS FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.

KEEP THE PACKAGE CLOSED WHEN NOT IN USE.

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