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**PHS DO BRASIL LTDA**

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**0800-000-3780**

[www.phsgroup.com.br](http://www.phsgroup.com.br)

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INSTRUCTIONS FOR USE FOR PROFESSIONALS

**PHS DO BRASIL LTDA**

Street Ottokar Doerffel, 1112, District: Atiradores, City: Joinville, State: Santa Catarina,

Country: Brasil Zip Code: 89203-212

CNPJ: 17.910.389/0001-05 - BRAZILIAN INDUSTRY

E-MAIL: contato@phsdo brasil.com.br

Health Ministry's Operating Licence: X401M5619X7L (8.11992.7)

Notification at ANVISA number 8.11992.79037

**Product Trade Name:** Potenza Silano.

**Product Technical Name:** Adhesion/Silanization Agent.

**READ CAREFULLY ALL THE INFORMATION IN THIS INSTRUCTION FOR USE BEFORE APPLYING THE PRODUCT.**

**KEEP THIS INSTRUCTION UNTIL THERE IS NO FURTHER INTERACTION OF THE PRODUCT WITH YOUR PATIENT.**

**Technical Support**

**0800-000-3780**

[www.phsgroup.com.br](http://www.phsgroup.com.br)

**BASIC COMPOSITION:** Hydrolyzing solution, 3-Methacryloxypropyltrimethoxysilane, and film former.

**PRODUCT'S DESCRIPTION:**

Silane has a bifunctional molecule that acts as adhesion promoters. It is a bonding agent with the purpose of increasing chemical adhesion between organic materials (adhesives containing methacrylic monomers) and inorganic materials, such as ceromers, composite resin, fiberglass posts and ceramics that contain a glass phase. The use of silane as a bonding agent is essential because the strength of the composite is determined by the adhesive strength between the filler and the matrix.

**PRODUCT'S INDICATION:**

Silane is intended to chemically alter the surface of inorganic materials such as ceromers, composite resin, fiberglass posts and ceramics that have a vitreous phase. Providing increased bond strength when interfaced with organic materials such as adhesives and resin cements.

**PRECAUTIONS:**

- Close the bottle immediately after use to prevent evaporation;
- Flammable product contains ethanol;
- Keep out of the reach of children.

**CONTRAINDICATIONS**

- SILANO is contraindicated for people with reported sensitivity to any of the product's components.

**SIDE EFFECTS:**

- When following the steps in this instruction for use in full, one does not expect to encounter possible side effects when using the product in question.

**WARNING:**

- Respect the expiration date. Never use the product if it has expired;
- Keep out of the reach of children;
- For dental use only;
- Before using, observe the appearance of the product; if you notice any anomaly, contact the manufacturer immediately;
- The manufacturer is not responsible for damages caused by inappropriate use or incorrect handling;
- This product was manufactured for dental use only and must be handled in accordance with the instructions for use;
- It is the professional's responsibility to check, before using the product, whether this product is compatible and suitable for the desired use;

- Do not reuse empty packaging for other purposes. To dispose of the product, follow local legislation;
- PRODUCT FOR EXCLUSIVE USE IN DENTAL OFFICES;
- ONLY FOR USE BY DENTAL PROFESSIONALS;
- KEEP THE PACKAGE CLOSED WHEN NOT IN USE;
- IMPORTANT: Suspend use of the product in case of allergic reactions.

## **INSTRUCTIONS FOR USE:**

**Indirect part cementation:** Before starting, make sure the part is eligible.

**1st STEP:** Condition the cementable portion of the piece (internal part) with hydrofluoric acid (e.g. Potenza Attacco HF 5% or 10%) for the time recommended by the manufacturer. Then, wash copiously and dry;

**2nd STEP:** Clean the piece with phosphoric acid (ex: Potenza Attacco 35%) and/or immersion in an ultrasonic tank. Then, dry the surface copiously;

**3rd STEP:** Apply Potenza Silano over the entire cementable area of the piece using a disposable micro applicator. Wait 1 minute and apply a new layer. Apply a light continuous jet of air for 15 seconds and wait another 1 minute;

**4th STEP:** Apply the adhesive and resin cement according to the manufacturer's instructions and cement the piece.

## **CONSERVATION AND STORAGE:**

- Keep the product in the original packaging;
- Protect from direct sunlight;
- Store the product at temperatures from 0°C to 30°C.

## **TRANSPORT CONDITIONS:**

- The product can be transported at temperatures between 0°C and 50°C.

**MANUFACTURING DATE: SEE PACKAGING.**

**SHELF LIFE: 2 YEARS AFTER THE MANUFACTURING DATE.**

**PRODUCT OF EXCLUSIVE USE IN DENTAL OFFICE. ONLY FOR USE OF DENTAL PROFESSIONALS.**

**MAKE SURE THAT THE INSTRUCTION FOR USE IS IN ACCORDANCE WITH THE PRODUCT TO BE USED.**

**TO ACCESS THE PRINTED VERSION OF THE INSTRUCTION FOR USE, CONTACT THE COMPANY'S TECHNICAL SUPPORT.**

**KEEP THE PACKAGING CLOSED WHEN NOT IN USE.**

Keep the Product Out of  
the Reach of Children

For Professional  
Use Only

Consult the Instructions  
for Use

Need for protection from  
direct light

Caution

Flammable

Need for protection against moisture  
during transport and storage

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